



HBV Longitudinal Panel Part Number: HBV6502

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECI ANTI-HIV ½ TEST DATE: 8/4/2014	ORTHO VITROS ECI ANTI-HCV TEST DATE: 8/4/2014	ABBOTT HBsAg TEST DATE: UNK	ORTHO VITROS ECI HBsAg TEST DATE: 8/4/2014	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBc TOTAL TEST DATE: 8/4/2014	ABBOTT ANTI-HBc IgM TEST DATE: UNK
6502-01	03/22/01	0.07	35.5	>40.8	4380	0.06	0.05	1.3
6502-02	04/05/01	0.08	35.2	>40.8	1540	0.06	0.04	1.4
6502-03	04/19/01	0.09	35.2	>40.8	441	0.07	0.05	1.2
6502-04	05/03/01	0.09	35.0	>40.8	75.0	0.07	0.05	0.9
6502-05	05/17/01	0.09	34.9	20.5	21.3	0.07	0.05	0.9
6502-06	05/31/01	0.09	34.6	8.2	8.10	0.06	0.06	0.9
6502-07	06/14/01	0.09	34.4	4.0	2.10	0.10	0.05	0.8
6502-08	06/27/01	0.09	34.3	1.7	0.85	0.08	0.06	0.8
6502-09	07/11/01	0.09	33.7	1.4	0.43	0.08	0.06	0.8
6502-10	07/25/01	0.08	33.7	1.57	0.25	0.04	0.04	0.89
6502-11	08/09/01	0.09	33.0	1.74	0.17	0.05	0.04	0.81
6502-12	08/22/01	0.08	32.6	1.68	0.19	0.03	0.06	0.88
6502-13	11/26/01	0.09	30.7	0.08	0.09	0.08	0.04	0.71

PANEL NUMBER	ORTHO VITROS ECI ANTI-HBc IgM TEST DATE: 8/4/2014	DIASORIN ANTI-HBe TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBe TEST DATE: 8/4/2014	ABBOTT ANTI-HBs QUANT TEST DATE: UNK	ORTHO VITROS ECI ANTI-HBs QUANT TEST DATE: 8/4/2014	ORTHO VITROS ECI HBeAg TEST DATE: 8/4/2014	DIASORIN HBeAg TEST DATE: UNK
6502-01	0.71	0.8	3.82	0.3	2.8	0.12	0.2
6502-02	0.71	0.7	4.72	0.3	1.7	0.15	0.2
6502-03	0.68	0.5	9.08	0.3	1.2	0.13	0.2
6502-04	0.51	0.5	9.68	0.3	1.8	0.12	0.2
6502-05	0.46	0.4	11.2	0.2	1.4	0.14	0.2
6502-06	0.44	0.3	14.0	0.4	3.2	0.14	0.3
6502-07	0.37	0.4	12.9	0.3	2.1	0.12	0.2
6502-08	0.36	0.3	14.0	0.3	2.0	0.12	0.2
6502-09	0.32	0.3	11.5	0.3	1.9	0.13	0.3
6502-10	0.30	0.1	11.2	0.35	1.6	0.13	0.3
6502-11	0.30	0.1	7.77	0.29	2.3	0.14	0.2
6502-12	0.30	0.1	9.37	0.37	1.6	0.12	0.3
6502-13	0.25	0.1	3.56	0.31	1.1	0.13	0.2

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6502 Rev03
Effective Date: 09/01/2021