



HBV Longitudinal Panel Part Number: HBV6507

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/4/2014	ORTHO VITROS ECi ANTI-HCV TEST DATE: 8/4/2014	ABBOTT HBsAg TEST DATE: N/A	ORTHO VITROS ECi HBsAg TEST DATE: 8/4/2014	ABBOTT ANTI-HBc TOTAL TEST DATE: N/A	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/4/2014	ABBOTT ANTI-HBc IgM TEST DATE: N/A
6507-01	04/16/01	0.06	25.5	>40.8	5240	0.04	0.02	>6.9
6507-02	04/30/01	0.06	26.4	>40.8	7400	0.06	0.05	>6.9
6507-03	05/14/01	0.12	19.7	>40.8	2640	0.07	0.01	6.0
6507-04	05/28/01	0.12	17.2	>40.8	2130	0.08	0.01	5.9
6507-05	06/11/01	0.08	16.4	0.04	0.12	0.08	0.03	4.0
6507-06	06/25/01	0.07	14.1	0.06	0.13	0.09	0.04	3.9
6507-07	07/09/01	0.08	11.7	0.04	0.12	0.09	0.02	3.7
6507-08	07/23/01	0.06	11.1	0.15	0.15	0.04	0.04	3.8
6507-09	08/06/01	QNS	QNS	0.15	QNS	0.03	QNS	2.3
6507-10	08/21/01	0.08	34.8	0.14	0.12	0.03	0.03	2.7
6507-11	09/03/01	0.08	35.0	0.09	0.13	0.04	0.03	2.6
6507-12	09/17/01	0.07	35.0	0.02	0.11	0.04	0.03	2.3
6507-13	11/27/01	0.06	34.6	0.04	0.13	0.08	0.02	1.6
6507-14	12/11/01	0.08	34.4	0.09	0.11	0.09	0.03	1.4

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/4/2014	DIASORIN ANTI-Hbe TEST DATE: N/A	ORTHO VITROS ECi ANTI-Hbe TEST DATE: 8/4/2014	ABBOTT ANTI-HBs QUANT TEST DATE: N/A	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/4/2014	ORTHO VITROS ECi HBeAg TEST DATE: 8/4/2014	DIASORIN HBeAg TEST DATE: N/A
6507-01	3.85	0.6	0.34	0.3	4.3	4.01	0.7
6507-02	3.80	0.6	0.46	0.3	3.4	0.28	0.2
6507-03	2.52	0.6	1.04	0.2	3.9	0.12	0.1
6507-04	2.58	0.7	0.59	0.2	4.1	0.12	0.2
6507-05	2.00	0.3	3.43	0.4	4.5	0.12	0.3
6507-06	1.90	0.2	10.2	0.5	8.0	0.12	0.1
6507-07	1.74	0.2	15.9	0.5	13.2	0.12	0.2
6507-08	1.66	0.1	16.6	1.9	28.2	0.13	0.3
6507-09	QNS	0.3	QNS	0.55	QNS	QNS	0.3
6507-10	1.45	0.1	26.3	5.3	74.2	0.18	0.4
6507-11	1.39	0.1	27.2	4.8	109	0.13	0.4
6507-12	1.28	0.1	28.9	4.4	113	0.11	0.4
6507-13	0.97	0.03	27.5	17.8	374	0.12	0.3
6507-14	0.89	0.04	25.7	19.0	410	0.12	0.3

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6507 Rev03
Effective Date: 09/01/2021