



**HBV Longitudinal Panel
Part Number: HBV6509**

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½	ORTHO VITROS ECi ANTI-HCV	ABBOTT HBsAg	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	ABBOTT ANTI-HBc IgM
		TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: N/A	TEST DATE: 8/4/2014	TEST DATE: N/A	TEST DATE: 8/4/2014	TEST DATE: N/A
6509-01	5/16/01	0.04	0.09	>40.8	5110	0.04	0.12	>6.9
6509-02	5/28/01	0.08	0.04	>40.8	6070	0.08	0.09	>6.9
6509-03	6/13/01	0.08	0.01	>40.8	7850	0.05	0.03	>6.9
6509-04	6/27/01	0.08	0.05	>40.8	7660	0.05	0.11	>6.9
6509-05	7/11/01	0.02	0.01	>40.8	6860	0.08	0.17	6.8
6509-06	7/25/01	0.04	0.02	5.8	0.31	0.04	0.06	4.3
6509-07	8/8/01	0.08	0.04	0.04	0.04	0.04	0.08	2.7
6509-08	8/22/01	0.07	0.04	0.09	0.04	0.04	0.06	2.8
6509-09	9/5/01	0.07	0.05	0.15	0.04	0.56	0.07	2.1
6509-10	9/19/01	0.06	0.04	0.49	0.04	0.05	0.07	2.1
6509-11	10/3/01	0.07	0.05	0.05	0.04	0.05	0.06	1.9
6509-12	10/16/01	0.08	0.05	0.02	0.05	0.07	0.03	2.0

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN ANTI-Hbe	ORTHO VITROS ECi ANTI-Hbe	ABBOTT ANTI-HBs QUANT	ORTHO VITROS ECi ANTI-HBs QUANT	ORTHO VITROS ECi HBeAg	DIASORIN HBeAg
	TEST DATE: 8/4/2014	TEST DATE: N/A	TEST DATE: 8/4/2014	TEST DATE: N/A	TEST DATE: 8/4/2014	TEST DATE: 8/4/2014	TEST DATE: N/A
6509-01	3.57	0.9	0.20	0.3	0.4	0.68	1.1
6509-02	3.79	1.0	0.29	0.3	QNS	0.52	0.4
6509-03	3.78	1.3	0.34	0.3	0.6	0.19	0.4
6509-04	3.90	1.3	0.37	0.2	1.7	0.30	0.4
6509-05	2.73	0.5	9.16	0.3	1.7	0.08	0.2
6509-06	1.72	0.2	10.1	0.3	3.4	0.07	0.14
6509-07	1.62	0.1	8.69	0.93	4.7	0.08	0.09
6509-08	1.55	0.1	10.3	1.9	17.0	0.07	0.03
6509-09	1.44	0.2	8.74	2.4	34.2	0.09	0.08
6509-10	1.37	0.2	4.28	6.9	91.5	0.09	0.05
6509-11	1.22	0.2	6.29	8.1	111	0.10	0.08
6509-12	1.30	0.2	7.75	8.1	187	0.12	0.05

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com