



## HBV Longitudinal Panel Part Number: HBV6512

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.  
For Research Use Only. Not for use in diagnostic procedures.*

| PANEL NUMBER | DATE OF DRAW | ORTHO VITROS ECi ANTI-HIV ½<br>TEST DATE: 8/8/2014 | ORTHO VITROS ECi ANTI-HCV<br>TEST DATE: 8/8/2014 | ABBOTT HBsAg<br>TEST DATE: UNK | ORTHO VITROS ECi HBsAg<br>TEST DATE: 8/8/2014 | ABBOTT ANTI-HBc TOTAL<br>TEST DATE: UNK | ORTHO VITROS ECi ANTI-HBc TOTAL<br>TEST DATE: 8/8/2014 | ABBOTT ANTI-HBc IgM<br>TEST DATE: UNK |
|--------------|--------------|----------------------------------------------------|--------------------------------------------------|--------------------------------|-----------------------------------------------|-----------------------------------------|--------------------------------------------------------|---------------------------------------|
| 6512-01      | 05/30/01     | 0.08                                               | 0.19                                             | >40.8                          | 9870                                          | 0.07                                    | 0.03                                                   | >6.9                                  |
| 6512-02      | 06/13/01     | 0.08                                               | 0.16                                             | >40.8                          | 2220                                          | 0.08                                    | 0.06                                                   | >6.9                                  |
| 6512-03      | 06/27/01     | 0.09                                               | 0.11                                             | 18.8                           | 44.9                                          | 0.09                                    | 0.03                                                   | >6.9                                  |
| 6512-04      | 07/11/01     | 0.08                                               | 0.12                                             | 1.6                            | 2.41                                          | 0.11                                    | 0.04                                                   | >6.9                                  |
| 6512-05      | 07/25/01     | 0.06                                               | 0.13                                             | 0.3                            | 0.06                                          | 0.03                                    | 0.03                                                   | >6.5                                  |
| 6512-06      | 08/08/01     | 0.09                                               | 0.07                                             | 0.1                            | 0.05                                          | 0.03                                    | 0.02                                                   | 5.9                                   |
| 6512-07      | 08/22/01     | 0.09                                               | 0.13                                             | 0.1                            | 0.07                                          | 0.04                                    | 0.02                                                   | 5.9                                   |
| 6512-08      | 09/05/01     | 0.09                                               | 0.11                                             | 0.1                            | 0.06                                          | 0.04                                    | 0.03                                                   | 5.1                                   |
| 6512-09      | 09/19/01     | 0.08                                               | 0.10                                             | 0.8                            | 0.05                                          | 0.03                                    | 0.02                                                   | 4.7                                   |
| 6512-10      | 10/03/01     | 0.08                                               | 0.27                                             | 0.02                           | 0.06                                          | 0.03                                    | 0.03                                                   | 4.1                                   |
| 6512-11      | 10/17/01     | 0.09                                               | 0.07                                             | 0.1                            | 0.05                                          | 0.05                                    | 0.02                                                   | 5.8                                   |
| 6512-12      | 10/30/01     | 0.09                                               | 0.10                                             | 0.02                           | 0.05                                          | 0.04                                    | 0.02                                                   | 5.4                                   |
| 6512-13      | 12/11/01     | 0.07                                               | 0.04                                             | 0.1                            | 0.06                                          | 0.06                                    | 0.02                                                   | 4.4                                   |
| 6512-14      | 12/25/01     | 0.09                                               | 0.07                                             | 0.1                            | 0.09                                          | 0.06                                    | 0.03                                                   | 4.1                                   |
| 6512-15      | 01/08/02     | 0.10                                               | 0.09                                             | 0.1                            | 0.05                                          | 0.05                                    | 0.02                                                   | 4.1                                   |

| PANEL NUMBER | ORTHO VITROS ECi ANTI-HBc IgM<br>TEST DATE: 8/8/2014 | DIASORIN ANTI-HBe<br>TEST DATE: UNK | ORTHO VITROS ECi ANTI-HBe<br>TEST DATE: 8/8/2014 | ABBOTT ANTI-HBs QUANT<br>TEST DATE: UNK | ORTHO VITROS ECi ANTI-HBs QUANT<br>TEST DATE: 8/8/2014 | ORTHO VITROS ECi HBeAg<br>TEST DATE: 8/8/2014 | DIASORIN HBeAg<br>TEST DATE: UNK |
|--------------|------------------------------------------------------|-------------------------------------|--------------------------------------------------|-----------------------------------------|--------------------------------------------------------|-----------------------------------------------|----------------------------------|
| 6512-01      | 3.80                                                 | 0.6                                 | 0.53                                             | 0.3                                     | 1.5                                                    | 0.72                                          | 0.5                              |
| 6512-02      | 3.83                                                 | 0.9                                 | 0.27                                             | 0.5                                     | 1.6                                                    | 1.26                                          | 0.4                              |
| 6512-03      | 3.57                                                 | 1.0                                 | 0.38                                             | 0.3                                     | 1.5                                                    | 0.20                                          | 0.2                              |
| 6512-04      | 3.34                                                 | 1.0                                 | 0.50                                             | 0.3                                     | 1.2                                                    | 0.13                                          | 0.2                              |
| 6512-05      | 3.08                                                 | 0.3                                 | 0.85                                             | 0.4                                     | 1.1                                                    | 0.12                                          | 0.3                              |
| 6512-06      | 2.65                                                 | 0.4                                 | 0.90                                             | 0.5                                     | 2.8                                                    | 0.13                                          | 0.2                              |
| 6512-07      | 2.73                                                 | 0.3                                 | 1.71                                             | 0.6                                     | 2.5                                                    | 0.13                                          | 0.3                              |
| 6512-08      | 2.60                                                 | 0.2                                 | 1.73                                             | 0.7                                     | 3.1                                                    | 0.13                                          | 0.2                              |
| 6512-09      | 2.48                                                 | 0.2                                 | 1.73                                             | 0.9                                     | 3.7                                                    | 0.12                                          | 0.3                              |
| 6512-10      | 2.31                                                 | 0.2                                 | 1.45                                             | 1.1                                     | 5.9                                                    | 0.14                                          | 0.2                              |
| 6512-11      | 2.51                                                 | 0.2                                 | 2.70                                             | 1.1                                     | 7.1                                                    | 0.12                                          | 0.3                              |
| 6512-12      | 2.40                                                 | 0.2                                 | 1.83                                             | 1.2                                     | 9.7                                                    | 0.13                                          | 0.2                              |
| 6512-13      | 2.20                                                 | 0.2                                 | 1.27                                             | 1.6                                     | 8.8                                                    | 0.15                                          | 0.4                              |
| 6512-14      | QNS                                                  | 0.3                                 | 1.03                                             | 1.5                                     | QNS                                                    | 0.13                                          | 0.2                              |
| 6512-15      | 2.11                                                 | 0.4                                 | 0.78                                             | 1.31                                    | 8.0                                                    | 0.12                                          | 0.3                              |

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6512 Rev03  
Effective Date: 03/24/2022