



HBV Longitudinal Panel Part Number: HBV6516

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS Eci ANTI-HIV ½ TEST DATE: 8/8/2014	ORTHO VITROS Eci ANTI-HCV TEST DATE: 8/8/2014	ABBOTT HBsAg TEST DATE: UNK	ORTHO VITROS Eci HBsAg TEST DATE: 8/8/2014	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBc TOTAL TEST DATE: 8/8/2014	ABBOTT ANTI-HBc IgM TEST DATE: UNK
6516-01	06/19/01	0.10	0.11	0.06	0.07	0.05	0.06	>6.4
6516-02	07/03/01	0.07	0.09	0.06	0.12	0.04	0.02	>6.4
6516-03	07/17/01	0.06	0.10	0.08	0.07	0.02	0.02	>6.0
6516-04	07/31/01	0.07	0.08	0.08	0.05	0.03	0.02	>6.0
6516-05	08/14/01	0.08	0.07	0.02	0.05	0.04	0.02	4.9
6516-06	08/28/01	0.07	0.08	0.04	0.06	0.09	0.03	3.9
6516-07	09/14/01	0.07	0.09	0.05	0.07	0.06	0.02	3.3
6516-08	09/25/01	0.08	0.09	0.04	0.06	0.09	0.02	3.0
6516-09	10/09/01	0.06	0.18	0.05	0.08	0.05	0.02	3.4
6516-10	11/08/01	0.05	0.24	0.08	0.08	0.09	0.03	2.6
6516-11	10/23/01	0.07	0.20	0.08	0.06	0.06	QNS	2.4
6516-12	11/20/01	0.07	0.22	0.3	0.05	0.05	0.03	2.1

PANEL NUMBER	ORTHO VITROS Eci ANTI-HBc IgM TEST DATE: 8/8/2014	DIASORIN ANTI-HBe TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBe TEST DATE: 8/8/2014	ABBOTT ANTI-HBs QUANT TEST DATE: UNK	ORTHO VITROS Eci ANTI-HBs QUANT TEST DATE: 8/8/2014	ORTHO VITROS Eci HBeAg TEST DATE: 8/8/2014	DIASORIN HBeAg TEST DATE: UNK
6516-01	3.81	0.1	18.8	1.8	11.2	0.12	0.2
6516-02	3.73	0.8	17.3	1.0	7.3	0.14	0.2
6516-03	3.48	0.02	16.4	2.6	8.5	0.14	0.1
6516-04	2.78	0.01	29.9	3.1	13.9	0.12	0.2
6516-05	2.46	0.01	34.1	2.7	20.4	0.11	0.1
6516-06	2.17	0.01	25.8	5.4	27.5	0.12	0.2
6516-07	1.87	0.01	36.3	8.1	52.6	0.11	0.2
6516-08	1.87	0.01	30.6	7.1	68.7	0.12	0.2
6516-09	1.66	0.02	25.9	8.9	94.9	0.12	0.1
6516-10	1.42	0.01	30.3	9.1	97.3	0.12	0.2
6516-11	1.43	0.01	QNS	8.1	119	0.11	0.1
6516-12	1.34	0.01	39.9	9.7	141	0.13	0.2

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6516 Rev02
Effective Date: 09/01/2021