



HBV Longitudinal Panel Part Number: HBV6518

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECI ANTI-HIV ½ TEST DATE: UNK	ORTHO VITROS ECI ANTI-HCV TEST DATE: UNK	ABBOTT HBsAg TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ABBOTT ANTI-HBc IgM TEST DATE: UNK	DIASORIN ANTI-HBe TEST DATE: UNK	ABBOTT ANTI-HBs QUANT TEST DATE: UNK	DIASORIN HBeAg TEST DATE: UNK
6518-01	07/03/01	0.06	0.30	>40.8	0.05	>6.4	0.40	0.3	0.3
6518-02	07/17/01	0.06	0.24	>40.8	0.06	>6.4	0.30	0.3	0.3
6518-03	07/31/01	0.06	0.36	>37.7	0.03	>6.9	0.07	0.2	0.2
6518-04	08/14/01	0.07	0.26	>37.7	0.05	6.1	0.02	0.2	0.2
6518-05	08/28/01	0.07	0.69	>37.7	0.07	4.8	0.02	0.2	0.1
6518-06	09/11/01	0.07	19.4	>37.7	0.06	3.3	0.01	0.2	0.2
6518-07	09/25/01	0.07	21.1	>37.7	0.05	4.3	0.02	0.3	0.1
6518-08	10/12/01	0.07	21.0	0.4	0.05	3.1	0.01	0.3	0.2
6518-09	10/25/01	0.06	21.9	0.4	0.03	3.8	0.02	0.5	0.1
6518-10	11/06/01	0.06	21.8	0.1	0.06	3.1	0.01	0.6	0.2
6518-11	11/22/01	0.06	23.3	0.1	0.04	2.9	0.02	1.3	0.1
6518-12	12/04/01	0.06	24.0	0.1	0.04	2.9	0.01	1.4	0.2

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6518 Rev03
Effective Date: 09/01/2021