



HBV Longitudinal Panel Part Number: HBV6528

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

***Donor is co-infected with Hepatitis C.**

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/8/2014	ORTHO VITROS ECi ANTI-HCV TEST DATE: 8/8/2014	ABBOTT HBsAg TEST DATE: UNK	ORTHO VITROS ECi HBsAg TEST DATE: 8/8/2014	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/8/2014	ABBOTT ANTI-HBc IgM TEST DATE: UNK
6528-01	09/18/01	0.08	0.69	>37.7	4560	0.06	0.16	>6.9
6528-02	10/02/01	0.07	1.10	>37.7	9000	0.04	0.23	6.6
6528-03	10/17/01	0.08	1.07	>37.7	9780	0.04	0.09	4.7
6528-04	10/30/01	0.08	1.10	>37.7	6790	0.05	0.02	3.8
6528-05	11/13/01	0.07	1.06	>37.7	1460	0.04	0.01	3.4
6528-06	11/28/01	0.07	1.02	17.2	7.41	0.06	0.02	3.1
6528-07	12/11/01	0.08	0.95	0.7	0.08	0.05	0.02	2.6
6528-08	12/26/01	0.08	0.92	0.02	0.05	0.06	0.02	2.5
6528-09	01/08/02	0.07	0.95	0.02	0.05	0.05	0.03	2.2
6528-10	01/22/02	0.10	0.85	0.02	0.05	0.06	0.11	1.9
6528-11	02/05/02	0.08	0.99	0.02	0.05	0.05	0.02	1.8
6528-12	02/19/02	0.09	0.95	0.03	0.06	0.06	0.03	2.2

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/8/2014	DIASORIN ANTI-HBe TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/8/2014	ABBOTT ANTI-HBs QUANT TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/8/2014	ORTHO VITROS ECi HBeAg TEST DATE: 8/8/2014	DIASORIN HBeAg TEST DATE: UNK
6528-01	3.39	0.30	0.83	0.2	1.3	0.31	0.75
6528-02	2.92	0.10	13.6	0.2	2.0	0.12	0.03
6528-03	2.42	0.06	24.4	0.3	1.7	0.12	0.01
6528-04	2.16	0.03	27.3	0.2	2.2	0.11	0.08
6528-05	2.01	0.01	27.5	0.3	1.7	0.13	0.13
6528-06	1.90	0.02	31.8	3.8	7.0	0.11	0.08
6528-07	1.76	0.01	32.4	4.1	9.1	0.11	0.08
6528-08	1.57	0.02	29.2	3.2	14.1	0.13	0.03
6528-09	1.48	0.01	29.2	3.2	23.3	0.12	0.08
6528-10	1.85	0.01	28.8	3.7	96.8	0.11	0.02
6528-11	1.42	0.01	26.3	5.3	83.1	0.12	0.02
6528-12	1.31	0.03	26.5	5.8	121	0.13	0.33

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6528 Rev03
Effective Date: 09/01/2021