



HBV Longitudinal Panel Part Number: HBV6529

*These reagents are not a substitute for the mandatory positive and negative control reagents provided with licensed test kits.
For Research Use Only. Not for use in diagnostic procedures.*

PANEL NUMBER	DATE OF DRAW	ORTHO VITROS ECi ANTI-HIV ½ TEST DATE: 8/8/2014	ORTHO VITROS ECi ANTI-HCV TEST DATE: 8/8/2014	ABBOTT HBsAg TEST DATE: UNK	ORTHO VITROS ECi HBsAg TEST DATE: 8/8/2014	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/8/2014	ABBOTT ANTI-HBc IgM TEST DATE: UNK
6529-01	10/08/01	0.07	0.07	2.5	0.12	0.03	0.05	>4.7
6529-02	10/22/01	0.07	0.07	0.07	0.09	0.04	0.13	>4.7
6529-03	11/05/01	0.08	0.07	0.1	0.08	0.06	0.11	>4.7
6529-04	11/19/01	0.07	0.07	0.1	0.09	0.04	0.02	>4.7
6529-05	12/03/01	0.09	0.06	0.07	0.08	0.05	0.02	>4.7
6529-06	12/18/01	0.08	0.04	0.05	0.08	0.05	0.02	>4.7
6529-07	01/01/02	0.09	0.07	0.04	0.09	0.06	QNS	>4.7
6529-08	01/14/02	0.07	0.06	0.2	0.12	0.06	0.02	4.6
6529-09	01/28/02	0.08	0.05	0.1	0.08	0.06	0.01	4.3
6529-10	02/11/02	0.08	0.06	0.07	0.08	0.06	0.01	4.3
6529-11	02/25/02	0.08	0.06	0.1	0.08	0.06	0.01	3.8
6529-12	03/09/02	0.08	0.06	0.07	0.07	0.07	0.01	3.5

PANEL NUMBER	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/8/2014	DIASORIN ANTI-HBe TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/8/2014	ABBOTT ANTI-HBs QUANT TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/8/2014	ORTHO VITROS ECi HBeAg TEST DATE: 8/8/2014	DIASORIN HBeAg TEST DATE: UNK
6529-01	3.70	0.6	0.28	0.6	3.9	0.39	1.29
6529-02	3.83	0.4	0.45	0.5	4.4	0.13	0.29
6529-03	3.56	0.2	0.68	0.5	3.5	0.14	0.13
6529-04	3.17	0.1	1.44	0.5	4.8	0.13	0.14
6529-05	2.90	0.8	4.04	0.6	8.8	0.12	0.04
6529-06	2.57	0.09	4.65	0.9	11.4	0.13	0.30
6529-07	2.35	0.03	12.5	2.0	25.6	0.14	0.10
6529-08	2.47	0.03	15.8	3.1	54.9	0.17	0.16
6529-09	QNS	0.03	24.2	3.5	QNS	0.12	0.10
6529-10	2.30	0.03	QNS	5.9	160	0.14	0.19
6529-11	2.17	0.03	29.4	5.6	201	0.12	0.04
6529-12	QNS	0.07	25.4	9.3	QNS	0.12	0.39

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antyia.com

PIHBV6529 Rev02
Effective Date: 09/02/2021