

ZeptoMetrix®

HBV Seroconversion Panel Donor No. 65449 Part Number: HBV11000

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL NUMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	GENETIC SYSTEMS HBsAg (2.0)	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	Abbott MUREX HBsAg Version 3	ORTHO HBsAg ELISA Testsystem 3	ROCHE ELECSYS HBsAg
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11000-01	10/30/97	3	1210	0.449	0.200	0.094	0.68	0.414	0.450
11000-02	11/02/97	3	650	0.449	0.200	0.075	0.67	0.173	0.510
11000-03	11/11/97	5	2772	0.449	0.260	0.189	0.83	0.147	0.530
11000-04	11/13/97	4	3739	0.577	0.470	0.472	1.02	0.388	0.700
11000-05	11/18/97	4	4817	0.513	0.520	0.887	1.24	0.441	0.870
11000-06	11/20/97	5	6064	0.285	0.800	0.981	1.55	0.655	1.200
11000-07	11/25/97	3	13892	0.256	1.390	2.547	2.70	1.191	1.930
11000-08	11/28/97	5	12964	0.641	1.940	2.849	3.23	1.750	2.850
11000-09	12/02/97	7	36539	3.462	9.760	7.189	10.51	7.799	9.530

PANEL NUMBER	ORTHO VITROS ECi HBsAg	ABBOTT PRISM HBsAg	DIASORIN HBsAg	DADE BEHRING Enzygnost HBsAg 5.0	ORTHO VITROS HBsAg Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack	SORIN ETI-MAK 4	ORTHO VITROS ECi ANTI-HBs QUANT	DIASORIN ANTI-HBs QUANT
	TEST DATE: 8/14/2015	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/14/2015	TEST DATE: UNK
11000-01	0.17	0.49	0.195	0.280	0.140	0.550	0.330	33.3	4.11
11000-02	0.16	0.52	0.176	0.280	0.130	0.610	0.360	26.5	3.16
11000-03	0.14	1.11	0.155	0.280	0.120	0.640	0.430	16.4	2.34
11000-04	0.15	1.23	0.184	0.320	0.180	0.780	0.500	12.8	1.52
11000-05	0.17	1.51	0.168	0.370	0.240	0.900	0.620	10.1	1.12
11000-06	0.17	2.05	0.187	0.480	0.390	1.030	0.690	8.4	0.93
11000-07	0.33	4.36	0.201	1.530	0.950	1.820	1.520	3.9	0.39
11000-08	0.77	5.24	0.254	1.660	1.600	2.390	2.110	2.0	0.21
11000-09	10.1	14.37	0.885	9.700	11.080	5.570	11.010	0.5	0.05

PANEL NUMBER	ORTHO VITROS ECi HBeAg	ABBOTT HBeAg (Co/S)	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	DIASORIN HBc IgM	ABBOTT ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HBc TOTAL	DIASORIN ANTI-HBc TOTAL	ORTHO VITROS ECi ANTI-HAV TOTAL
	TEST DATE: 8/14/2015	TEST DATE: UNK	TEST DATE: 8/14/2015	TEST DATE: 8/14/2015	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/14/2015	TEST DATE: UNK	TEST DATE: 8/14/2015
11000-01	0.23	Not Tested	0.15	0.02	0.2	Not Tested	2.87	4.33	3.02
11000-02	0.24	Not Tested	0.16	0.01	0.2	Not Tested	3.00	4.27	3.17
11000-03	0.25	Not Tested	0.18	0.02	0.2	Not Tested	2.77	4.93	2.93
11000-04	0.23	Not Tested	0.15	0.01	0.2	Not Tested	2.75	3.33	2.91
11000-05	0.25	Not Tested	0.15	0.03	0.2	Not Tested	2.32	4.67	2.24
11000-06	0.24	Not Tested	0.15	0.02	0.2	Not Tested	2.47	4.53	2.43
11000-07	0.26	Not Tested	0.15	0.01	0.2	Not Tested	2.92	4.47	2.78
11000-08	0.28	Not Tested	0.16	0.01	0.2	Not Tested	3.04	4.33	3.00
11000-09	0.35	0.338	0.18	0.01	0.2	0.395	3.05	4.07	3.16

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11000 Rev03
Effective Date: 08/11/2021