



**HBV Seroconversion Panel Donor No. 65524
Part Number: HBV11001**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45u/L) TEST DATE: 3/15/1999 | ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK | ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005 | ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK | ROCHE Cobas Core HBsAg II EIA TEST DATE: UNK | DIASORIN HBsAg TEST DATE: UNK | GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK |
|--------------|--------------|--|--|--|--|---|----------------------------------|---|
| 11001-01 | 10/12/97 | 2 | NOT DETECTED | Not Tested | 0.300 | 0.200 | 0.206 | 0.513 |
| 11001-02 | 10/18/97 | 4 | NOT DETECTED | Not Tested | 0.190 | 0.180 | 0.209 | 0.577 |
| 11001-03 | 11/14/97 | 7 | 73 | NEG | 0.190 | 0.190 | 0.163 | 0.449 |
| 11001-04 | 11/16/97 | 4 | 156 | 3280 | 0.080 | 0.230 | 0.246 | 0.577 |
| 11001-05 | 11/25/97 | 2 | 1358 | 5330 | 2.780 | 1.490 | 0.270 | 0.577 |
| 11001-06 | 11/28/97 | 3 | 2709 | 11700 | 2.320 | 2.510 | 0.401 | 1.667 |
| 11001-07 | 12/10/97 | 1 | 107834 | Not Tested | 59.140 | 69.600 | 4.888 | 9.167 |
| 11001-08 | 12/14/97 | 4 | 303917 | 832000 | OVER | OVER | 5.168 | 10.449 |

| PANEL MEMBER | DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000 | ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000 | ROCHE Elecsys HBsAg TEST DATE: 2/9/2000 | ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/22/2000 | SORIN ETI-MAK 4 TEST DATE: UNK | ABBOTT PRISM HBsAg TEST DATE: 8/27/2009 | ABBOTT MUREX HBsAg Version 3 TEST DATE: 8/27/2009 | ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 2/28/2000 | ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK |
|--------------|--|---|--|---|-----------------------------------|--|--|--|---|
| 11001-01 | 0.340 | 0.090 | 0.520 | 0.190 | 0.150 | 0.77 | 0.51 | 0.370 | 0.075 |
| 11001-02 | 0.290 | 0.090 | 0.540 | 0.190 | 0.150 | 0.52 | 0.50 | 0.340 | 0.057 |
| 11001-03 | 0.260 | 0.090 | 0.580 | 0.190 | 0.250 | 0.74 | 0.61 | 0.390 | 0.113 |
| 11001-04 | 0.350 | 0.090 | 0.610 | 0.320 | 0.280 | 0.63 | 0.64 | 0.420 | 0.113 |
| 11001-05 | 1.600 | 1.840 | 1.800 | 2.080 | 1.830 | 4.50 | 2.53 | 0.920 | 1.642 |
| 11001-06 | 2.810 | 3.340 | 2.950 | 2.750 | 3.330 | 8.36 | 4.37 | 1.540 | 3.226 |
| 11001-07 | 35.030 | 144.000 | 72.490 | OVER | 70.150 | 141.70 | 31.27 | 33.340 | >37.736 |
| 11001-08 | OVER | 521.000 | 258.500 | OVER | 87.250 | 44.92 | 32.01 | 60.870 | >37.736 |

| PANEL MEMBER | ORTHO VITROS ECi HBsAg TEST DATE: UNK | ABBOTT ANTI-HBc TOTAL (OD) TEST DATE: UNK | ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: UNK | ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: UNK | ORTHO VITROS ECi HBeAg TEST DATE: UNK | ABBOTT HBeAg (OD) TEST DATE: UNK | ORTHO VITROS ECi ANTI-HBe TEST DATE: UNK | ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: UNK |
|--------------|--|--|---|---|--|-------------------------------------|---|---|
| 11001-01 | 0.14 | Not Tested | 0.01 | 3.06 | 0.22 | Not Tested | 0.17 | 1.4 |
| 11001-02 | 0.13 | Not Tested | 0.01 | 3.07 | 0.23 | Not Tested | 0.15 | 0.5 |
| 11001-03 | 0.14 | Not Tested | 0.01 | 3.21 | 0.22 | Not Tested | 0.14 | 1.7 |
| 11001-04 | 0.17 | Not Tested | 0.01 | 3.03 | 0.21 | Not Tested | 0.17 | 0.8 |
| 11001-05 | 1.66 | Not Tested | 0.01 | 3.01 | 0.23 | Not Tested | 0.15 | 1.8 |
| 11001-06 | 3.45 | Not Tested | 0.01 | 3.15 | 0.23 | Not Tested | 0.15 | 0.5 |
| 11001-07 | 118 | Not Tested | 0.02 | 3.08 | 0.55 | Not Tested | 0.15 | 1.4 |
| 11007-08 | 448 | 0.88 | 0.01 | 3.19 | 2.26 | 0.608 | 0.14 | 2.3 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11001 Rev03
Effective Date: 08/11/2021