

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 65556 Part Number: HBV11002

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 5/17/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: UNK	ROCHE Cobas Core HBsAg II EIA  TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: 6/2/2005	DIASORIN HBsAg   TEST DATE: UNK
11002-01	11/06/97	4	183	0.270	0.220	NEG	0.072
11002-02	11/08/97	2	183	0.490	0.250	NEG	0.104
11002-03	11/13/97	12	1030	0.580	0.590	1390	0.102
11002-04	11/15/97	5	1389	0.680	0.880	2750	0.168
11002-05	12/11/97	9	1029804	OVER	OVER	1150000	4.324
11002-06	12/15/97	11	373899	72.550	OVER	537000	5.655

PANEL MEMBER	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0  TEST DATE: 1/24/2000	ORTHO VITROS HBsAg Reagent Pack  TEST DATE: 2/24/2000	ROCHE Elecys HBsAg  TEST DATE: 2/9/2000	ABBOTT ANTI-HBc TOTAL (Co/S)  TEST DATE: UNK	SORIN ETI-MAK 4  TEST DATE: UNK
11002-01	0.513	0.360	0.140	0.680	NOT TESTED	0.330
11002-02	0.577	0.420	0.130	0.680	NOT TESTED	0.300
11002-03	0.577	1.120	0.530	1.070	NOT TESTED	0.700
11002-04	0.513	1.120	0.820	1.340	NOT TESTED	1.000
11002-05	11.538	35.170	253.000	176.700	NOT TESTED	86.680
11002-06	7.692	OVER	1010.000	628.000	0.513	69.650

PANEL MEMBER	ABBOTT HBeAg  TEST DATE: 9/1/1999	ABBOTT PRISM HBsAg  TEST DATE: 8/27/2009	ABBOTT MUREX HBsAg Version 3  TEST DATE: 8/27/2009	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: 2/28/2000	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 3/22/2000	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: UNK
11002-01	0.288	0.46	0.63	0.420	0.250	0.132
11002-02	0.288	0.95	0.67	0.410	0.250	0.170
11002-03	0.288	2.18	1.23	0.570	0.690	0.660
11002-04	0.481	2.91	1.56	0.690	1.480	0.849
11002-05	2.019	364.79	31.70	89.950	OVER	>37.736
11002-06	4.135	227.22	30.85	56.660	OVER	>37.736

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11002 Rev04  
Effective Date: 08/11/2021