



**HBV Seroconversion Panel Donor No. 65584  
Part Number: HBV11003**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 2/11/1998	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: 6/2/2005	ROCHE Cobas Core HBsAg II EIA  TEST DATE: UNK	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 3/22/2000	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: UNK	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: 7/14/2000	DADE BEHRING Enzygnost HBsAg 5.0  TEST DATE: 1/24/2000
11003-01	07/22/97	12	NOT DETECTED	NEG	0.110	0.090	0.019	0.513	0.270
11003-02	10/14/97	5	NOT DETECTED	NEG	0.120	0.160	0.057	0.513	0.300
11003-03	11/24/97	7	NOT DETECTED	NEG	0.110	0.280	0.057	0.705	0.370
11003-04	12/02/97	10	103	NEG	0.240	0.350	0.189	0.577	0.450
11003-05	12/04/97	6	220	141	0.300	0.540	0.283	0.577	0.530
11003-06	12/11/97	10	983	2620	0.960	1.230	1.094	0.513	1.300
11003-07	12/18/97	6	2055	7490	4.970	6.160	5.358	2.949	6.160
11003-08	12/21/97	7	5044	20600	8.920	13.260	10.547	4.359	10.760

PANEL MEMBER	ORTHO VITROS Eci HBsAg Reagent Pack  TEST DATE: 2/24/2000	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: 2/28/2000	ROCHE ELECSYS HBsAg  TEST DATE: 2/9/2000	ABBOTT PRISM HBsAg  TEST DATE: UNK	ORTHO VITROS Eci HBsAg  TEST DATE: 8/14/2015	DIASORIN HBsAg  TEST DATE: 10/24/1999	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: UNK
11003-01	0.090	0.350	0.510	0.79	0.05	0.155	0.51	0.140
11003-02	0.070	0.360	0.540	0.82	0.04	0.158	0.48	0.110
11003-03	0.080	0.380	0.520	1.27	0.05	0.214	0.49	0.220
11003-04	0.120	0.450	0.610	1.57	0.07	0.203	0.70	0.440
11003-05	0.190	0.490	0.720	1.61	0.09	0.179	0.76	0.270
11003-06	1.100	0.810	1.350	3.05	0.78	0.219	1.62	1.230
11003-07	9.720	2.780	5.260	13.02	7.49	0.599	6.65	4.970
11003-08	18.600	4.730	10.460	28.84	16.8	1.083	15.27	11.220

PANEL MEMBER	SORIN ETI-MAK 4  TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL (OD)  TEST DATE: 9/1/1999	ORTHO VITROS Eci ANTI-HBc TOTAL  TEST DATE: 8/14/2015	ORTHO VITROS Eci HBeAg  TEST DATE: 8/14/2015	ABBOTT HBeAg (OD)  TEST DATE: 9/1/1999	ORTHO VITROS Eci ANTI-HBe  TEST DATE: 8/14/2015	ORTHO VITROS Eci ANTI-HBc IgM  TEST DATE: 8/14/2015	ORTHO VITROS Eci ANTI-HBs QUANT  TEST DATE: 8/14/2015
11003-01	0.130	Not Tested	3.34	0.21	Not Tested	0.16	0.01	1.7
11003-02	NR	Not Tested	3.46	0.2	Not Tested	0.15	0.01	0.7
11003-03	0.200	Not Tested	3.51	0.2	Not Tested	0.15	0.02	1.5
11003-04	0.500	Not Tested	3.75	0.21	Not Tested	0.15	0.01	3.4
11003-05	0.400	Not Tested	3.64	0.21	Not Tested	0.15	0.01	3.1
11003-06	1.300	Not Tested	3.63	0.24	Not Tested	0.17	0.01	2.5
11003-07	7.030	Not Tested	3.53	0.22	Not Tested	0.18	0.01	2.4
11003-08	12.450	1.0	3.84	0.31	0.05	0.15	0.01	1.6

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11003 Rev03  
Effective Date: 08/12/2021