



HBV Seroconversion Panel Donor No. 65732
Part Number: HBV11004

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE CAP- G/CTM HBV (cp/mL)	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3	ROCHE Cobas Core HBsAg II EIA	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0
		TEST DATE: 3/10/1998	TEST DATE: UNK	TEST DATE: 6/2/2005	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/4/2000	TEST DATE: 1/24/2000
11004-01	11/18/97	11	NOT DETECTED	NEGATIVE	0.140	0.54	0.140	0.321	0.320
11004-02	11/21/97	10	NOT DETECTED	NEGATIVE	0.330	0.50	0.130	0.577	0.320
11004-03	11/25/97	18	<52	NEGATIVE	0.460	0.50	0.130	0.449	0.450
11004-04	12/01/97	6	78	NEGATIVE	0.820	0.56	0.130	0.513	0.350
11004-05	12/03/97	6	236	NEGATIVE	0.440	0.53	0.150	0.385	0.400
11004-06	01/05/98	6	1155459	2760000	OVER	30.56	OVER	7.692	OVER
11004-07	01/07/98	7	1632129	3160000	OVER	30.67	OVER	7.692	OVER
11004-08	01/12/98	5	7993820	52100000	OVER	30.51	OVER	8.333	OVER

PANEL MEMBER	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ORTHO VITROS ECi HBsAg	ROCHE ELECSYS HBsAg	ABBOTT PRISM HBsAg	DIASORIN HBsAg	ORTHO VITROS HBsAg Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack	SORIN ETI-MAK 4
	TEST DATE: 3/22/2000	TEST DATE: UNK	TEST DATE: 8/14/2015	TEST DATE: 2/9/2000	TEST DATE: UNK	TEST DATE: 10/25/1999	TEST DATE: 2/24/2000	TEST DATE: 2/28/2000	TEST DATE: UNK
11004-01	0.090	0.077	0.05	0.460	0.35	0.174	0.100	0.340	0.300
11004-02	0.160	0.038	0.05	0.500	0.36	0.118	0.120	0.380	NR
11004-03	0.090	0.019	0.06	0.520	0.32	0.091	0.100	0.440	0.200
11004-04	0.130	0.096	0.05	0.520	1.47	0.056	0.090	0.370	0.200
11004-05	0.090	0.038	0.05	0.610	1.84	0.072	0.120	0.380	0.200
11004-06	OVER	>38.462	523	350.400	35.23	5.144	568.000	70.650	86.880
11004-07	OVER	>38.462	905	632.800	421.70	4.636	1050.000	88.340	87.180
11004-08	OVER	>38.462	2860	2040.000	570.55	4.206	3160.000	117.900	89.750

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs QUANT	ABBOTT ANTI-HBc TOTAL (Co/S)	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ABBOTT HBeAg
	TEST DATE: 8/14/2015	TEST DATE: 8/14/2015	TEST DATE: 8/14/2015	TEST DATE: UNK	TEST DATE: 8/14/2015	TEST DATE: 8/14/2015	TEST DATE: UNK
11004-01	0.19	0.03	3.1	NOT TESTED	2.68	0.21	0.385
11004-02	0.18	0.03	4.7	NOT TESTED	2.68	0.21	0.288
11004-03	0.19	QNS	QNS	NOT TESTED	2.66	0.12	0.865
11004-04	0.18	0.02	3.7	NOT TESTED	2.65	0.20	0.288
11004-05	0.16	0.02	5.4	NOT TESTED	2.85	0.19	0.288
11004-06	0.10	0.02	2.0	NOT TESTED	2.70	11.5	3.365
11004-07	0.09	0.02	7.6	NOT TESTED	2.74	16.2	2.885
11004-08	0.03	0.02	6.1	0.595	2.69	89.5	6.827

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com