



HBV Seroconversion Panel Donor No. 65777
Part Number: HBV11005

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE CAP-G/CTM HBV (cp/mL)	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3	ABBOTT PRISM HBsAg
		TEST DATE: 9/21/1998	TEST DATE: UNK	TEST DATE: 6/2/2005	TEST DATE: 7/14/2000	TEST DATE: 1/24/2000	TEST DATE: 4/18/2000	TEST DATE: UNK	TEST DATE: UNK
11005-01	08/19/97	6	<52	NEGATIVE	0.256	0.300	0.070	0.56	0.41
11005-02	08/21/97	9	NOT DETECTED	NEGATIVE	0.513	0.390	0.400	0.55	0.88
11005-03	09/30/97	17	<52	NEGATIVE	0.577	0.460	0.070	0.53	0.51
11005-04	10/31/97	21	NOT DETECTED	NEGATIVE	0.321	0.370	0.130	0.54	0.13
11005-05	11/18/97	13	NOT DETECTED	NEGATIVE	0.577	0.330	0.040	0.50	0.94
11005-06	11/23/97	17	NOT DETECTED	NEGATIVE	0.256	0.290	0.130	0.49	0.35
11005-07	11/25/97	15	NOT DETECTED	NEGATIVE	0.256	0.250	-0.160	0.54	0.82
11005-08	11/30/97	17	NOT DETECTED	NEGATIVE	0.192	0.220	0.100	0.65	0.67
11005-09	12/02/97	16	NOT DETECTED	NEGATIVE	0.256	0.240	-0.010	0.64	0.30
11005-10	12/07/97	14	NOT DETECTED	NEGATIVE	0.256	0.280	0.430	0.53	0.37
11005-11	12/09/97	13	NOT DETECTED	NEGATIVE	0.449	0.330	0.250	0.57	0.73
11005-12	01/08/98	11	56592	198000	7.500	24.420	34.980	24.73	11.04
11005-13	01/13/98	9	85655	227000	7.949	38.810	81.080	29.86	8.88
11005-14	01/15/98	19	120991	252000	7.756	OVER	OVER	30.16	160.87

PANEL MEMBER	ROCHE ELECSYS HBsAg	DIASORIN HBsAg	ABBOTT AxSYM HBsAg (V2)	ABBOTT Auszyme Monoclonal HBsAg	ABBOTT Auszyme MC Dynamic	ABBOTT IMx HBsAg (V2) Reagent Pack	ORTHO VITROS HBsAg Reagent Pack	SORIN ETI-MAK 3	SORIN ETI-MAK 4
	TEST DATE: 2/9/2000	TEST DATE: 10/24/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 2/10/2000	TEST DATE: 2/24/2000	TEST DATE: UNK	TEST DATE: UNK
11005-01	0.480	0.166	0.410	0.058	0.100	0.360	0.100	0.180	0.280
11005-02	0.550	0.190	0.560	0.058	0.170	0.450	0.100	0.180	0.300
11005-03	0.500	0.214	0.440	0.058	0.140	0.430	0.090	0.190	0.280
11005-04	0.490	0.187	0.400	0.058	0.170	0.430	0.080	0.090	0.300
11005-05	0.450	0.163	0.530	0.096	0.140	0.380	0.090	0.240	0.230
11005-06	0.480	0.201	0.440	0.077	0.100	0.470	0.080	0.120	0.250
11005-07	0.470	0.211	0.470	NEGATIVE	0.140	0.480	0.080	0.210	0.280
11005-08	0.430	0.230	0.590	0.115	0.140	0.410	0.090	0.150	0.250
11005-09	0.410	0.198	0.390	0.019	0.100	0.360	0.090	0.340	0.180
11005-10	NOT TESTED	0.198	0.430	NEGATIVE	0.140	0.400	0.090	0.180	0.280
11005-11	0.480	0.163	0.440	NEGATIVE	0.140	0.410	0.070	0.240	0.330
11005-12	22.680	1.684	12.590	19.808	29.410	7.330	40.600	15.490	32.680
11005-13	77.230	4.848	39.030	>38.462	OVER	27.500	128.000	36.940	77.080
11005-14	90.870	4.848	41.410	>38.462	OVER	24.500	158.000	33.500	78.800

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



**HBV Seroconversion Panel Donor No. 65777
Part Number: HBV11005**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	ABBOTT ANTI-HBc TOTAL (OD) TEST DATE: 9/9/1999	ORTHO VITROS ECi ANTI-HBc TEST DATE: 8/30/2018	ABBOTT HBeAg (OD) TEST DATE: 9/9/1999	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs TEST DATE: 8/30/2018
11005-01	0.05	Not Tested	3.50	Not Tested	0.14	0.18	0.01	0.5
11005-02	0.01	Not Tested	3.49	Not Tested	0.15	0.17	0.01	0.7
11005-03	0.05	Not Tested	3.44	Not Tested	0.15	0.17	0.01	0.3
11005-04	0.05	Not Tested	3.58	Not Tested	0.15	0.18	0.01	0.9
11005-05	0.05	Not Tested	3.54	Not Tested	0.16	0.17	0.01	0.2
11005-06	0.06	Not Tested	3.49	Not Tested	0.13	0.17	0.01	0.0
11005-07	0.04	Not Tested	3.54	Not Tested	0.14	0.16	0.01	0.0
11005-08	0.04	Not Tested	3.72	Not Tested	0.14	0.19	0.01	0.2
11005-09	0.05	Not Tested	3.55	Not Tested	0.13	0.18	0.01	0.0
11005-10	0.04	Not Tested	3.41	Not Tested	0.19	0.18	0.01	0.0
11005-11	0.07	Not Tested	3.60	Not Tested	0.19	0.17	0.00	0.0
11005-12	34.7	Not Tested	3.49	Not Tested	0.39	0.17	0.01	0.1
11005-13	103	Not Tested	3.66	Not Tested	1.59	0.16	0.01	0.0
11005-14	114	0.80	3.42	0.09	2.35	0.15	0.01	0.0

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11005 Rev03
Effective Date: 08/12/2021