



**HBV Seroconversion Panel Donor No. 67694
Part Number: HBV11011**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45u/L) TEST DATE: UNK | ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK | ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005 | GENETIC SYSTEMS HBsAg (2.0) TEST DATE: UNK | DADE BEHRING Enzygnost HBsAg 5.0 TEST DATE: 1/24/2000 | ORTHO HBsAg ELISA Testsystem 3 TEST DATE: UNK | ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK |
|--------------|--------------|--|--|--|---|--|--|--|
| 11011-01 | 03/14/98 | 9 | NOT DETECTED | Negative | 0.286 | 0.360 | Negative | 0.51 |
| 11011-02 | 04/30/98 | 9 | NOT DETECTED | Negative | 0.357 | 0.270 | 0.550 | 0.47 |
| 11011-03 | 05/02/98 | 7 | NOT DETECTED | Negative | 0.357 | 0.220 | 0.330 | 0.54 |
| 11011-04 | 05/07/98 | 3 | NOT DETECTED | Negative | 0.571 | 0.290 | 0.190 | 0.55 |
| 11011-05 | 05/09/98 | 1 | NOT DETECTED | Negative | 0.429 | 0.290 | 0.460 | 0.50 |
| 11011-06 | 05/16/98 | 5 | NOT DETECTED | Negative | 0.429 | 0.310 | 0.190 | 0.49 |
| 11011-07 | 05/21/98 | 6 | NOT DETECTED | Negative | 0.786 | 0.500 | 0.190 | 0.48 |
| 11011-08 | 05/23/98 | 7 | NOT DETECTED | Negative | 0.643 | 0.610 | 0.380 | 0.48 |
| 11011-09 | 06/25/98 | 8 | 1327 | 6610 | 0.929 | 0.810 | 0.930 | 1.04 |
| 11011-10 | 06/27/98 | 8 | 1559 | 5860 | 1.000 | 1.130 | 0.980 | 1.76 |
| 11011-11 | 07/02/98 | 4 | 9178 | 41100 | 2.786 | 4.200 | 3.900 | 4.90 |
| 11011-12 | 07/04/98 | 4 | 18313 | 86800 | 8.786 | 6.390 | 7.560 | 8.76 |
| 11011-13 | 07/09/98 | 6 | 66490 | 228000 | 15.000 | 24.830 | 30.360 | 27.72 |
| 11011-14 | 07/11/98 | 6 | 112916 | 366000 | 17.143 | 34.460 | 50.400 | 29.29 |

| PANEL MEMBER | DIASORIN HBsAg TEST DATE: UNK | ABBOTT PRISM HBsAg TEST DATE: UNK | ROCHE ELECSYS HBsAg TEST DATE: 2/9/2000 | ROCHE Cobas Core HBsAg II EIA TEST DATE: UNK | SORIN ETI-MAK 4 TEST DATE: UNK | ABBOTT HBeAg (OD) TEST DATE: 9/1/1999 | ABBOTT ANTI-HBc TOTAL (OD) TEST DATE: 9/1/1999 | ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 3/22/2000 |
|--------------|----------------------------------|--------------------------------------|--|---|-----------------------------------|--|---|---|
| 11011-01 | 0.136 | 0.46 | 0.510 | 0.130 | 0.230 | Not Tested | Not Tested | 0.130 |
| 11011-02 | 0.104 | 0.42 | 0.550 | 0.120 | 0.200 | Not Tested | Not Tested | 0.090 |
| 11011-03 | 0.142 | 0.52 | 0.560 | 0.110 | 0.180 | Not Tested | Not Tested | 0.130 |
| 11011-04 | 0.157 | 0.60 | 0.520 | 0.110 | 0.180 | Not Tested | Not Tested | 0.160 |
| 11011-05 | 0.157 | 0.49 | 0.480 | 0.120 | 0.150 | Not Tested | Not Tested | 0.220 |
| 11011-06 | 0.151 | 0.56 | 0.560 | 0.130 | 0.180 | Not Tested | Not Tested | 0.000 |
| 11011-07 | 0.127 | 0.42 | 0.490 | 0.120 | 0.250 | Not Tested | Not Tested | 0.060 |
| 11011-08 | 0.154 | 0.36 | 0.540 | 0.140 | 0.280 | Not Tested | Not Tested | 0.060 |
| 11011-09 | 0.219 | 1.80 | 0.820 | 0.820 | 0.750 | Not Tested | Not Tested | 0.470 |
| 11011-10 | 0.166 | 3.35 | 1.050 | 1.080 | 1.230 | Not Tested | Not Tested | 0.510 |
| 11011-11 | 0.269 | 10.93 | 2.650 | 4.200 | 11.230 | Not Tested | Not Tested | 4.230 |
| 11011-12 | 0.550 | 18.51 | 4.430 | 8.500 | 8.250 | Not Tested | Not Tested | 8.340 |
| 11011-13 | 1.876 | 65.36 | 17.400 | 37.100 | 32.580 | Not Tested | Not Tested | 26.120 |
| 11011-14 | 2.633 | 94.81 | 27.310 | 56.900 | 47.080 | 0.08 | 0.84 | 54.000 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11011 Rev03
Effective Date: 08/12/2021



**HBV Seroconversion Panel Donor No. 67694
Part Number: HBV11011**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK | ORTHO VITROS HBsAg Reagent Pack TEST DATE: 2/24/2000 | ABBOTT AxSYM HBsAg (V2) Reagent Pack TEST DATE: 2/28/2000 | ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018 | ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018 | ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018 | ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018 | ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/30/2018 | ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018 |
|--------------|---|---|--|--|---|--|---|---|---|
| 11011-01 | 0.019 | 0.140 | 0.320 | 0.07 | 3.08 | 0.14 | 0.16 | 0.02 | 0.0 |
| 11011-02 | 0.038 | 0.130 | 0.380 | 0.08 | 3.25 | 0.14 | 0.17 | 0.02 | 0.0 |
| 11011-03 | 0.038 | 0.130 | 0.370 | 0.08 | 3.34 | 0.14 | 0.17 | 0.02 | 0.0 |
| 11011-04 | 0.057 | 0.120 | 0.370 | 0.26 | 3.46 | 0.14 | 0.16 | 0.02 | 0.0 |
| 11011-05 | 0.057 | 0.130 | 0.390 | 0.07 | 3.54 | 0.14 | 0.17 | 0.01 | 0.0 |
| 11011-06 | 0.019 | 0.130 | 0.380 | 0.06 | 3.64 | 0.13 | 0.16 | 0.01 | 0.0 |
| 11011-07 | 0.094 | 0.140 | 0.370 | 0.06 | 3.32 | 0.15 | 0.16 | 0.02 | 0.0 |
| 11011-08 | 0.019 | 0.130 | 0.500 | 0.06 | 3.34 | 0.14 | 0.16 | 0.01 | 0.0 |
| 11011-09 | 0.604 | 0.900 | 0.630 | 0.65 | 3.35 | 0.14 | 0.17 | 0.02 | 0.0 |
| 11011-10 | 0.774 | 1.240 | 0.670 | 1.59 | 3.45 | 0.15 | 0.21 | 0.02 | 0.0 |
| 11011-11 | 3.660 | 8.140 | 1.740 | 6.73 | 3.34 | 0.18 | 0.17 | 0.02 | 0.0 |
| 11011-12 | 6.566 | 15.700 | 3.170 | 13.5 | 3.39 | 0.22 | 0.17 | 0.02 | 0.0 |
| 11011-13 | 26.038 | 71.400 | 9.500 | 56.1 | 3.38 | 1.24 | 0.16 | 0.02 | 0.0 |
| 11011-14 | 36.792 | 117.000 | 15.880 | 89.3 | 3.53 | 2.64 | 0.14 | 0.01 | 0.0 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com