



**HBV Seroconversion Panel Donor No. 67962
Part Number: HBV11013**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE CAP-G/CTM HBV (cp/mL)	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0	ORTHO HBsAg ELISA Testsystem 3	ABBOTT MUREX HBsAg Version 3
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11013-01	11/25/97	17	NOT DETECTED	NOT TESTED	0.375	0.220	0.190	0.61
11013-02	04/01/98	39	NOT DETECTED	NOT TESTED	0.375	0.250	0.080	0.95/1.06
11013-03	04/22/98	12	NOT DETECTED	NOT TESTED	0.500	0.200	0.080	0.54
11013-04	04/24/98	14	NOT DETECTED	NOT TESTED	0.563	0.190	0.000	0.55
11013-05	04/29/98	16	NOT DETECTED	NOT TESTED	0.438	0.200	0.230	0.54
11013-06	05/01/98	15	NOT DETECTED	NEGATIVE	0.500	0.170	0.020	0.51
11013-07	05/06/98	12	NOT DETECTED	NEGATIVE	0.250	0.260	0.110	0.50
11013-08	05/08/98	4	NOT DETECTED	NEGATIVE	0.250	0.470	0.170	0.55
11013-09	05/13/98	13	NOT DETECTED	NEGATIVE	0.250	0.220	0.110	0.57
11013-10	05/15/98	13	NOT DETECTED	NEGATIVE	0.563	0.230	0.020	0.56
11013-11	05/20/98	24	NOT DETECTED	NEGATIVE	0.375	0.200	0.020	0.56
11013-12	05/22/98	30	NOT DETECTED	NEGATIVE	0.813	0.290	0.000	0.61
11013-13	05/27/98	17	NOT DETECTED	NEGATIVE	0.375	0.190	0.020	0.58
11013-14	05/29/98	21	NOT DETECTED	NEGATIVE	0.313	0.200	0.000	0.54
11013-15	06/03/98	19	NOT DETECTED	NEGATIVE	0.313	0.220	0.110	0.55
11013-16	06/05/98	13	NOT DETECTED	NEGATIVE	0.250	0.190	0.050	0.53
11013-17	06/10/98	16	NOT DETECTED	NEGATIVE	0.500	0.230	0.520	0.58
11013-18	06/13/98	20	NOT DETECTED	NEGATIVE	0.563	0.250	0.020	0.55
11013-19	06/17/98	16	NOT DETECTED	NEGATIVE	0.563	0.190	0.050	0.60
11013-20	06/21/98	17	NOT DETECTED	NEGATIVE	0.563	0.280	0.080	0.53
11013-21	06/25/98	29	NOT DETECTED	NEGATIVE	0.500	0.230	0.140	0.59
11013-22	06/28/98	15	NOT DETECTED	NEGATIVE	0.500	0.200	0.020	0.54
11013-23	07/06/98	16	NOT DETECTED	NEGATIVE	0.375	0.250	0.020	0.62
11013-24	07/08/98	12	NOT DETECTED	NEGATIVE	0.313	0.230	0.020	0.56
11013-25	07/13/98	14	<52	NEGATIVE	0.500	0.220	0.140	0.66
11013-26	07/15/98	11	<52	NEGATIVE	0.625	0.250	0.020	0.62
11013-27	07/22/98	15	53	NEGATIVE	0.813	0.280	0.000	0.69
11013-28	07/27/98	16	86	712	0.563	0.800	0.300	1.05
11013-29	07/29/98	12	566	2440	0.688	1.120	0.550	1.36
11013-30	08/03/98	18	837	3940	0.813	1.590	1.270	1.75
11013-31	08/05/98	17	3182	14900	0.938	2.300	1.390	2.77
11013-32	08/10/98	19	7124	31800	0.938	3.430	1.730	4.42
11013-33	08/12/98	16	12099	38200	2.563	6.570	3.770	8.14
11013-34	08/17/98	17	21026	73500	3.125	11.270	7.050	15.88
11013-35	08/19/98	9	35707	249000	3.750	17.370	11.670	28.95

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11013 Rev04
Effective Date: 08/12/2021



**HBV Seroconversion Panel Donor No. 67962
Part Number: HBV11013**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	BIORAD MONALISA HBsAg PLUS	ABBOTT IMx HBsAg (V2) Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack	ORTHO VITROS HBsAg Reagent Pack	ROCHE ELECSYS HBsAg	ABBOTT PRISM HBsAg	SORIN ETI-MAK 3	SORIN ETI-MAK 4
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11013-01	0.190	0.460	0.420	0.110	0.540	0.35	0.110	0.160
11013-02	0.250	0.350	0.570	0.090	0.460	0.34	0.160	0.160
11013-03	0.230	0.380	0.410	0.120	0.430	0.88	0.070	0.130
11013-04	0.210	0.450	0.420	0.100	0.430	0.52	0.070	0.260
11013-05	0.300	0.380	0.410	0.110	0.410	1.54	0.140	0.210
11013-06	0.320	0.410	0.410	0.100	0.400	0.54	0.120	0.180
11013-07	0.320	0.380	0.450	0.100	0.430	0.84	0.120	0.130
11013-08	0.170	0.390	0.450	0.080	0.390	0.80	0.090	0.180
11013-09	0.170	0.450	0.440	0.090	0.400	0.31	0.070	0.210
11013-10	0.260	0.410	0.430	0.080	0.400	0.38	0.070	0.180
11013-11	0.150	0.390	0.410	0.090	0.370	0.38	0.050	0.160
11013-12	0.280	0.370	0.430	0.090	0.410	0.49	0.090	0.210
11013-13	0.260	0.360	0.470	0.090	0.400	0.61	0.110	0.310
11013-14	0.300	0.380	0.410	0.080	0.400	0.47	0.120	0.260
11013-15	0.340	0.370	0.400	0.110	0.390	0.28	0.120	0.180
11013-16	0.190	0.410	0.470	0.090	0.390	0.04	0.110	0.130
11013-17	0.190	0.500	0.440	0.080	0.430	0.93	0.120	0.160
11013-18	0.190	0.390	0.460	0.080	0.430	0.33	0.020	0.260
11013-19	0.190	0.370	0.390	0.100	0.410	0.40	0.040	0.160
11013-20	0.280	0.430	0.400	0.120	0.420	0.39	0.090	0.160
11013-21	0.340	0.340	0.470	0.100	0.410	0.32	0.200	0.240
11013-22	0.320	0.410	0.440	0.100	0.440	0.46	0.120	0.180
11013-23	0.340	0.430	0.350	0.100	0.400	0.54	0.120	0.180
11013-24	0.190	0.380	0.450	0.120	0.380	0.61	0.090	0.180
11013-25	0.170	0.400	0.450	0.100	0.400	0.42	0.110	0.160
11013-26	0.230	0.360	0.400	0.100	0.370	0.34	0.110	0.210
11013-27	0.310	0.400	0.510	0.120	0.490	0.71	0.180	0.260
11013-28	0.360	0.480	0.640	0.390	0.720	1.49	0.390	0.790
11013-29	0.570	0.610	0.760	0.640	1.020	1.31	0.760	1.290
11013-30	0.700	0.810	1.070	1.110	1.380	5.79	0.850	1.500
11013-31	0.940	1.020	1.530	2.490	2.240	5.54	1.390	2.180
11013-32	0.740	1.230	1.940	3.860	2.720	13.20	2.600	3.260
11013-33	1.550	2.090	3.510	7.190	4.880	15.78	5.140	8.500
11013-34	2.960	3.720	5.920	15.600	9.240	37.90	10.440	17.950
11013-35	3.910	5.400	8.890	26.400	16.140	89.15	15.360	24.260

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



**HBV Seroconversion Panel Donor No. 67962
Part Number: HBV11013**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT Auszyme MC Dynamic TEST DATE: UNK	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: UNK	ORTHO VITROS ECI HBsAg TEST DATE: 8/30/2018	ORTHO VITROS ECI ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS ECI HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECI ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS ECI ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECI ANTI-HBs QUANT TEST DATE: 8/30/2018
11013-01	0.120	0.038	0.07	3.47	0.24	0.18	0.31	0.4
11013-02	0.000	0.038	0.07	3.40	0.24	0.19	0.28	0.0
11013-03	0.240	0.094	0.10	2.87	0.25	0.19	0.31	0.2
11013-04	0.000	0.019	0.09	3.06	0.30	0.21	0.33	0.4
11013-05	0.120	0.057	0.06	3.42	0.22	0.18	0.28	0.0
11013-06	0.040	0.075	0.06	3.51	0.19	0.17	0.27	0.0
11013-07	0.080	0.057	0.06	3.34	0.26	0.17	0.28	2.5
11013-08	0.000	0.057	0.06	3.57	0.20	0.17	0.25	0.6
11013-09	0.000	0.057	0.05	3.40	0.19	0.17	0.25	0.9
11013-10	0.040	0.132	0.06	3.61	0.21	0.18	0.22	0.0
11013-11	0.000	0.075	0.12	3.65	0.21	0.18	0.26	0.3
11013-12	0.000	0.038	0.06	3.57	0.19	0.17	0.24	0.0
11013-13	0.000	0.057	0.05	3.45	0.18	0.16	0.25	1.0
11013-14	0.000	0.132	0.08	3.51	0.25	0.16	0.24	0.8
11013-15	0.120	0.038	0.17	3.19	0.25	0.18	0.29	0.8
11013-16	0.000	0.094	0.09	3.51	0.21	0.17	0.26	0.0
11013-17	0.000	0.094	0.05	3.53	0.19	0.17	0.24	0.0
11013-18	0.000	0.075	0.05	3.61	0.20	0.16	0.24	0.5
11013-19	0.040	0.189	0.06	3.45	0.21	0.17	0.28	0.1
11013-20	0.040	0.075	0.06	3.52	0.18	0.16	0.26	0.1
11013-21	0.040	0.075	0.06	3.34	0.21	0.19	0.27	0.0
11013-22	0.000	0.075	0.07	3.22	0.23	0.20	0.31	0.0
11013-23	0.000	0.057	0.14	3.67	0.27	0.20	0.33	0.2
11013-24	0.120	0.057	0.10	3.57	0.19	0.17	0.25	0.4
11013-25	0.040	0.075	0.06	3.38	0.21	0.17	0.28	0.0
11013-26	0.120	0.057	0.08	3.41	0.22	0.17	0.26	0.0
11013-27	0.240	0.113	0.13	3.31	0.23	0.17	0.25	0.2
11013-28	0.070	0.453	0.22	3.44	0.27	0.18	0.30	0.0
11013-29	0.130	0.755	0.39	3.39	0.20	0.17	0.26	0.0
11013-30	0.220	1.604	0.89	3.48	0.21	0.15	0.24	0.0
11013-31	2.840	1.811	1.15	3.45	0.21	0.16	0.24	0.3
11013-32	5.080	3.283	2.59	3.70	0.22	0.14	0.22	0.0
11013-33	9.040	5.472	5.81	3.24	0.25	0.16	0.26	0.0
11013-34	18.200	11.321	13.5	3.37	0.33	0.16	0.26	0.1
11013-35	31.600	13.340	20.8	3.55	0.35	0.13	0.14	0.0

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV11013 Rev04
Effective Date: 08/12/2021