



**HBV Seroconversion Panel Donor No. 68029
Part Number: HBV11014**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	ROCHE CAP-G/CTM HBV (cp/mL)	GENETIC SYSTEMS HBsAg (2.0)	DADE BEHRING Enzygnost HBsAg 5.0	ORTHO HBsAg ELISA Testsystem 3
		TEST DATE: 11/2/1998	TEST DATE: UNK	TEST DATE: 6/2/2005	TEST DATE: UNK	TEST DATE: 12/15/1999	TEST DATE: 1/4/2000
11014-01	06/26/98	3	NOT DETECTED	NEGATIVE	0.571	0.230	0.000
11014-02	06/28/98	7	NOT DETECTED	NEGATIVE	0.357	0.170	0.000
11014-03	07/03/98	6	NOT DETECTED	NEGATIVE	0.357	0.300	0.000
11014-04	07/05/98	5	NOT DETECTED	NEGATIVE	0.500	0.280	0.000
11014-05	07/12/98	20	NOT DETECTED	NEGATIVE	0.571	0.250	0.000
11014-06	07/17/98	17	NOT DETECTED	NEGATIVE	0.643	0.290	0.000
11014-07	07/19/98	12	NOT DETECTED	NEGATIVE	0.714	0.230	0.000
11014-08	07/31/98	0	71	NEGATIVE	0.571	0.340	0.000
11014-09	08/02/98	7	139	263	1.357	0.320	0.000
11014-10	08/16/98	2	3654	21400	3.214	5.170	3.980
11014-11	08/20/98	2	13892	85200	7.143	15.310	13.360
11014-12	08/23/98	4	41952	136000	13.571	33.160	32.260

PANEL MEMBER	ABBOTT MUREX HBsAg Version 3	BIORAD MONALISA HBsAg PLUS	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	ABBOTT PRISM HBsAg	BIOTEST ELISA HBsAg	DIASORIN HBsAg
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 12/15/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 4/18/2000	TEST DATE: UNK
11014-01	0.52	0.470	0.110	0.038	0.97	0.140	0.142
11014-02	0.55	0.350	0.030	0.019	0.26	0.140	0.109
11014-03	0.50	0.420	0.170	0.019	0.72	0.050	0.095
11014-04	0.50	0.400	0.110	NOT REACTIVE	0.74	0.220	0.151
11014-05	0.54	0.350	0.230	0.096	0.78	0.140	0.189
11014-06	0.57	0.350	0.110	0.038	1.46	0.180	0.142
11014-07	0.55	0.200	0.090	0.058	1.80	0.120	0.166
11014-08	0.69	0.270	0.140	0.058	1.18	0.180	0.198
11014-09	0.64	0.400	0.370	0.058	1.88	0.140	0.186
11014-10	4.88	2.000	5.140	3.077	11.51	1.010	0.716
11014-11	13.96	11.240	16.620	8.731	25.18	3.390	1.284
11014-12	24.33	20.620	36.030	22.692	74.26	9.540	2.843

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



HBV Seroconversion Panel Donor No. 68029
Part Number: HBV11014

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS HBsAg Reagent Pack	ABBOTT AxSYM HBsAg (V2) Reagent Pack	ROCHE Cobas Core HBsAg II EIA	SORIN ETI-MAK 3	SORIN ETI-MAK 4	ABBOTT HBsAg (OD)	ABBOTT ANTI-HBc TOTAL (OD)
	TEST DATE: 3/23/2000	TEST DATE: 1/10/2000	TEST DATE: 4/18/2000	TEST DATE: 12/8/1999	TEST DATE: UNK	TEST DATE: 9/4/1999	TEST DATE: 9/9/1999
11014-01	0.100	0.750	0.160	0.020	0.200	NOT TESTED	NOT TESTED
11014-02	0.100	0.580	0.140	0.020	0.150	NOT TESTED	NOT TESTED
11014-03	0.100	0.520	0.160	0.020	0.130	NOT TESTED	NOT TESTED
11014-04	0.100	0.500	0.180	0.000	0.200	NOT TESTED	NOT TESTED
11014-05	0.100	0.550	0.220	0.060	0.150	NOT TESTED	NOT TESTED
11014-06	0.100	0.420	0.160	0.040	0.230	NOT TESTED	NOT TESTED
11014-07	0.100	0.470	0.160	0.060	0.230	NOT TESTED	NOT TESTED
11014-08	0.200	0.600	0.220	0.170	0.350	NOT TESTED	NOT TESTED
11014-09	0.100	0.510	0.260	0.070	0.330	NOT TESTED	NOT TESTED
11014-10	6.090	2.720	3.180	2.900	6.100	NOT TESTED	NOT TESTED
11014-11	25.090	7.060	10.630	9.320	17.200	NOT TESTED	NOT TESTED
11014-12	60.560	15.860	25.150	22.880	39.900	0.05	0.91

PANEL MEMBER	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11014-01	0.04	3.09	0.11	0.20	0.01	0.0
11014-02	0.06	3.05	0.11	0.18	0.01	0.0
11014-03	0.06	3.01	0.13	0.17	0.01	0.0
11014-04	0.06	3.28	0.11	0.18	0.01	0.0
11014-05	0.03	3.43	0.11	0.16	0.01	0.0
11014-06	0.04	3.21	0.11	0.18	0.01	0.0
11014-07	0.04	3.35	0.11	0.17	0.01	0.0
11014-08	0.05	3.25	0.12	0.18	0.01	0.0
11014-09	0.06	3.13	0.11	0.17	0.01	0.0
11014-10	4.16	3.16	0.12	0.18	0.01	0.0
11014-11	16.2	3.26	0.20	0.17	0.01	0.0
11014-12	41.8	3.17	0.28	0.18	0.01	0.0

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com