

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 68739 Part Number: HBV11017

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 10/27/1998	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL)  TEST DATE: 6/2/2005	GENETIC SYSTEMS HBsAg (2.0)  TEST DATE: UNK	DADE BEHRING Enzygnost HBsAg 5.0  TEST DATE: 12/15/1999	BIORAD MONALISA HBsAg PLUS  TEST DATE: UNK	ROCHE Cobas Core HBsAg II EIA  TEST DATE: 4/18/2000
11017-01	08/26/98	7	NOT DETECTED	NEGATIVE	0.643	0.770	0.430	0.180
11017-02	08/31/98	6	NOT DETECTED	NEGATIVE	0.571	0.210	0.380	0.160
11017-03	09/02/98	9	NOT DETECTED	NEGATIVE	0.643	0.320	0.500	0.160
11017-04	09/11/98	8	NOT DETECTED	NEGATIVE	0.714	0.190	0.570	0.180
11017-05	09/18/98	6	NOT DETECTED	NEGATIVE	0.500	0.290	0.330	0.180
11017-06	09/20/98	4	<52	NEGATIVE	0.500	0.330	0.430	0.180
11017-07	09/27/98	8	66	NEGATIVE	0.571	0.300	0.280	0.220
11017-08	09/29/98	6	96	224	0.714	0.390	0.380	0.220
11017-09	10/05/98	13	318	619	0.643	0.610	0.600	0.480
11017-10	10/07/98	8	799	2230	0.643	0.870	0.820	0.660
11017-11	10/12/98	9	4100	21500	1.429	2.400	1.870	1.810
11017-12	10/15/98	3	5791	32000	2.357	4.760	2.570	3.890
11017-13	10/19/98	3	17896	61100	5.071	11.880	5.180	9.640
11017-14	10/22/98	9	41952	89300	7.857	17.700	11.700	15.380

PANEL MEMBER	ORTHO VITROS HBsAg Reagent Pack  TEST DATE: 3/23/2000	ABBOTT AxSYM HBsAg (V2) Reagent Pack  TEST DATE: 1/10/2000	ABBOTT PRISM HBsAg  TEST DATE: UNK	BIOTEST ELISA HBsAg  TEST DATE: 4/18/2000	ABBOTT Auszyme MC Dynamic HBsAg  TEST DATE: 12/15/1999	ABBOTT Auszyme Monoclonal HBsAg  TEST DATE: UNK
11017-01	0.150	0.570	0.39	0.220	0.110	0.055
11017-02	0.190	0.600	0.36	0.240	0.290	0.073
11017-03	0.200	0.420	0.33	0.180	0.260	0.073
11017-04	0.140	0.470	0.39	0.140	0.110	0.109
11017-05	0.130	0.480	0.35	0.180	0.140	0.073
11017-06	0.120	0.430	0.40	0.140	0.140	0.127
11017-07	0.160	0.590	0.45	0.140	0.260	0.182
11017-08	0.190	0.530	0.53	0.220	0.400	0.145
11017-09	0.320	0.650	1.08	0.240	0.690	0.327
11017-10	0.540	0.820	2.62	0.290	0.800	0.673
11017-11	2.180	1.700	2.02	1.140	3.070	1.655
11017-12	5.500	2.920	14.85	1.460	5.250	4.727
11017-13	14.940	6.280	24.83	4.200	13.380	12.182
11017-14	27.520	12.330	46.94	4.970	18.230	14.909

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11017 Rev03  
Effective Date: 06/15/2021

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 68739 Part Number: HBV11017

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO HBsAg ELISA Testsystem 3  TEST DATE: 4/18/2000	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	ABBOTT HBeAg  TEST DATE: 9/1/1999	SORIN ETI-MAK 3  TEST DATE: 12/8/1999	SORIN ETI-MAK 4  TEST DATE: UNK	ABBOTT ANTI-HBc TOTAL  TEST DATE: 9/1/1999
11017-01	0.040	0.47	NOT TESTED	0.070	0.230	NOT TESTED
11017-02	0.190	0.47	NOT TESTED	0.070	0.250	NOT TESTED
11017-03	0.130	0.63	NOT TESTED	0.090	0.330	NOT TESTED
11017-04	0.160	0.48	NOT TESTED	0.090	0.300	NOT TESTED
11017-05	0.070	0.45	NOT TESTED	0.060	0.250	NOT TESTED
11017-06	0.130	0.50	NOT TESTED	0.060	0.200	NOT TESTED
11017-07	0.190	0.55	NOT TESTED	0.090	0.400	NOT TESTED
11017-08	0.310	0.59	NOT TESTED	0.130	0.300	NOT TESTED
11017-09	0.430	0.96/1.095	NOT TESTED	0.350	0.730	NOT TESTED
11017-10	0.810	1.31	NOT TESTED	0.630	1.030	NOT TESTED
11017-11	2.300	3.55	NOT TESTED	1.560	2.850	NOT TESTED
11017-12	4.810	7.07	NOT TESTED	3.500	5.700	NOT TESTED
11017-13	13.560	19.02	NOT TESTED	8.780	9.850	NOT TESTED
11017-14	23.130	25.80	0.405	11.910	25.570	0.464

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11017 Rev03  
Effective Date: 06/15/2021