

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 70793 Part Number: HBV11024

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 6/21/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL)  TEST DATE: UNK	ROCHE HBV Ampliscreen  TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static  TEST DATE: UNK	ORTHO TM HBsAg ELISA Testsystem 3  TEST DATE: UNK
11024-01	02/03/99	15	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.29
11024-02	02/06/99	6	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.27
11024-03	02/10/99	17	NOT DETECTED	NEGATIVE	NEGATIVE	NEGATIVE	0.21
11024-04	02/14/99	10	NOT DETECTED	NEGATIVE/POSITIVE	NEGATIVE/POSITIVE	NEGATIVE	0.29
11024-05	02/18/99	10	NOT DETECTED	200	POSITIVE	NEGATIVE	0.18
11024-06	02/21/99	7	<52	200	POSITIVE	NEGATIVE	0.21
11024-07	02/25/99	11	70	200	POSITIVE	NEGATIVE	0.21
11024-08	02/28/99	16	259	700	POSITIVE	NEGATIVE	0.27
11024-09	03/04/99	12	450	500	POSITIVE	NEGATIVE	0.24
11024-10	03/07/99	12	763	7,300	POSITIVE	NEGATIVE	0.32
11024-11	03/15/99	15	4929	4,400	POSITIVE	NEGATIVE	0.77
11024-12	03/18/99	9	8371	24,000	POSITIVE	NEGATIVE	0.59
11024-13	03/24/99	16	44953	110,000	POSITIVE	NEGATIVE	2.15
11024-14	03/29/99	13	183128	350,000	POSITIVE	POSITIVE	15.18

PANEL MEMBER	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	BIORAD MONALISA HBsAg PLUS  TEST DATE: UNK	VENDOR HBsAg  TEST DATE: UNK	BIOTEST ELISA HBsAg  TEST DATE: UNK	ABBOTT ARCHITECT HBsAg  TEST DATE: UNK	ROCHE ELECSYS HBsAg  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: UNK	ABBOTT MUREX  TEST DATE: UNK
11024-01	0.48	0.28	NEGATIVE	0.15	0.00	0.36	0.68	0.52
11024-02	0.50	0.26	NEGATIVE	0.13	0.00	0.38	0.56	0.49
11024-03	0.49	0.30	NEGATIVE	0.13	0.00	0.40	0.30	0.48
11024-04	0.47	0.28	NEGATIVE	0.21	0.00	0.33	0.45	0.53
11024-05	0.62	0.30	NEGATIVE	0.19	0.00	0.39	0.34	0.49
11024-06	0.53	0.34	NEGATIVE	0.17	0.00	0.31	0.37	0.49
11024-07	0.53	0.26	NEGATIVE	0.15	0.00	0.34	0.69	0.51
11024-08	0.54	0.36	NEGATIVE	0.15	0.00	0.35	0.60	0.51
11024-09	0.53	0.26	NEGATIVE	0.19	0.00	0.39	0.73	0.44
11024-10	0.53	0.30	NEGATIVE	0.19	0.00	0.45	0.74	0.52
11024-11	1.30	0.47	NEGATIVE	0.36	0.03	1.29	2.21	0.81
11024-12	1.61	0.42	NEGATIVE	0.34	0.05	1.28	2.27	0.99
11024-13	3.99	0.74	NEGATIVE	0.87	0.14	3.27	6.31	1.97
11024-14	26.18	5.43	POSITIVE	9.13	1.12	27.71	48.14	13.57

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11024 Rev03  
Effective Date: 08/12/2021



**HBV Seroconversion Panel Donor No. 70793**  
**Part Number: HBV11024**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DADE BEHRING Enzygnost	ORTHO ANTI-HBc TOTAL S/CO	ABBOTT Auszyme MC Dynamic	ORTHO VITROS HBsAg Reagent Pack	ABBOTT AxSYM HBsAg V2 Reagent Pack	ABBOTT IMx HBsAg V2 Reagent Pack	ABBOTT AxSYM HBe 2.0 Reagent Pack S/CO	ABBOTT AxSYM CORE S/CO
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/8/2002	TEST DATE: 5/14/2002	TEST DATE: 7/9/2002	TEST DATE: 5/10/2002	TEST DATE: 6/19/2002
11024-01	0.27	0.989	0.11	0.08	0.45	0.33	0.20	0.21
11024-02	0.23	0.919	0.07	0.08	0.45	0.37	0.13	0.19
11024-03	0.43	0.905	0.07	0.08	0.41	0.37	0.10	0.18
11024-04	0.23	0.932	0.11	0.08	0.43	0.35	0.22	0.18
11024-05	0.22	0.947	0.07	0.08	0.37	0.25	0.20	0.17
11024-06	0.21	0.793	0.11	0.09	0.49	0.44	0.23	0.16
11024-07	0.28	0.923	0.11	0.09	0.37	0.33	0.14	0.17
11024-08	0.21	0.793	0.07	0.09	0.37	0.26	0.13	0.17
11024-09	0.27	0.749	0.11	0.09	0.38	0.40	0.16	0.16
11024-10	0.27	0.684	0.11	0.10	0.50	0.38	0.15	0.14
11024-11	0.94	0.844	0.86	0.46	0.73	1.10	0.17	0.17
11024-12	0.93	0.554	1.00	0.51	0.79	1.03	0.28	0.13
11024-13	2.04	0.613	2.79	2.01	1.50	2.85	0.81	0.14
11024-14	15.85	0.613	23.11	24.10	9.77	21.68	2.16	0.13

PANEL MEMBER	ABBOTT HBsAg Proc B	ABBOTT HBsAg Proc C	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11024-01	0.052	0.103	0.10	0.35	0.13	0.16	0.29	0.0
11024-02	0.086	0.207	0.10	0.31	0.14	0.17	0.29	0.0
11024-03	0.069	0.172	0.07	0.35	0.14	0.17	0.27	0.0
11024-04	0.138	0.138	0.06	0.26	0.13	0.18	0.29	0.0
11024-05	0.086	0.103	0.08	0.32	0.14	0.19	0.26	0.0
11024-06	0.052	0.207	0.08	0.39	0.14	0.17	0.24	0.0
11024-07	0.138	0.310	0.08	0.53	0.14	0.16	0.18	0.0
11024-08	0.052	0.241	0.08	0.43	0.14	0.16	0.21	0.0
11024-09	0.086	0.069	0.07	0.35	0.13	0.17	0.21	0.0
11024-10	0.121	0.034	0.10	0.38	0.14	0.18	0.22	0.0
11024-11	0.569	0.586	0.40	0.43	0.17	0.16	0.22	0.0
11024-12	0.655	0.552	0.52	0.41	0.20	0.18	0.21	0.0
11024-13	1.948	1.276	2.04	0.38	0.74	0.17	0.22	0.0
11024-14	12.241	9.931	21.0	0.39	3.84	0.14	0.22	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11024 Rev03  
 Effective Date: 08/12/2021