

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 69751 Part Number: HBV11029

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 1/21/1999	ABBOTT HBV REAL TIME (CP/ML)  TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL)  TEST DATE: UNK	ROCHE HBV Ampliscreen  TEST DATE: UNK	ORTHO HBsAg Proc B Rpt  TEST DATE: UNK
11029-01	11/18/98	7	NOT DETECTED	NOT TESTED	NEGATIVE	NOT TESTED
11029-02	11/20/98	12	NOT DETECTED	NOT TESTED	NEGATIVE	NOT TESTED
11029-03	11/29/98	1	NOT DETECTED	<100	POSITIVE	NOT TESTED
11029-04	12/05/98	6	NOT DETECTED	NOT TESTED	NEGATIVE	NOT TESTED
11029-05	12/09/98	3	<52	<100	POSITIVE	NOT TESTED
11029-06	12/12/98	5	<52	200	POSITIVE	NOT TESTED
11029-07	12/16/98	5	220	200	POSITIVE	NOT TESTED
11029-08	12/20/98	10	365	200	POSITIVE	NOT TESTED
11029-09	12/23/98	11	1078	2300	POSITIVE	NOT TESTED
11029-10	12/31/98	6	12964	11000	POSITIVE	NOT TESTED
11029-11	01/05/99	5	15950	71000	POSITIVE	2.033
11029-12	01/07/99	3	15232	41000	POSITIVE	NOT TESTED
11029-13	01/14/99	6	1709	9600	POSITIVE	NOT TESTED

PANEL MEMBER	ABBOTT HBsAg Proc B  TEST DATE: UNK	ORTHO HBsAg Proc B  TEST DATE: UNK	ABBOTT HBsAg Proc C  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: UNK	VENDOR HBsAg  TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static  TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK
11029-01	0.093	0.200	0.103	0.32	NEGATIVE	NEGATIVE	0.57
11029-02	0.037	0.533	0.069	0.25	NEGATIVE	NEGATIVE	0.55
11029-03	0.037	0.033	0.034	0.46	NEGATIVE	NEGATIVE	0.49
11029-04	0.074	0.067	0.103	0.54	NEGATIVE	NEGATIVE	0.52
11029-05	0.037	0.033	0.103	0.34	NEGATIVE	NEGATIVE	0.49
11029-06	0.074	0.067	0.034	0.49	NEGATIVE	NEGATIVE	0.50
11029-07	0.074	0.067	0.069	0.47	NEGATIVE	NEGATIVE	0.58
11029-08	0.093	0.100	0.276	0.55	NEGATIVE	NEGATIVE	0.72
11029-09	0.296	0.233	0.207	1.16	NEGATIVE	NEGATIVE	1.10
11029-10	2.315	0.700	1.655	7.61	NEGATIVE	NEGATIVE	6.05
11029-11	7.241	1.967	4.000	17.22	POSITIVE	POSITIVE	11.53
11029-12	7.241	2.733	4.483	20.60	POSITIVE	POSITIVE	11.76
11029-13	2.333	0.533	1.414	6.23	POSITIVE	POSITIVE	4.47

### Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11029 Rev04  
Effective Date: 06/15/2021

# ZeptoMetrix®

## HBV Seroconversion Panel Donor No. 69751 Part Number: HBV11029

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS Eci HBsAg	ORTHO VITROS Eci ANTI-HBc TOTAL	ORTHO VITROS Eci HBeAg	ORTHO VITROS Eci ANTI-HBe	ORTHO VITROS Eci ANTI-HBc IgM	ORTHO VITROS Eci ANTI-HBs QUANT
	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11029-01	0.07	3.07	0.14	0.21	0.04	0.0
11029-02	0.07	3.15	0.25	0.20	0.04	0.0
11029-03	0.07	3.13	0.12	0.18	0.03	0.0
11029-04	0.07	3.23	0.12	0.20	0.04	0.0
11029-05	0.07	3.21	0.14	0.17	0.04	0.0
11029-06	0.12	3.12	0.24	0.17	0.04	0.0
11029-07	0.14	2.95	0.20	0.20	0.04	0.0
11029-08	0.15	3.04	0.16	0.19	0.04	0.1
11029-09	0.31	3.15	0.16	0.15	0.03	0.0
11029-10	3.80	2.79	0.15	0.21	0.05	0.0
11029-11	9.35	2.75	0.17	0.19	0.04	0.0
11029-12	10.3	3.06	0.19	0.19	0.03	0.2
11029-13	1.70	2.78	0.17	0.17	0.03	0.9

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11029 Rev04  
Effective Date: 06/15/2021