



**HBV Seroconversion Panel Donor No. 72439**  
**Part Number: HBV11052**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)  TEST DATE: 8/12/1999	ABBOTT HBV REAL TIME CP/ML  TEST DATE: UNK	ROCHE HBV Ampliscreen  TEST DATE: UNK	ROCHE HBV Monitor Assay (copies/mL)  TEST DATE: UNK	GENETIC SYSTEMS HBsAg 2.0 Static  TEST DATE: UNK	ABBOTT HBsAg Proc C  TEST DATE: UNK	ABBOTT HBsAg Proc B  TEST DATE: UNK	ORTHO HBsAg Proc B S/CO  TEST DATE: UNK
11052-01	06/14/99	14	<52	POSITIVE	6,700	NEGATIVE	0.345	0.019	0.032
11052-02	06/16/99	11	NOT DETECTED	NEGATIVE	NOT TESTED	NEGATIVE	0.207	0.057	0.161
11052-03	06/21/99	21	NOT DETECTED	NEGATIVE	NOT TESTED	NEGATIVE	0.207	0.019	0.161
11052-04	06/23/99	11	NOT DETECTED	NEGATIVE	NOT TESTED	NEGATIVE	0.138	0.019	0.129
11052-05	07/05/99	26	NOT DETECTED	POSITIVE	<100	NEGATIVE	0.185	0.019	0.097
11052-06	07/07/99	19	NOT DETECTED	POSITIVE	500	NEGATIVE	0.148	0.019	0.226
11052-07	07/19/99	21	71	POSITIVE	500	NEGATIVE	0.296	0.321	0.258
11052-08	07/21/99	14	215	POSITIVE	500	NEGATIVE	0.407	0.528	0.258
11052-09	07/26/99	24	401	POSITIVE	2,100	NEGATIVE	1.037	2.321	0.968
11052-10	07/28/99	19	837	POSITIVE	8,000	POSITIVE	1.889	3.340	1.355
11052-11	08/02/99	20	3110	POSITIVE	7,800	POSITIVE	7.148	12.075	5.000
11052-12	08/04/99	20	2902	POSITIVE	9,200	POSITIVE	9.889	13.679	4.903
11052-13	08/09/99	20	35707	POSITIVE	130,000	POSITIVE	32.074	>37.736	24.355

PANEL MEMBER	ORTHO HBsAg Proc B Rpt  TEST DATE: UNK	ABBOTT PRISM HBsAg  TEST DATE: UNK	ABBOTT MUREX HBsAg Version 3  TEST DATE: UNK	ORTHO VITROS ECi HBsAg  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL  TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM  TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT  TEST DATE: 8/30/2018
11052-01	NOT TESTED	0.62	0.53	0.11	3.23	0.13	0.20	0.02	0.0
11052-02	NOT TESTED	0.64	0.50	0.11	3.10	0.23	0.19	0.02	0.0
11052-03	NOT TESTED	0.39	0.49	0.09	2.97	0.11	0.19	0.02	0.0
11052-04	NOT TESTED	0.38	0.53	0.21	3.18	0.11	0.18	0.02	0.0
11052-05	NOT TESTED	0.31	0.54	0.13	3.40	0.13	0.22	0.02	0.0
11052-06	NOT TESTED	0.41	0.52	0.09	3.43	0.11	0.18	0.02	0.0
11052-07	NOT TESTED	1.04	0.80	0.31	3.27	0.13	0.22	0.02	0.0
11052-08	NOT TESTED	2.54	0.98	0.70	3.22	0.14	0.19	0.02	0.0
11052-09	NOT TESTED	4.37	2.18	2.55	3.15	0.16	0.18	0.02	0.0
11052-10	1.267	7.75	3.60	4.42	3.31	0.20	0.19	0.02	0.0
11052-11	NOT TESTED	27.04	11.75	16.9	3.23	0.51	0.20	0.02	0.0
11052-12	NOT TESTED	39.00	16.25	23.9	3.23	0.75	0.17	0.02	0.0
11052-13	NOT TESTED	107.42	32.05	93.3	3.10	8.78	0.14	0.02	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV11052 Rev04  
 Effective Date: 08/13/2021