



HBV Seroconversion Panel Donor No. 71922
Part Number: HBV11062

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT HBV	ROCHE	ROCHE	GENETIC	VENDOR	ABBOTT	ABBOTT
		Normal Range (10-45u/L)	REAL TIME CP/ML	HBV Monitor Assay (copies/mL)	HBV Ampliscreen	SYSTEMS HBsAg 2.0 Static	HBsAg	MUREX HBsAg Version 3	PRISM HBsAg
		TEST DATE: 8/6/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
11062-01	05/18/99	6	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	NEGATIVE	0.44	1.04
11062-02	05/20/99	8	NOT DETECTED	NOT TESTED	NEGATIVE	NEGATIVE	NEGATIVE	0.41	1.77
11062-03	05/25/99	7	<52	200	POSITIVE	NEGATIVE	NEGATIVE	0.47	0.98
11062-04	05/27/99	0	NOT DETECTED	200	POSITIVE	NEGATIVE	NEGATIVE	0.45	1.08
11062-05	06/01/99	5	110	200	POSITIVE	NEGATIVE	NEGATIVE	0.69	0.92
11062-06	06/04/99	9	130	200	POSITIVE	NEGATIVE	NEGATIVE	0.92	1.82
11062-07	06/09/99	8	665	8000	POSITIVE	NEGATIVE	NEGATIVE	1.91	3.06
11062-08	06/11/99	14	918	9900	POSITIVE	POSITIVE	POSITIVE	2.75	5.09
11062-09	06/16/99	7	8969	75000	POSITIVE	POSITIVE	POSITIVE	8.49	21.33
11062-10	06/18/99	4	12381	110000	POSITIVE	POSITIVE	POSITIVE	15.21	27.16
11062-11	06/23/99	6	85655	650000	POSITIVE	POSITIVE	POSITIVE	30.04	112.40
11062-12	06/26/99	3	220168	1600000	POSITIVE	POSITIVE	POSITIVE	30.59	180.17

PANEL MEMBER	ABBOTT HBsAg Proc C	ABBOTT HBsAg Proc B	ORTHO HBsAg Proc B	ORTHO VITROS Eci HBsAg	ORTHO VITROS Eci ANTI-HBc TOTAL	ORTHO VITROS Eci HBeAg	ORTHO VITROS Eci ANTI-HBe	ORTHO VITROS Eci ANTI-HBc IgM	ORTHO VITROS Eci ANTI-HBs QUANT
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018
11062-01	0.185	0.019	0.031	0.05	3.33	0.11	0.17	0.01	0.0
11062-02	0.222	0.019	0.031	0.05	3.20	0.12	0.17	0.01	0.0
11062-03	0.037	0.019	0.094	0.05	3.24	0.11	0.13	0.01	0.0
11062-04	0.148	0.302	0.000	0.05	3.19	0.12	0.14	0.01	0.0
11062-05	0.259	0.038	0.094	0.08	3.39	0.12	0.15	0.01	0.0
11062-06	0.296	0.340	0.281	0.24	3.12	0.12	0.16	0.01	0.0
11062-07	0.778	1.226	0.594	1.48	3.28	0.12	0.16	0.01	0.0
11062-08	1.111	2.113	1.188	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED	NOT TESTED
11062-09	5.222	7.642	5.281	14.8	3.45	0.22	0.15	0.01	0.0
11062-10	7.667	15.226	7.656	26.3	3.33	0.38	0.15	0.01	0.0
11062-11	42.852	>37.736	44.063	136	3.13	4.49	0.13	0.01	0.0
11062-12	76.259	>37.736	88.344	364	3.20	20.8	0.09	0.01	0.0

Notes on Data:

- This panel is for **Research Use Only. Not for use in diagnostic procedures.**
- Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
- All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
- No preservatives have been added.
- All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 For Customer Support, please visit www.zeptometrix.com or email custserv@zeptometrix.com