



**HBV Seroconversion Panel Donor No.61066**  
**Part Number: HBV6275**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	ABBOTT HBV REAL TIME CP/ML	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM ELECSYS HBsAg	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3
		TEST DATE: 11/15/1996	TEST DATE: UNK	TEST DATE: 1/16/1997	TEST DATE: 10/9/1997	TEST DATE: 12/18/1996	TEST DATE: 12/16/1996
6275-01	02/06/96	8	98	0.15	0.41	0.44	0.06
6275-02	02/08/96	5	75	0.14	0.50	0.46	0.17
6275-03	02/13/96	4	<52	0.33	0.90	0.55	0.43
6275-04	02/15/96	5	163	0.40	0.90	0.51	0.70
6275-05	02/28/96	8	1670	3.15	6.48	1.29	5.68
6275-06	03/04/96	7	5926	8.54	17.21	2.78	16.41
6275-07	03/06/96	1	10064	11.15	28.41	3.53	23.36

PANEL MEMBER	MUREX HBsAg	SORIN ETI- MAK 3 HBsAg	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	ORTHO AMERLITE HBsAg II	ABBOTT AxSYM HBsAg	ABBOTT Austria II-125
	TEST DATE: 1/9/1997	TEST DATE: 1/16/1997	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 1/13/1997	TEST DATE: 1/6/1997	TEST DATE: 12/3/1996
6275-01	0.55	0.12	1.26	0.53	0.47	0.55	0.32
6275-02	0.54	0.25	1.07	0.58	0.09	0.54	0.32
6275-03	0.65	0.46	2.25	0.90	0.16	0.90	0.41
6275-04	0.57	0.63	2.15	0.99	0.20	0.79	0.39
6275-05	1.53	4.52	12.83	5.72	2.70	3.44	1.85
6275-06	3.38	12.4	31.50	15.59	9.94	8.88	4.64
6275-07	4.09	18.03	43.59	22.01	13.01	11.58	5.69

PANEL MEMBER	ABBOTT ANTI-HBc TOTAL (EIA) (Co/S)	ABBOTT HBeAg (EIA)	ABBOTT Auszyme MC Dynamic	ABBOTT Auszyme Monoclonal	ROCHE Cobas Core HBsAg II (EIA)	BEHRINGWERKE Enzygnost HBsAg Monoclonal II	DIASORIN HBc IgM
	TEST DATE: 12/17/1996	TEST DATE: 12/17/1996	TEST DATE: 1/21/1997	TEST DATE: 10/21/1997	TEST DATE: 12/17/1996	TEST DATE: 12/19/1996	TEST DATE: 10/5/2011
6275-01	0.523	0.314	0.36	0.15	0.25	0.45	0.17
6275-02	0.418	0.298	0.36	0.15	0.21	0.41	0.17
6275-03	0.550	0.264	0.79	0.37	0.57	0.40	0.17
6275-04	0.531	0.322	0.97	0.42	0.63	0.42	0.17
6275-05	0.499	0.355	8.61	4.75	5.27	1.99	0.17
6275-06	0.459	0.322	24.91	12.04	15.2	5.31	0.17
6275-07	0.515	0.380	31.55	21.26	22.12	7.28	0.17

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6275 Rev03  
 Effective Date: 06/23/2021



**HBV Seroconversion Panel Donor No.61066**  
**Part Number: HBV6275**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ORTHO VITROS ECi HBeAg TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBe TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 8/30/2018	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018
6275-01	407	0.09	2.36	0.16	0.21	0.02	0.0
6275-02	NOT TESTED	0.09	2.48	0.20	0.20	0.02	0.1
6275-03	NEGATIVE	0.23	2.58	0.19	0.21	0.02	0.0
6275-04	NEGATIVE	0.32	2.61	0.17	0.20	0.02	0.0
6275-05	NOT TESTED	5.26	2.61	0.19	0.20	0.02	0.0
6275-06	25500	18.4	2.69	0.18	0.20	0.02	0.4
6275-07	20500	26.3	2.55	0.18	0.20	0.01	0.0

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV6275 Rev03  
Effective Date: 06/23/2021