



**HBV Seroconversion Panel Donor No. 62347  
Part Number: HBV6284**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45u/L)	BIOMERIEUX VIDAS HBsAg	ABBOTT HBV REAL TIME CP/ML	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems	ROCHE Cobas Core HBsAg II EIA	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II
		TEST DATE: 1/26/1998	TEST DATE: 4/22/1998	TEST DATE: UNK	TEST DATE: 4/23/1998	TEST DATE: 10/31/1999	TEST DATE: 4/23/1998
6284-01	08/09/96	2	0.08	NOT DETECTED	0.27	0.83	0.29
6284-02	08/11/96	2	0.10	NOT DETECTED	0.36	0.80	0.36
6284-03	08/24/96	2	0.08	NOT DETECTED	0.33	0.17	0.39
6284-04	08/26/96	2	0.10	NOT DETECTED	0.31	0.06	0.35
6284-05	08/31/96	1	0.10	NOT DETECTED	0.31	0.30	0.35
6284-06	09/02/96	3	0.07	NOT DETECTED	0.33	0.13	0.36
6284-07	09/07/96	0	0.06	NOT DETECTED	0.31	0.65	0.34
6284-08	09/09/96	2	0.07	NOT DETECTED	0.33	0.91	0.33
6284-09	09/14/96	9	0.07	<52	0.31	0.07	0.35
6284-10	09/17/96	8	0.09	52	0.31	0.52	0.39
6284-11	09/21/96	8	0.13	290	0.45	0.28	0.43
6284-12	09/24/96	5	0.23	712	0.54	1.19/0.43	0.53
6284-13	09/28/96	2	0.87	1918	0.85	1.83/1.01	1.17
6284-14	10/02/96	0	3.40	4006	1.92	5.74	3.26
6284-15	10/05/96	4	6.95	20547	4.20	9.54	6.83
6284-16	10/09/96	1	24.75	142152	16.43	38.94	21.20
6284-17	10/12/96	*	71.74	459996	62.06	MAX.ABS.	MAX.ABS.
6284-18	10/16/96	3	100.71	1078337	192.72	MAX.ABS.	MAX.ABS.
6284-19	10/19/96	5	104.02	4099722	MAX.ABS.	MAX.ABS.	MAX.ABS.

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HBV Seroconversion Panel Donor No. 62347  
Part Number: HBV6284**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 3/18/1998	ORTHO VITROS ECi HBsAg TEST DATE: 8/30/2018	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: 2/4/1998	ABBOTT AxSYM Core-M TEST DATE: 4/7/1998	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 8/30/2018	ABBOTT ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 8/30/2018
6284-01	0.07	0.21	0.148	0.05	0.35	3.35	0.075	0.5
6284-02	0.01	0.06	0.205	0.05	<0.23	3.24	0.075	0.0
6284-03	0.01	0.09	0.159	0.05	0.39	3.31	0.075	0.0
6284-04	0.07	0.05	0.125	0.04	0.48	3.44	0.075	0.3
6284-05	0.13	0.06	0.136	0.05	<0.23	3.32	0.075	0.7
6284-06	0.13	0.06	0.273	0.05	0.39	3.32	0.075	0.2
6284-07	0.01	0.08	0.318	0.05	0.39	3.49	0.075	0.2
6284-08	0.07	0.06	0.307	0.05	0.52	3.61	0.075	0.3
6284-09	0.07	0.06	0.352	0.05	0.42	3.21	0.075	0.8
6284-10	0.01	0.27	0.284	0.05	0.39	3.18	0.075	0.3
6284-11	0.15	0.08	0.227	0.05	0.36	3.23	0.075	1.1
6284-12	0.38	0.14	0.284	0.05	0.39	3.42	0.075	1.3
6284-13	4.41	0.87	0.898	0.06	0.39	3.21	0.094	0.0
6284-14	4.27	4.65	2.955	0.06	0.29	3.29	0.094	1.4
6284-15	9.69	11.6	4.807	0.05	0.29	3.18	0.113	0.3
6284-16	34.84	47.0	13.943	0.06	0.51	3.54	0.094	0.0
6284-17	MAX.ABS.	189	25.557	0.06	0.42	3.38	0.094	0.6
6284-18	MAX.ABS.	604	30.466	0.05	0.39	3.14	0.094	0.7
6284-19	MAX.ABS.	1680	32.216	0.05	0.52	3.40	0.113	1.5

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HBV Seroconversion Panel Donor No. 62347**  
**Part Number: HBV6284**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi HBeAg	ABBOTT HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ABBOTT AxSYM Core	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	BEHRING-WERKE Enzygnost HBsAg Monoclonal II	BIOTEST ELISA HBsAg
	TEST DATE: 8/30/2018	TEST DATE: UNK	TEST DATE: 8/30/2018	TEST DATE: 8/30/2018	TEST DATE: 4/7/1998	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 5/4/1998	TEST DATE: 4/6/1998
6284-01	3.88	0.215	0.19	0.01	0.61	0.40	0.49	0.34	0.28
6284-02	0.21	0.269	0.19	0.01	0.64	0.38	0.48	0.27	0.26
6284-03	0.19	0.238	0.21	0.01	0.65	0.36	0.50	0.35	0.29
6284-04	0.20	0.200	0.17	0.01	0.65	0.40	0.50	0.37	0.33
6284-05	0.19	0.177	0.16	0.01	0.61	0.39	0.51	0.46	0.38
6284-06	0.20	0.231	0.17	0.01	0.60	0.34	0.51	0.54	0.32
6284-07	0.20	0.177	0.19	0.01	0.66	0.33	0.50	0.47	0.31
6284-08	0.20	0.169	0.18	0.01	0.67	0.33	0.49	0.38	0.27
6284-09	0.19	0.800	0.17	0.01	0.64	0.46	0.50	0.38	0.26
6284-10	0.19	0.269	0.18	0.01	0.64	1.51	0.50	0.41	0.27
6284-11	0.20	0.285	0.16	0.01	0.65	1.46	0.61	0.45	0.34
6284-12	0.24	0.223	0.16	0.01	0.65	2.02	0.72	0.55	0.40
6284-13	0.26	0.231	0.18	0.01	0.64	3.03	1.63	0.90	0.99
6284-14	0.21	0.300	0.17	0.01	0.63	10.56	5.54	2.05	2.13
6284-15	0.22	0.246	0.17	0.01	0.65	20.50	11.38	3.91	4.13
6284-16	0.97	0.385	0.17	0.01	0.71	70.24	29.83	13.99	12.71
6284-17	3.63	0.731	0.17	0.01	0.71	181.73	32.46	33.96	21.48
6284-18	22.5	2.030	0.11	0.01	0.70	310.05	32.61	36.35	23.91
6284-19	69.6	4.769	0.08	0.01	0.72	439.46	32.33	34.54	24.69

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HBV Seroconversion Panel Donor No. 62347  
Part Number: HBV6284**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	RESEARCH USE PCR (copies/ml)	ABBOTT AxSYM HBsAg	ABBOTT Auszyme MC Dynamic HBsAg	ABBOTT Auszyme Monoclonal HBsAg	SORIN ETI-MAK 3 HBsAg	SORIN (INCSTAR) HBsAg	SORIN ETI-Core IGMK2	SORIN ETI-AB CoreK2 (Co/S)
	TEST DATE: 6/2/1998	TEST DATE: 3/19/1998	TEST DATE: 3/31/1998	TEST DATE: 4/30/1998	TEST DATE: 5/5/1998	TEST DATE: UNK	TEST DATE: 5/11/1998	TEST DATE: 5/7/1998
6284-01	<100	0.56	0.03	0.30	0.17	0.117	0.50	0.20
6284-02	<100	0.46	0.07	0.20	0.24	0.117	0.48	0.19
6284-03	100	0.60	0.03	0.34	0.39	0.117	0.54	0.18
6284-04	<100	0.51	0.00	0.34	0.17	0.078	0.53	0.18
6284-05	<100	0.51	0.03	0.24	0.23	0.157	0.44	0.17
6284-06	500	0.47	0.03	0.30	0.07	0.157	0.44	0.18
6284-07	<100	0.51	0.16	0.20	0.37	0.157	0.47	0.19
6284-08	500	0.46	0.20	0.37	0.17	0.196	0.47	0.20
6284-09	500	0.49	0.23	0.27	0.09	0.157	0.46	0.19
6284-10	100	0.49	0.30	0.24	0.25	0.118	0.50	0.19
6284-11	550	0.62	0.20	0.40	0.29	0.118	0.50	0.17
6284-12	3400	0.64	0.40	0.44	0.63	0.118	0.46	0.18
6284-13	1000	1.01	1.48	0.67	1.29	0.314	0.42	0.18
6284-14	<100	2.06	5.18	1.82	7.71	0.694	0.45	0.17
6284-15	<100	4.07	12.03	3.61	6.92	1.463	0.39	0.19
6284-16	260000	27.67	44.97	14.22	22.48	5.294	0.48	0.19
6284-17	1100000	30.61	MAX.ABS.	53.19	MAX.ABS.	10.157	0.42	0.19
6284-18	3300000	60.41	MAX.ABS.	MAX.ABS.	MAX.ABS.	6.235	0.40	0.18
6284-19	2700000	72.96	MAX.ABS.	MAX.ABS.	MAX.ABS.	5.450	0.39	0.18

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)