



**HBV Seroconversion Panel Donor No.63568**  
**Part Number: HBV6288**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45uL)	ABBOTT HBV REAL TIME CP/ML	RESEARCH USE PCR	BIOMERIEUX VIDAS HBsAg	BOEHR. MANNHEIM HBsAg Ensymun-test ES Systems	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II	ORTHO HBsAg ELISA Testsystem 3	ROCHE Cobas Core HBsAg II EIA
		TEST DATE: 2/4/1998	TEST DATE: UNK	TEST DATE: 6/2/1998	TEST DATE: 10/25/1998	TEST DATE: 12/31/1998	TEST DATE: 7/28/1999	TEST DATE: 8/4/1998	TEST DATE: 11/30/1998
6288-01	03/16/97	11	108	500	0.27	0.38	0.33	0.38	0.19
6288-02	03/19/97	12	175	4000	0.15	0.85	0.37	0.15	0.20
6288-03	03/23/97	10	553	3200	0.29	0.45	0.44	0.35	0.41
6288-04	03/26/97	10	1078	6200	0.41	0.63	0.68	0.80	0.80
6288-05	03/30/97	12	2709	10000	0.65	0.94	0.97	1.19	1.85
6288-06	04/02/97	1	5530	24000	1.23	1.63	1.76	2.84	2.37
6288-07	04/06/97	8	5161	22000	2.12	2.46	2.84	4.94	4.31
6288-08	04/09/97	3	14546	44000	3.21	4.80	4.81	9.13	7.67
6288-09	04/13/97	10	22017	72000	4.95	6.72	6.42	12.74	9.52

PANEL MEMBER	SORIN ETI-MAK 3	SORIN (INCSTAR) HBsAg	ABBOTT PRISM HBsAg	ABBOTT MUREX HBsAg Version 3	SORIN ETI-Core IGMK2	SORIN ETI-AB CoreK2 (Co/S)	ABBOTT AxSYM Core- M	ABBOTT AxSYM Core (Co/S)	ABBOTT ANTI-HBc TOTAL
	TEST DATE: 6/23/1998	TEST DATE: UNK	TEST DATE: 8/27/2009	TEST DATE: 8/27/2009	TEST DATE: 5/7/1998	TEST DATE: 5/7/1998	TEST DATE: 4/7/1998	TEST DATE: 4/7/1998	TEST DATE: UNK
6288-01	0.11	0.173	0.93	0.54	0.01	0.18	0.09	0.51	0.45
6288-02	0.11	0.116	1.50	0.61	0.06	0.17	0.09	0.53	0.39
6288-03	0.32	0.173	1.72	0.72	0.00	0.18	0.09	0.53	0.42
6288-04	0.47	0.173	2.30	0.92	0.04	0.17	0.09	0.53	0.39
6288-05	0.70	0.347	4.61	1.70	0.00	0.15	0.08	0.53	0.48
6288-06	1.67	0.462	6.32	2.90	0.00	0.16	0.07	0.49	0.42
6288-07	2.68	0.893	12.96	6.05	0.00	0.17	0.08	0.51	0.45
6288-08	4.08	1.676	19.06	9.06	0.00	0.18	0.07	0.51	0.37
6288-09	5.85	2.023	25.21	12.56	0.00	0.17	0.08	0.51	Not Tested

PANEL MEMBER	GENETIC SYSTEMS (2.0)	ORTHO VITROS ECi HBsAg	ABBOTT ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBc TOTAL	ORTHO VITROS ECi HBeAg	ORTHO VITROS ECi ANTI-HBe	ORTHO VITROS ECi ANTI-HBc IgM	ORTHO VITROS ECi ANTI-HBs QUANT
	TEST DATE: 2/6/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6288-01	0.167	0.05	0.091	3.37	0.25	0.18	0.03	0.90
6288-02	0.157	0.06	0.091	3.49	0.25	0.15	0.02	0.00
6288-03	0.213	0.1	0.091	3.49	0.24	0.18	0.04	0.00
6288-04	0.324	0.21	0.091	3.23	0.25	0.16	0.03	0.4
6288-05	0.639	1.1	0.091	3.22	0.28	0.17	0.04	0.00
6288-06	1.019	2.22	0.073	3.15	0.27	0.16	0.03	0.00
6288-07	1.907	5.62	0.091	3.06	0.36	0.15	0.05	0.00
6288-08	2.546	9.44	0.109	3.15	0.61	0.15	0.06	0.20
6288-09	4.028	12.5	Not Tested	3.25	1.13	0.14	0.07	1.1

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)