

ZeptoMetrix®

HBV Seroconversion Panel Donor No.63997 Part Number: HBV6290

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: 8/12/1997	ABBOTT HBV REAL TIME CP/ML TEST DATE: UNK	ROCHE CAP-G/CTM HBV (cp/mL) TEST DATE: 6/2/2005	RESEARCH USE PCR (copies/ml) TEST DATE: 6/13/1998	GENETIC SYSTEMS HBsAg (2.0) TEST DATE: 2/6/1998	ABBOTT Auszyme MC Dynamic HBsAg TEST DATE: 4/6/1998	ABBOTT Auszyme Monoclonal HBsAg TEST DATE: 5/5/1998	BEHRINGWERKE Enzygnost HBsAg Monoclonal II TEST DATE: 5/4/1998
6290-01	04/21/97	4	<52	NEGATIVE	<100	0.148	0.20	0.37	0.26
6290-02	04/23/97	9	<52	NEGATIVE	<100	0.194	0.07	0.31	0.26
6290-03	04/28/97	3	101	NEGATIVE	500	0.222	0.03	0.31	0.26
6290-04	04/30/97	2	152	NEGATIVE	500	0.185	0.07	0.40	0.27
6290-05	05/05/97	9	179	231	500	0.148	0.07	0.49	0.32
6290-06	05/07/97	9	635	755	1000	0.333	0.16	0.49	0.35
6290-07	05/12/97	6	1559	2250	3200	0.417	0.75	0.49	0.49
6290-08	05/14/97	5	1918	6580	3000	0.537	1.43	0.98	0.63
6290-09	05/19/97	11	7124	16700	5200	1.194	4.47	2.08	1.52
6290-10	05/21/97	10	10298	NOT TESTED	6100	2.065	7.60	3.00	2.12
6290-11	05/28/97	13	78119	158000	64000	8.000	39.42	16.04	11.11
6290-12	06/01/97	9	187393	297000	87000	12.370	MAX ABS.	36.24	21.93

PANEL MEMBER	ABBOTT PRISM HBsAg TEST DATE: UNK	ABBOTT AxSYM HBsAg TEST DATE: 3/19/1998	ORTHO VITROS Eci HBsAg TEST DATE: 6/2/2016	SORIN (INCSTAR) HBsAg TEST DATE: UNK	BIOMERIEUX VIDAS HBsAg TEST DATE: 4/30/1998	BIOTEST ELISA HBsAg TEST DATE: 4/6/1998	BOEHR. MANNHEIM HBsAg Enzymun-test ES Systems TEST DATE: 4/27/1998	ORGANON TEKNIKA Hepanostika HBsAg Uni-Form II TEST DATE: 4/23/1998	ABBOTT MUREX HBsAg Version 3 TEST DATE: UNK
6290-01	0.76	0.54	0.07	0.116	0.37	0.28	0.05	0.47	0.51
6290-02	0.90	0.53	0.07	0.173	0.32	0.28	0.13	0.49	0.52
6290-03	1.04	0.60	0.07	0.231	0.25	0.28	0.05	0.46	0.53
6290-04	1.01	0.50	0.08	0.231	0.27	0.22	0.00	0.46	0.58
6290-05	1.01	0.56	0.10	0.231	0.33	0.29	0.05	0.40	0.72
6290-06	1.64	0.66	0.15	0.173	0.40	0.31	0.10	0.53	0.95
6290-07	3.10	0.97	0.75	0.347	0.76	0.41	0.40	0.82	1.77
6290-08	4.45	1.17	1.44	0.520	1.06	0.63	0.85	1.01	2.57
6290-09	10.80	2.78	5.50	1.214	2.69	1.36	2.22	2.17	5.91
6290-10	15.32	3.55	8.65	1.850	3.45	2.10	3.65	2.82	8.93
6290-11	73.04	15.61	53.4	6.994	23.20	11.30	21.55	17.75	30.85
6290-12	114.31	25.26	115	13.237	43.03	21.56	48.58	29.29	31.50

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6290 Rev03
Effective Date: 06/24/2021



HBV Seroconversion Panel Donor No.63997
Part Number: HBV6290

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO HBsAg ELISA Testsystem 3 TEST DATE: 3/18/1998	ROCHE Cobas Core HBsAg II EIA TEST DATE: 4/27/1998	ABBOTT AxSYM Core-M TEST DATE: 4/7/1998	SORIN ETI-Core IGMK2 TEST DATE: 5/11/1998	SORIN ETI-AB CoreK2 (Co/S) TEST DATE: 5/7/1998	ABBOTT ANTI-HBc TOTAL TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc TOTAL TEST DATE: 6/2/2016	ORTHO VITROS ECi HBeAg TEST DATE: 6/2/2016
6290-01	0.29	0.24	0.16	0.31	0.36	0.61	1.53	0.26
6290-02	0.24	0.56	0.14	0.27	0.37	0.61	2.06	0.25
6290-03	0.10	0.43	0.15	0.37	0.35	0.61	2.00	0.24
6290-04	0.04	0.67	0.14	0.29	0.31	0.56	2.31	0.24
6290-05	0.32	0.05	0.14	0.28	0.30	0.56	2.15	0.24
6290-06	0.32	0.72	0.14	0.35	0.29	0.52	2.24	0.27
6290-07	0.74	1.39/0.71	0.14	0.21	0.26	0.61	2.26	0.27
6290-08	1.58	2.11/1.19	0.14	0.38	0.29	0.48	2.32	0.26
6290-09	4.32	3.76	0.13	0.34	0.32	0.56	2.30	0.27
6290-10	6.62	5.63	0.13	0.31	0.29	0.48	2.32	0.28
6290-11	34.34	18.56	0.16	0.37	0.36	0.52	2.27	0.48
6290-12	66.70	MAX ABS	0.15	0.37	0.32	0.56	2.46	1.26

PANEL MEMBER	ABBOTT HBeAg TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBe TEST DATE: 6/2/2016	ABBOTT ANTI-HBc IgM TEST DATE: UNK	ORTHO VITROS ECi ANTI-HBc IgM TEST DATE: 6/2/2016	ORTHO VITROS ECi ANTI-HBs QUANT TEST DATE: 6/2/2016
6290-01	0.213	0.18	0.255	0.09	0.00
6290-02	0.236	0.16	0.255	0.06	0.90
6290-03	Not Tested	0.16	0.255	0.06	1.10
6290-04	0.247	0.16	0.236	0.06	0.00
6290-05	0.270	0.16	0.255	0.07	0.00
6290-06	0.180	0.16	0.255	0.06	0.10
6290-07	0.236	0.16	0.236	0.06	0.10
6290-08	0.225	0.16	0.218	0.06	0.10
6290-09	0.213	0.15	0.236	0.05	0.00
6290-10	0.258	0.15	0.200	0.05	0.70
6290-11	0.315	0.15	0.200	0.07	0.50
6290-12	0.506	0.15	0.218	0.06	0.50

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHBV6290 Rev03
 Effective Date: 06/24/2021