



**HBV Seroconversion Panel Donor No.1087380  
Part Number: HBV9093**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ABBOTT HBV REAL TIME CP/ML	VENDOR HBsAg	DIASORIN HBsAg	ORTHO VITROS ECi HBsAg	ORTHO VITROS ECi ANTI-HBc TOTAL
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 2/16/2010	TEST DATE: 5/18/2016	TEST DATE: 5/18/2016
9093-01	06/28/07	<52	NEGATIVE	0.60	0.05	3.15
9093-02	07/03/07	383	NEGATIVE	0.20	0.09	3.34
9093-03	07/05/07	1182	NEGATIVE	0.40	0.07	3.37
9093-04	07/10/07	4293	NEGATIVE	0.60	0.82	3.22
9093-05	07/12/07	6205	NEGATIVE	1.40	2.12	3.63
9093-06	07/17/07	46000	POSITIVE	5.60	20.2	3.49
9093-07	07/19/07	93919	POSITIVE	7.60	34.1	3.37
9093-08	07/24/07	729045	POSITIVE	31.40	190	3.23
9093-09	08/02/07	64976210	POSITIVE	60.80	5370	2.6
9093-10	08/09/07	3039165699	POSITIVE	60.40	5450	0.93
9093-11	08/16/07	3039165699	POSITIVE	60.40	1090	0.07
9093-12	08/23/07	<3410000000	POSITIVE	60.20	507	0.11
9093-13	09/10/07	1182372660	POSITIVE	59.60	2050	0.04
9093-14	09/12/07	1129157124	POSITIVE	60.20	2020	0.04
9093-15	10/11/07	22016809	POSITIVE	59.20	4730	0.06
9093-16	10/15/07	3109957	POSITIVE	59.80	6670	0.06
9093-17	10/22/07	64976	POSITIVE	60.00	7770	0.05
9093-18	10/29/07	5161	POSITIVE	60.40	5990	0.04
9093-19	11/01/07	3915	POSITIVE	60.40	4630	0.07
9093-20	11/14/07	2305	POSITIVE	61.60	830	0.07
9093-21	11/27/07	133	POSITIVE	34.80	80.8	0.07
9093-22	11/29/07	4006	POSITIVE	15.00	44.9	0.07
9093-23	12/04/07	5926	POSITIVE	3.20	14	0.07
9093-24	12/06/07	2902	POSITIVE	2.00	5.83	0.07
9093-25	12/11/07	2772	NEGATIVE	14.40	3.83	0.07
9093-26	12/13/07	2359	NEGATIVE	0.60	3.22	0.05
9093-27	12/18/07	983	NEGATIVE	0.80	0.68	0.04
9093-28	12/27/07	781	NEGATIVE	0.60	0.2	0.04
9093-29	12/31/07	983	NEGATIVE	0.80	0.63	0.03
9093-30	01/03/08	1749	NEGATIVE	0.20	0.61	0.04
9093-31	01/07/08	2103	NEGATIVE	0.20	0.47	0.03

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HBV Seroconversion Panel Donor No.1087380  
Part Number: HBV9093**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS ECi ANTI-HBc IgM  TEST DATE: 5/18/2016	ORTHO VITROS ECi HBeAg  TEST DATE: 5/18/2016	ORTHO VITROS ECi ANTI-HBe  TEST DATE: 5/18/2016	ORTHO VITROS ECi ANTI-HBs QUANT  TEST DATE: 5/18/2016	DIASORIN HBsAb  TEST DATE: 2/17/2010	DIASORIN HBc TOTAL  TEST DATE: 3/31/2010	DIASORIN HBc IgM  TEST DATE: 4/1/2010
9093-01	0.03	0.21	0.15	0.7	0.08	1.52	0.20
9093-02	0.03	0.24	NOT TESTED	1.5	0.03	1.32	0.20
9093-03	0.03	0.24	0.15	0.8	0.08	1.20	0.20
9093-04	0.02	0.24	0.17	1.7	0.03	1.60	0.16
9093-05	0.02	0.24	0.15	1.8	0.08	1.84	0.16
9093-06	0.03	0.35	NOT TESTED	0.4	0.02	2.52	0.16
9093-07	0.02	0.65	0.2	1.5	0.08	2.20	0.16
9093-08	0.03	11.6	NOT TESTED	0.7	0.02	1.80	0.16
9093-09	0.02	694	6380	0.8	0.08	1.52	0.20
9093-10	NOT TESTED	3010	0.00	NOT TESTED	0.08	0.68	0.20
9093-11	1.68	2280	0.00	0.00	0.03	0.12	1.60
9093-12	NOT TESTED	2420	NOT TESTED	NOT TESTED	0.03	0.16	1.16
9093-13	3.89	2820	0.00	1.9	0.08	0.08	8.48
9093-14	3.79	2450	0.00	1.1	0.03	0.12	7.88
9093-15	2.99	2870	0	3	0.03	0.12	5.32
9093-16	2.88	2730	0	1.8	0.03	0.04	5.28
9093-17	2.68	659	0.01	2.3	0.08	0.04	5.32
9093-18	2.15	30.4	1.1	0.6	0.08	0.04	2.24
9093-19	1.99	16.7	0.14	0.7	0.08	0.04	2.68
9093-20	0.92	6.96	0.17	1.00	0.08	0.04	1.24
9093-21	0.58	2.61	0.24	1.2	0.08	0.04	0.76
9093-22	0.46	1.93	0.23	0.8	0.08	0.04	0.76
9093-23	0.4	1.48	0.25	1.6	0.08	0.04	0.56
9093-24	0.33	1.1	0.27	1.00	0.08	0.08	0.64
9093-25	0.3	0.54	0.32	0.3	0.08	0.08	0.60
9093-26	0.27	0.46	0.31	1.5	0.08	0.08	0.44
9093-27	0.26	0.31	0.5	1.2	0.08	0.08	0.44
9093-28	0.29	0.24	NOT TESTED	4.9	0.08	0.08	0.44
9093-29	0.25	0.27	1.09	2.3	0.08	0.12	0.36
9093-30	0.19	0.23	1.17	1.6	0.08	0.08	0.32
9093-31	0.20	0.24	NOT TESTED	1.6	0.08	0.12	0.40

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV9093 Rev04  
Effective Date: 09/23/2021