



**HBV Seroconversion Panel Donor No.75255**  
**Part Number: HBV9099**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	(QUEST) ROCHE REAL TIME PCR IU/mL  TEST DATE: UNK	FUJIREBIO LUMIPULSE G HBsAg-QUANT (233122) G1200 miU/MI  TEST DATE: UNK	ROCHE HBsAg II (04687787190) COBAS e411 COi  TEST DATE: UNK	DIASORIN MUREX HBsAg Version 3 (GE36) S/CO  TEST DATE: UNK	SIEMENS ENZYGNOST HBsAg 6.0 (OPFM055)  TEST DATE: UNK	GENERAL BIOLOGICALS SURASE B-96 (TMB) (4SGE3) BEP III S/CO  TEST DATE: UNK	ABBOTT ARCHITECT HBsAg QUALITATIVE II (2G22) i200 S/CO  TEST DATE: UNK
9099-01	11/21/99	<20	1.70	0.42	0.55	0.17	0.86	0.23
9099-02	11/26/99	<20	2.10	0.36	0.50	0.20	INDETERMINATE	0.24
9099-03	11/30/99	35	3.80	0.46	0.58	0.36	0.94	0.32
9099-04	12/03/99	194	7.20	0.55	0.61	0.49	INDETERMINATE	0.51
9099-05	12/07/99	354	30.30	0.88	0.76	1.09	1.00	1.13
9099-06	12/10/99	1824	65.40	1.78	1.38	2.65	1.82	2.63
9099-07	12/14/99	3229	79.00	1.91	1.52	3.05	1.74	2.47
9099-08	12/19/99	19463	520.20	10.01	6.82	17.31	6.92	16.12
9099-09	12/23/99	51948	2285.40	43.76	25.93	53.33	27.65	66.42
9099-10	12/26/99	72900	3494.70	90.69	36.13	53.33	56.80	143.48
9099-11	12/30/99	184898	8842.60	178.70	35.98	53.33	69.78	304.03
9099-12	01/02/00	294788	13971.90	306.80	39.44	53.33	78.16	485.43
9099-13	01/06/00	504414	34514.70	741.60	38.69	53.33	83.59	1167.66
9099-14	01/11/00	1727717	120000	2803.00	37.44	53.33	83.08	3723.32
9099-15	01/21/00	111900000	150000	10457.00	37.59	53.33	80.71	6130.61
9099-16	02/03/00	1390000000	150000	807.40	38.88	53.33	79.86	995.64
9099-17	02/07/00	940000000	150000	677.90	39.03	53.33	81.18	859.64
9099-18	02/11/00	584000000	150000	729.40	38.95	53.33	81.82	829.64
9099-19	02/15/00	336000000	150000	777.30	37.53	53.33	82.35	907.15
9099-20	02/18/00	169000000	150000	1070.00	38.94	49.40	79.55	1233.29

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV9099 Rev04  
 Effective Date: 06/14/2021



**HBV Seroconversion Panel Donor No.75255**  
**Part Number: HBV9099**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO VITROS Eci HBsAg	SIEMENS IMMULITE ANTI-HBc AG (LKHC) COI	SIEMENS ADVIA CENTAUR ANTI-HBC TOTAL (7566733)	ORTHO VITROS Eci ANTI-HBc TOTAL	ROCHE COBAS ANTI-HBc (11820559) e411 COI	ORTHO VITROS Eci ANTI-HBc IgM	ORTHO VITROS Eci ANTI-HBs QUANT	ORTHO VITROS Eci HBeAg	ORTHO VITROS Eci ANTI-HBe
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
9099-01	0.05	0.45	0.14	3.22	1.91	0.01	2.1	0.17	0.23
9099-02	0.09	0.44	0.14	2.78	1.87	0.01	0.6	0.10	0.17
9099-03	0.09	0.46	0.14	2.89	1.83	0.01	1.0	0.08	0.16
9099-04	0.08	0.44	0.15	0.01	2.01	0.01	2.8	0.14	0.23
9099-05	0.25	0.47	0.15	2.84	1.91	0.01	1.2	0.09	0.17
9099-06	0.72	0.43	0.15	3.30	1.97	0.01	2.9	0.17	0.23
9099-07	0.89	0.46	0.15	2.95	1.86	0.01	1.6	0.11	0.16
9099-08	7.92	0.46	0.15	3.51	1.87	0.01	2.6	0.29	0.25
9099-09	40.8	0.46	0.15	3.07	1.74	0.01	1.6	0.51	0.18
9099-10	76.2	0.50	0.16	3.38	1.74	0.01	2.3	0.80	0.23
9099-11	145	0.46	0.16	3.01	1.72	0.01	2.2	2.33	0.21
9099-12	233	0.47	0.16	3.43	1.72	0.01	3.6	4.71	0.19
9099-13	503	0.47	0.16	3.11	1.75	0.01	2.7	20.3	0.13
9099-14	2010	0.46	0.17	3.46	1.84	0.01	2.4	132	0.05
9099-15	4980	4.46	4.15	0.11	0.99	0.20	1.5	2790	0.00
9099-16	318	4.59	3.28	0.17	INDETERMINATE	0.34	1.4	1390	0.01
9099-17	209	7.82	>8.0	0.07	0.11	1.52	2.1	969	0.01
9099-18	208	11.45	>8.0	0.04	0.01	3.02	1.9	1030	0.01
9099-19	315	13.19	>8.0	0.02	0.01	3.13	1.3	1260	0.01
9099-20	227	16.15	>8.0	0.02	0.01	3.05	2.3	1110	0.01

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for Anti-HIV1/2, HIV p24 Ag, Anti HCV by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHBV9099 Rev04  
 Effective Date: 06/14/2021