



**HCV Seroconversion Panel Donor No. 65470  
Part Number: HCV10010**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ALPHA HCV by PCR TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 9/7/1999	ABBOTT ANTI-HCV 3.0 Lab # 5 TEST DATE: 9/21/2000	ABBOTT PRISM HCV Lab # 5 TEST DATE: 10/4/2000
10010-01	07/27/97	5	POSITIVE	0.150	1.844	0.333	0.170	0.220
10010-02	07/29/97	4	POSITIVE	0.141	3.034	0.097	0.170	0.260
10010-03	08/03/97	6	POSITIVE	0.105	0.296	0.057	0.270	0.060
10010-04	08/07/97	6	POSITIVE	0.174	5.412	0.041	0.120	0.280
10010-05	08/10/97	3	POSITIVE	0.128	3.429	0.038	0.170	0.350
10010-06	08/14/97	0	POSITIVE	0.140	5.088	0.039	0.170	0.350
10010-07	08/17/97	6	POSITIVE	0.128	3.371	0.029	0.170	0.400

PANEL MEMBER	ABBOTT AxSYM ANTI-HCV 3.0 Lab # 5 TEST DATE: 7/20/2000	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ROCHE Cobas Core ANTI-HCV EIA TEST DATE: 9/6/2000	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5 TEST DATE: 9/18/2000	ORTHO Eci HBsAg TEST DATE: 11/16/2018	ORTHO Eci ANTI-HCV TEST DATE: 11/16/2018	ORTHO Eci ANTI-HIV 1/2 TEST DATE: 11/16/2018
10010-01	0.120	0.190	0.352	0.380	0.010	0.05	0.19	0.13
10010-02	0.140	0.160	0.298	0.330	0.010	0.05	0.14	0.10
10010-03	0.200	0.090	0.102	0.090	0.010	0.05	0.01	0.12
10010-04	0.210	0.280	0.434	0.210	0.000	0.06	0.39	0.11
10010-05	0.240	0.210	0.431	0.290	0.010	0.05	0.20	0.10
10010-06	0.230	0.210	0.480	0.300	0.020	0.05	0.22	0.11
10010-07	0.320	0.230	0.601	0.020	0.010	0.05	0.22	0.10

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)