



**HCV Seroconversion Panel Donor No. 65064  
Part Number: HCV10017**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)  TEST DATE: UNK	ALPHA HCV by PCR  TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0  TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo  TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1  TEST DATE: UNK	ABBOTT ANTI-HCV 3.0 Lab # 5  TEST DATE: 7/31/2000
10017-01	08/28/97	2	NEGATIVE	0.134	0.296	0.237	0.300
10017-02	08/30/97	0	NEGATIVE	0.125	0.293	0.237	0.290
10017-03	09/08/97	4	NEGATIVE	0.120	0.313	0.237	0.200
10017-04	09/10/97	4	NOT TESTED	0.117	0.320	0.211	0.180
10017-05	09/15/97	0	NEGATIVE	0.107	0.303	0.211	0.190
10017-06	09/17/97	3	NEGATIVE	0.110	0.323	0.184	0.160
10017-07	09/22/97	9	NOT TESTED	0.119	0.306	0.211	0.250
10017-08	10/01/97	0	POSITIVE	0.128	0.299	0.211	0.120
10017-09	10/03/97	4	POSITIVE	0.122	2.282	0.211	0.220
10017-10	10/08/97	10	POSITIVE	0.135	10.966	0.211	0.250
10017-11	10/12/97	11	POSITIVE	0.125	6.327	0.211	0.280
10017-12	10/16/97	10	POSITIVE	0.237	11.122	0.237	0.240
10017-13	10/19/97	4	POSITIVE	0.204	8.609	0.211	0.250
10017-14	10/23/97	10	POSITIVE	0.230	10.531	0.211	0.170
10017-15	10/26/97	12	POSITIVE	0.186	10.500	0.237	0.220
10017-16	11/03/97	7	POSITIVE	3.809	9.446	1.553	1.840

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HCV Seroconversion Panel Donor No. 65064  
Part Number: HCV10017**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5  TEST DATE: 8/10/2000	ABBOTT Architect ANTI-HCV  TEST DATE: UNK	ABBOTT PRISM ANTI-HCV  TEST DATE: UNK	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5  TEST DATE: 10/16/2000	ORTHO ECi HBsAg  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½  TEST DATE: 11/16/2018
10017-01	0.110	0.110	0.173	0.000	0.06	0.02	0.14
10017-02	0.160	0.100	0.132	0.000	0.06	0.02	0.14
10017-03	0.080	0.080	0.139	0.000	0.06	0.02	0.15
10017-04	0.120	0.090	0.122	0.000	0.05	0.01	0.15
10017-05	0.120	0.080	0.188	0.000	0.06	0.01	0.15
10017-06	0.140	0.080	0.117	0.000	0.05	0.01	0.15
10017-07	0.100	0.080	0.124	0.000	0.06	0.01	0.13
10017-08	0.100	0.100	0.111	0.000	0.06	0.01	0.14
10017-09	0.110	0.100	0.145	0.000	0.06	0.02	0.13
10017-10	0.100	0.100	0.221	0.000	0.05	0.01	0.16
10017-11	0.110	0.080	0.119	0.000	0.06	0.01	0.17
10017-12	0.160	0.130	0.139	0.000	0.05	0.03	0.16
10017-13	0.150	0.100	0.182	0.000	0.05	0.03	0.16
10017-14	0.130	0.130	0.153	0.000	0.06	0.03	0.15
10017-15	0.130	0.090	0.117	0.000	0.05	0.02	0.17
10017-16	0.430	1.080	0.549	0.460	0.06	4.71	0.15

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)