



**HCV Seroconversion Panel Donor No. 68470
Part Number: HCV10021**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV 3.0 Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/27/1999	TEST DATE: 7/31/2000
10021-01	08/02/98	11	NEGATIVE	0.114	0.732	0.148	0.090
10021-02	08/09/98	24	NEGATIVE	0.124	1.607	0.131	0.130
10021-03	08/11/98	25	NEGATIVE	0.124	1.844	0.148	0.110
10021-04	08/16/98	14	NEGATIVE	0.118	1.380	0.148	0.110
10021-05	08/18/98	13	NEGATIVE	0.120	2.139	0.115	0.100
10021-06	08/23/98	14	NEGATIVE	0.112	1.268	0.131	0.110
10021-07	08/30/98	16	NEGATIVE	0.126	1.067	0.082	0.100
10021-08	09/06/98	15	POSITIVE	0.127	1.546	0.148	0.100
10021-09	09/08/98	12	POSITIVE	0.111	5.041	0.148	0.110
10021-10	09/13/98	24	POSITIVE	0.114	4.719	0.131	0.120
10021-11	09/15/98	21	POSITIVE	0.102	4.759	0.115	QNS
10021-12	09/27/98	23	POSITIVE	0.118	10.129	0.148	0.100
10021-13	09/30/98	14	POSITIVE	0.130	11.688	0.131	0.110
10021-14	10/04/98	30	POSITIVE	0.132	11.258	0.131	0.070

PANEL MEMBER	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack Lab # 5	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: 8/10/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/16/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10021-01	0.120	0.060	0.151	0.000	0.05	0.01	0.11
10021-02	0.250	0.070	0.136	0.000	0.06	0.01	0.12
10021-03	0.110	0.090	0.163	0.000	0.06	0.01	0.12
10021-04	0.170	0.070	0.147	0.000	0.05	0.01	0.13
10021-05	0.110	0.080	0.166	0.000	0.06	0.01	0.12
10021-06	0.100	0.070	0.169	0.000	0.06	0.01	0.16
10021-07	0.170	0.070	0.143	0.000	0.06	0.01	0.15
10021-08	0.100	0.080	0.145	0.000	0.06	0.01	0.14
10021-09	0.150	0.060	0.145	0.000	0.06	0.01	0.15
10021-10	0.110	0.060	0.134	0.000	0.05	0.01	0.11
10021-11	QNS	0.060	0.136	QNS	0.05	0.01	0.12
10021-12	0.100	0.070	0.156	0.000	0.05	0.01	0.15
10021-13	0.130	0.080	0.134	0.000	0.06	0.01	0.14
10021-14	0.160	0.130	0.169	0.000	0.06	0.02	0.14

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10021 Rev04
Effective Date: 10/12/2021