



**HCV Seroconversion Panel Donor No. 66801
Part Number: HCV10025**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45 u/L) | ALPHA HCV by PCR | ABBOTT MUREX ANTI-HCV 4 0 | ABBOTT MUREX HCV Ag/Ab Combo | ABBOTT ANTI-HCV Lab # 1 | ABBOTT AxSYM ANTI-HCV3.0 Reagent Pack Lab # 5 |
|--------------|--------------|------------------------------|------------------|---------------------------|------------------------------|-------------------------|---|
| | | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 10/25/1999 | TEST DATE: UNK |
| 10025-01 | 02/19/98 | 11 | NEGATIVE | 0.123 | 0.303 | 0.135 | 0.170 |
| 10025-02 | 02/24/98 | 9 | NEGATIVE | 0.150 | 0.303 | 0.162 | 0.200 |
| 10025-03 | 02/26/98 | 11 | NEGATIVE | 0.123 | 0.286 | 0.162 | 0.150 |
| 10025-04 | 03/03/98 | 8 | NEGATIVE | 0.119 | 0.354 | 0.081 | 0.240 |
| 10025-05 | 03/05/98 | 8 | NEGATIVE | 0.104 | 0.327 | 0.135 | 0.180 |
| 10025-06 | 03/12/98 | 10 | NEGATIVE | 0.119 | 0.337 | 0.162 | 0.140 |
| 10025-07 | 03/16/98 | 7 | NEGATIVE | 0.120 | 0.316 | 0.135 | 0.170 |
| 10025-08 | 03/19/98 | 7 | POSITIVE | 0.120 | 0.871 | 0.135 | 0.170 |
| 10025-09 | 03/31/98 | 23 | POSITIVE | 0.131 | 10.350 | 0.054 | 0.210 |
| 10025-10 | 04/13/98 | 51 | POSITIVE | 0.126 | 9.541 | 0.162 | 0.280 |
| 10025-11 | 04/16/98 | 132 | POSITIVE | 0.162 | 6.612 | 0.162 | 0.290 |

| PANEL MEMBER | ABBOTT Architect ANTI-HCV | ABBOTT Prism ANTI-HCV | SANOFI Monolisa ANTI-HCV PLUS Lab # 5 | ORTHO ECi HBsAg | ORTHO ECi ANTI-HCV | ORTHO ECi ANTI-HIV 1/2 |
|--------------|---------------------------|-----------------------|---------------------------------------|-----------------------|-----------------------|------------------------|
| | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 10/20/2000 | TEST DATE: 11/16/2018 | TEST DATE: 11/16/2018 | TEST DATE: 11/16/2018 |
| 10025-01 | 0.060 | 0.120 | 0.200 | 0.07 | 0.03 | 0.10 |
| 10025-02 | 0.080 | 0.102 | 0.180 | 0.08 | 0.05 | 0.09 |
| 10025-03 | 0.060 | 0.129 | 0.180 | 0.07 | 0.02 | 0.09 |
| 10025-04 | 0.070 | 0.118 | 0.200 | 0.07 | 0.02 | 0.09 |
| 10025-05 | 0.050 | 0.154 | 0.180 | 0.07 | 0.02 | 0.09 |
| 10025-06 | 0.060 | 0.124 | 0.180 | 0.07 | 0.02 | 0.09 |
| 10025-07 | 0.050 | 0.119 | 0.180 | 0.06 | 0.02 | 0.09 |
| 10025-08 | 0.060 | 0.118 | 0.190 | 0.07 | 0.02 | 0.09 |
| 10025-09 | 0.070 | 0.171 | 0.210 | 0.07 | 0.03 | 0.10 |
| 10025-10 | 0.060 | 0.149 | 0.180 | 0.07 | 0.03 | 0.10 |
| 10025-11 | 0.150 | 0.175 | 0.220 | 0.06 | 0.05 | 0.09 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com