



**HCV Seroconversion Panel Donor No.66074
Part Number: HCV10045**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV 3.0 Lab # 5
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/28/1999	TEST DATE: 10/16/2000
10045-01	01/12/98	13	POSITIVE	0.178	0.540	0.412	0.330
10045-02	01/14/98	8	POSITIVE	0.176	0.474	0.392	0.250
10045-03	01/20/98	43	POSITIVE	0.151	0.415	0.431	0.330
10045-04	01/22/98	45	POSITIVE	0.152	0.463	0.373	0.290
10045-05	01/27/98	88	POSITIVE	0.122	0.443	0.314	0.170

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack Lab # 5	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: 10/17/2000	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 9/29/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10045-01	0.280	0.090	0.118	0.010	0.07	0.02	0.08
10045-02	0.240	0.070	0.084	0.010	0.07	0.02	0.08
10045-03	0.290	0.060	0.105	0.010	0.07	0.02	0.09
10045-04	0.270	0.060	0.098	0.000	0.07	0.02	0.08
10045-05	0.240	0.060	0.098	0.000	0.06	0.02	0.08

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV10045 Rev04
Effective Date: 10/14/2021