



**HCV Seroconversion Panel Donor No.67766  
Part Number: HCV10050**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1b**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/19/1999
10050-01	05/01/98	1	NEGATIVE	0.154	0.387	0.179
10050-02	05/04/98	8	NEGATIVE	0.167	0.394	0.154
10050-03	05/08/98	15	NEGATIVE	0.143	0.394	0.154
10050-04	05/12/98	16	NEGATIVE	0.139	0.359	0.179
10050-05	05/15/98	10	NEGATIVE	0.160	0.380	0.154
10050-06	05/19/98	10	NEGATIVE	0.131	0.380	0.154
10050-07	05/22/98	8	NEGATIVE	0.151	0.404	0.154
10050-08	05/31/98	10	NEGATIVE	0.158	0.355	0.179
10050-09	06/03/98	9	NEGATIVE	0.160	0.373	0.154
10050-10	06/08/98	13	NEGATIVE	0.139	0.369	0.128
10050-11	06/11/98	12	NEGATIVE	0.139	0.359	0.128
10050-12	06/22/98	21	POSITIVE	0.148	0.666	0.128
10050-13	06/26/98	18	POSITIVE	0.136	0.394	0.128
10050-14	07/01/98	18	POSITIVE	0.134	0.488	0.128
10050-15	07/03/98	21	POSITIVE	0.137	0.411	0.103

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/4/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10050-01	0.060	0.121	0.010	0.07	0.03	0.09
10050-02	0.060	0.157	0.010	0.07	0.03	0.08
10050-03	0.060	0.528	0.010	0.07	0.02	0.08
10050-04	0.050	0.118	0.010	0.07	0.02	0.08
10050-05	0.050	0.983	0.010	0.07	0.02	0.08
10050-06	0.040	0.108	0.010	0.06	0.01	0.08
10050-07	0.050	0.129	0.000	0.07	0.02	0.08
10050-08	0.050	0.136	0.010	0.08	0.02	0.08
10050-09	0.050	0.091	0.010	0.07	0.02	0.08
10050-10	0.060	0.173	0.010	0.07	0.02	0.08
10050-11	0.050	0.129	0.010	0.07	0.02	0.09
10050-12	0.050	0.142	0.010	0.07	0.03	0.09
10050-13	0.060	0.088	0.010	0.07	0.03	0.08
10050-14	0.050	0.151	0.000	0.07	0.03	0.09
10050-15	0.050	0.085	0.010	0.06	0.02	0.09

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10050 Rev04  
Effective Date: 10/14/2021