



**HCV Seroconversion Panel Donor No.67787  
Part Number: HCV10051**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility. Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1b**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ALPHA HCV by PCR	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Architect ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
10051-01	05/29/98	6	NEGATIVE	0.125	0.296	0.070
10051-02	05/31/98	9	NEGATIVE	0.127	0.314	0.070
10051-03	06/05/98	11	NEGATIVE	0.110	0.279	0.060
10051-04	06/07/98	10	NEGATIVE	0.110	0.307	0.060
10051-05	06/12/98	11	NEGATIVE	0.112	0.328	0.080
10051-06	06/14/98	6	NEGATIVE	0.109	0.321	0.060
10051-07	06/21/98	10	NEGATIVE	0.109	0.307	0.050
10051-08	06/27/98	11	NEGATIVE	0.125	0.341	0.060
10051-09	06/29/98	15	POSITIVE	0.112	0.348	0.050
10051-10	07/04/98	31	POSITIVE	0.124	0.359	0.050
10051-11	07/06/98	24	POSITIVE	0.113	1.557	0.050
10051-12	07/11/98	20	POSITIVE	0.110	1.599	0.050
10051-13	07/14/98	13	POSITIVE	0.110	3.355	0.060
10051-14	07/18/98	13	POSITIVE	0.112	3.742	0.060

PANEL MEMBER	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve ANTI-HCV 3.0 Lab # 5	ORTHO Eci HBsAg	ORTHO Eci ANTI-HCV	ORTHO Eci ANTI-HIV ½
	TEST DATE: UNK	TEST DATE: 10/4/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
10051-01	0.197	0.010	0.06	0.02	0.10
10051-02	0.201	0.010	0.06	0.02	0.09
10051-03	0.204	0.020	0.06	0.01	0.10
10051-04	0.206	0.010	0.05	0.02	0.09
10051-05	0.230	0.010	0.06	0.02	0.09
10051-06	0.246	0.010	0.05	0.02	0.10
10051-07	0.197	0.010	0.05	0.01	0.11
10051-08	0.196	0.020	0.06	0.02	0.10
10051-09	0.172	0.010	0.06	0.02	0.10
10051-10	0.190	0.120	0.06	0.03	0.10
10051-11	0.256	0.010	0.05	0.02	0.10
10051-12	0.212	0.010	0.05	0.02	0.09
10051-13	0.204	0.010	0.05	0.02	0.10
10051-14	0.190	0.000	0.05	0.02	0.10

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV10051 Rev04  
Effective Date: 10/14/2021