



**HCV Seroconversion Panel Donor No. 61251  
Part Number: HCV6218**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4 0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT IMx ANTI-HCV TEST DATE: 9/23/1996	ABBOTT ANTI-HCV 2.0 TEST DATE: 9/6/1996	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK
6218-01	02/25/96	14	0.132	0.334	1.081	0.550	0.100	0.119
6218-02	03/05/96	18	0.136	0.355	1.165	0.553	0.100	0.129
6218-03	03/09/96	29	0.120	0.368	1.030	0.561	0.110	0.196
6218-04	03/12/96	19	0.115	0.361	1.073	0.553	0.160	0.201
6218-05	03/17/96	33	0.121	0.324	1.108	0.485	0.130	0.201
6218-06	03/21/96	16	0.106	0.324	1.108	0.576	0.100	0.133
6218-07	03/25/96	19	0.117	0.324	1.228	0.889	0.150	0.152
6218-08	03/31/96	13	0.127	0.324	1.144	4.257	0.840	0.362

PANEL MEMBER	ORTHO ANTI-HCV 3.0 (SAVe) TEST DATE: 8/15/1996	ORTHO ANTI-HCV 3.0 TEST DATE: UNK	SANOFI Monolisa ANTI-HCV New Antigens TEST DATE: 9/25/1996	ORTHO RIBA 3.0 C100 TEST DATE: UNK	ORTHO RIBA 3.0 C33 TEST DATE: UNK	ORTHO RIBA 3.0 C22 TEST DATE: UNK	ORTHO RIBA 3.0 NS5 TEST DATE: UNK	ORTHO RIBA 3.0 SOD TEST DATE: UNK
6218-01	0.042	0.023	0.035	-	-	-	-	-
6218-02	0.075	0.003	0.048	-	-	-	-	-
6218-03	0.060	0.010	0.044	-	-	-	-	-
6218-04	0.075	0.018	0.039	-	-	-	-	-
6218-05	0.051	0.005	0.024	-	-	-	-	-
6218-06	0.078	0.003	0.047	-	-	-	-	-
6218-07	0.164	0.033	0.045	-	-	-	-	-
6218-08	1.654	0.508	0.040	-	-	-	-	-

PANEL MEMBER	ORTHO RIBA 3.0 TEST DATE: UNK	CHIRON HCV RNA bDNA (copies/mL) TEST DATE: 12/17/1996	PCR RNA Qualitative Result Lab # 3 TEST DATE: UNK	PCR RNA Quantitative Result Lab # 3 TEST DATE: UNK	ORTHO Eci HBsAg TEST DATE: 11/16/2018	ORTHO Eci ANTI-HCV TEST DATE: 11/16/2018	ORTHO Eci ANTI-HIV 1/2 TEST DATE: 11/16/2018
6218-01	-	<200,000	NEGATIVE	<100	0.06	0.05	0.09
6218-02	-	<200,000	NEGATIVE	<100	0.06	0.05	0.09
6218-03	-	<200,000	NEGATIVE	<100	0.06	0.05	0.09
6218-04	-	<200,000	NEGATIVE	<100	0.05	0.04	0.09
6218-05	-	<200,000	NEGATIVE	<100	0.06	0.05	0.10
6218-06	-	<200,000	NEGATIVE	<100	0.06	0.05	0.10
6218-07	-	<200,000	NEGATIVE	<100	0.06	0.17	0.10
6218-08	-	<200,000	NEGATIVE	<100	0.07	2.12	0.09

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)