



**HCV Seroconversion Panel Donor No. 61404  
Part Number: HCV6220**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT IMx ANTI-HCV TEST DATE: 1/23/1997	ABBOTT PRISM ANTI-HCV TEST DATE: 1/30/1997	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT ANTI-HCV 2.0 Lab # 1 TEST DATE: 1/17/1997
6220-01	04/16/96	6	0.276	0.334	0.46	0.16	0.270	0.31
6220-02	04/19/96	12	0.288	0.365	0.46	0.13	0.260	0.27
6220-03	04/24/96	0	0.358	0.385	0.45	0.13	0.260	0.25
6220-04	04/26/96	4	0.341	0.398	0.45	0.15	0.240	0.29
6220-05	05/04/96	4	5.127	0.625	0.61	1.56	7.160	6.20
6220-06	05/06/96	9	3.614	0.468	0.52	1.13	5.690	4.53

PANEL MEMBER	ABBOTT ANTI-HCV 2.0 Lab # 2 TEST DATE: 2/26/1997	ABBOTT ANTI-HCV 2.0 Lab # 3 TEST DATE: 2/5/1997	ABBOTT AxSYM ANTI-HCV TEST DATE: 3/13/1997	ABBOTT ANTI-HBc EIA TEST DATE: 1/17/1997	MUREX ANTI-HCV Version III TEST DATE: 12/17/1996	ORTHO ANTI-HCV 3.0 Testsystem 3 TEST DATE: 2/5/1997	ORTHO ANTI-HCV 3.0 Lab # 1 TEST DATE: 11/15/1996	ORTHO RIBA 2.0 5-1-1 TEST DATE: UNK
6220-01	0.35	0.28	0.73	1.94	0.16	0.42	0.378	-
6220-02	0.29	0.29	0.57	1.94	0.16	0.03	0.303	-
6220-03	0.33	0.28	0.68	2.24	0.17	0.04	0.358	-
6220-04	0.25	0.25	0.58	1.87	0.17	0.05	0.293	-
6220-05	MAX. ABS.	4.22	0.75	2.02	0.19	7.66	>4.843	-
6220-06	4.04	3.19	0.60	2.03	0.17	4.82	4.726	-

PANEL MEMBER	ORTHO RIBA 2.0 C100-3 TEST DATE: UNK	ORTHO RIBA 2.0 C-33C TEST DATE: UNK	ORTHO RIBA 2.0 C22-3 TEST DATE: UNK	ORTHO RIBA 2.0 HSOD TEST DATE: UNK	SORIN ANTI-HCV EYI-AB-HCVK-3 TEST DATE: 1/13/1997	ORTHO RIBA 3.0 C100 TEST DATE: 4/22/1997	ORTHO RIBA 3.0 C33 TEST DATE: 4/22/1997	ORTHO RIBA 3.0 C22 TEST DATE: 4/22/1997
6220-01	-	+/-	-	-	0.35	NEGATIVE	+/-	NEGATIVE
6220-02	-	-	-	-	0.29	NEGATIVE	+/-	NEGATIVE
6220-03	-	+/-	-	-	0.29	NEGATIVE	+/-	NEGATIVE
6220-04	-	+/-	-	-	0.28	NEGATIVE	+/-	NEGATIVE
6220-05	-	+/-	-	-	9.20	NEGATIVE	4+	NEGATIVE
6220-06	-	+/-	-	-	8.56	NEGATIVE	4+	NEGATIVE

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)



**HCV Seroconversion Panel Donor No. 61404**  
**Part Number: HCV6220**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO RIBA 3.0 NS5 TEST DATE: 4/22/1997	ORTHO RIBA 3.0 SOD TEST DATE: 4/22/1997	PCR RNA Qualitative Results TEST DATE: UNK	PCR RNA Quantitative Results (Copies/mL) TEST DATE: UNK	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV 1/2 TEST DATE: 11/16/2018
6220-01	+/-	NEGATIVE	NEGATIVE	<400	0.12	0.17	0.10
6220-02	-	NEGATIVE	NEGATIVE	<400	0.11	0.14	0.10
6220-03	+/-	NEGATIVE	NEGATIVE	<400	0.10	0.21	0.10
6220-04	+/-	NEGATIVE	NEGATIVE	<400	0.11	0.18	0.10
6220-05	+/-	NEGATIVE	NEGATIVE	<400	0.15	8.85	0.10
6220-06	-	NEGATIVE	NEGATIVE	<400	0.12	6.53	0.09

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)