



**HCV Seroconversion Panel Donor No. 62258  
Part Number: HCV6221**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)  TEST DATE: 5/28/1997	ABBOTT MUREX ANTI-HCV 4.0  TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo  TEST DATE: UNK	ABBOTT IMx ANTI-HCV Lab # 1  TEST DATE: 6/18/1997	ABBOTT IMx ANTI-HCV Lab # 2  TEST DATE: 6/30/1997	ABBOTT ANTI-HCV 2.0 Lab # 1  TEST DATE: 8/14/1997
6221-01	08/20/96	2	0.773	1.331	12.21	12.67	0.38
6221-02	08/22/96	7	0.706	1.157	12.20	12.86	0.40
6221-03	08/29/96	8	0.782	1.298	13.54	15.41	0.52
6221-04	08/31/96	0	0.685	1.254	8.98	12.09	0.32
6221-05	09/05/96	3	0.800	1.311	10.02	15.42	0.42
6221-06	09/17/96	7	0.915	1.548	19.89	20.12	0.78
6221-07	09/20/96	4	0.814	1.251	17.91	23.36	1.12
6221-08	10/14/96	11	0.880	1.107	21.08	25.01	1.53

PANEL MEMBER	ABBOTT ANTI-HCV 2.0 Lab # 2  TEST DATE: 6/25/1997	ABBOTT PRISM ANTI-HCV  TEST DATE: 6/18/1997	ABBOTT Architect ANTI-HCV  TEST DATE: UNK	ABBOTT AxSYM ANTI-HCV  TEST DATE: 6/18/1997	ORTHO VITROS ECi ANTI-HCV  TEST DATE: UNK	ORTHO ANTI-HCV 3.0 (SAVe)  TEST DATE: 6/17/1997	BOEHRINGER MANNHEIM Enzymuntest ANTI-HCV  TEST DATE: 6/5/1997
6221-01	0.24	0.79	0.900	9.57	1.28	0.29	0.24
6221-02	0.37	0.68	0.830	12.19	1.12	0.49	0.30
6221-03	0.33	0.87	0.980	16.35	1.27	0.46	0.32
6221-04	0.29	0.76	0.810	8.69	1.00	0.27	0.23
6221-05	0.34	0.83	0.780	11.93	1.08	0.35	0.31
6221-06	0.59	0.92	1.180	16.39	2.07	0.71	0.47
6221-07	0.77	1.46	1.240	19.24	2.52	0.78	0.64
6221-08	1.12	1.44	1.950	26.04	4.38	1.44	0.91

PANEL MEMBER	ORTHO RIBA 3.0 C100  TEST DATE: 6/20/1997	ORTHO RIBA 3.0 C33  TEST DATE: 6/20/1997	ORTHO RIBA 3.0 C22  TEST DATE: 6/20/1997	ORTHO RIBA 3.0 NS5  TEST DATE: 6/20/1997	ORTHO RIBA 3.0 SOD  TEST DATE: 6/20/1997	PCR RNA Qualitative Results  TEST DATE: UNK	PCR RNA Quantitative Results (Copies/mL)  TEST DATE: UNK
6221-01	-	+/-	+/-	+/-	-	NOT DETECTED	<400
6221-02	-	+/-	+/-	+/-	-	NOT DETECTED	<400
6221-03	-	+/-	+/-	+/-	-	NOT DETECTED	<400
6221-04	-	+/-	+/-	+/-	-	NOT DETECTED	<400
6221-05	-	+/-	+/-	+/-	-	NOT DETECTED	<400
6221-06	-	+/-	+/-	+/-	-	NOT DETECTED	<400
6221-07	-	+/-	+/-	+/-	-	NOT DETECTED	<400
6221-08	-	1	+/-	+/-	-	NOT DETECTED	<400

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV6221 Rev03  
Effective Date: 10/07/2021