



**HCV Seroconversion Panel Donor No. 62286**  
**Part Number: HCV6222**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT IMx ANTI-HCV	ABBOTT ANTI-HCV 2.0	ABBOTT PRISM ANTI-HCV	ABBOTT AxSYM ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/18/1997	TEST DATE: 6/25/1997	TEST DATE: 6/18/1997	TEST DATE: 6/18/1997
6222-01	08/18/96	6	0.12	0.39	0.72	0.21	0.07	0.59
6222-02	08/20/96	1	0.13	0.37	0.53	0.17	0.07	0.59
6222-03	09/04/96	11	0.16	3.35	0.60	0.16	0.07	0.60
6222-04	09/06/96	12	0.20	5.59	0.49	0.15	0.06	0.53
6222-05	09/11/96	9	0.22	7.99	0.57	0.16	0.06	0.57
6222-06	09/13/96	7	0.21	6.97	0.56	0.11	0.10	0.57
6222-07	09/23/96	25	0.87	9.79	0.59	0.15	0.34	0.66
6222-08	09/27/96	31	5.23	11.04	1.09	0.20	2.89	1.08

PANEL MEMBER	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO ANTI-HCV 3.0 (SAVe)	ORTHO ELISA ANTI-HCV 3.0	BOEHRINGER MANNHEIM Ezymuntest ANTI-HCV	ORTHO RIBA 3.0 C100	ORTHO RIBA 3.0 C33	ORTHO RIBA 3.0 C22
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/17/1997	TEST DATE: 6/5/1997	TEST DATE: 9/25/1997	TEST DATE: 9/25/1997	TEST DATE: 9/25/1997
6222-01	0.09	0.14	0.007	0.13	0.03	-	-	-
6222-02	0.08	0.17	0.005	0.06	0.01	-	-	-
6222-03	0.08	0.10	0.010	0.05	0.04	-	-	-
6222-04	0.07	0.07	0.002	0.04	0.03	-	-	-
6222-05	0.07	0.09	0.026	0.04	0.03	-	-	-
6222-06	0.24	0.08	0.003	0.04	0.02	-	-	-
6222-07	0.49	0.03	0.043	0.07	0.24	-	+/-	+/-
6222-08	6.29	4.06	1.597	1.55	2.40	-	2	+/-

PANEL MEMBER	ORTHO RIBA 3.0 NS5	ORTHO RIBA 3.0 SOD	PCR RNA Qualitative Result Lab # 3	PCR RNA Quantitative Result Lab # 4	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: 9/25/1997	TEST DATE: 9/25/1997	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
6222-01	-	-	NEGATIVE	<100	0.06	0.02	0.12
6222-02	-	-	NEGATIVE	<100	0.05	0.02	0.11
6222-03	-	-	NEGATIVE	290000	0.05	0.01	0.11
6222-04	-	-	NEGATIVE	560000	0.04	0.02	0.11
6222-05	-	-	NEGATIVE	710000	0.05	0.02	0.11
6222-06	-	-	NEGATIVE	830000	0.05	0.01	0.12
6222-07	-	-	NEGATIVE	1600000	0.06	0.27	0.12
6222-08	-	-	NEGATIVE	450000	0.06	5.48	0.12

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV6222 Rev03  
 Effective Date: 06/15/2021