



**HCV Seroconversion Panel Donor No. 63135**  
**Part Number: HCV6226**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Corzyme ANTI-HBc	ABBOTT ANTI-HCV 2.0 Lab # 1	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV
		Normal Range (10-45 u/L)	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6226-01	12/17/96	16	0.14	0.473	0.11	0.336	0.07	0.102
6226-02	12/20/96	19	0.15	0.375	0.11	0.179	0.06	0.112
6226-03	12/26/96	9	0.16	0.527	0.10	0.164	0.07	0.102
6226-04	12/28/96	11	0.16	0.429	0.12	0.145	0.06	0.089
6226-05	01/09/97	37	0.16	0.314	0.10	0.189	0.07	0.097
6226-06	01/11/97	19	0.15	0.399	0.12	0.038	0.07	0.101
6226-07	01/16/97	19	0.15	0.456	0.10	0.162	0.15	0.188
6226-08	01/18/97	25	0.16	0.402	0.09	0.202	0.23	0.342
6226-09	01/23/97	37	0.76	0.412	0.12	0.126	2.27	2.633
6226-10	01/25/97	40	2.32	0.456	0.11	0.155	4.52	4.867
6226-11	01/30/97	60	5.13	0.622	0.11	0.176	9.96	7.948
6226-12	02/02/97	52	5.29	0.939	0.11	0.195	10.82	8.769

PANEL MEMBER	ROCHE DIAGNOSTICS Cobas ANTI-HCV	ORTHO ELISA ANTI-HCV 3.0	ORTHO RIBA 2.0	ORTHO RIBA 3.0 C100	ORTHO RIBA 3.0 C33	ORTHO RIBA 3.0 C22	ORTHO RIBA 3.0 NS5	ORTHO RIBA 3.0 SOD
	TEST DATE: 11/7/1997	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997	TEST DATE: 10/29/1997
6226-01	0.16	0.005	-	-	-	-	-	-
6226-02	0.12	0.002	-	-	-	-	-	-
6226-03	0.11	0.002	-	-	-	-	-	-
6226-04	0.12	0.003	-	-	-	-	-	-
6226-05	0.10	0.002	-	-	-	-	-	-
6226-06	0.16	0.003	-	-	-	-	-	-
6226-07	0.11	0.003	-	-	-	-	-	-
6226-08	0.12	0.007	-	-	-	-	-	-
6226-09	0.14	0.375	-	-	-	-	-	-
6226-10	0.17	1.218	-	-	+/-	-	-	-
6226-11	0.24	4.016	-	+/-	2+	-	-	-
6226-12	0.60	4.072	-	+/-	2+	-	-	-

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV6226 Rev03  
 Effective Date: 10/08/2021



**HCV Seroconversion Panel Donor No. 63135  
Part Number: HCV6226**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	CHIRON bDNA (MEq/mL)  TEST DATE: 9/23/1997	ROCHE HCV MONITOR (copies/mL)  TEST DATE: UNK	PCR RNA Quantitative Result Lab # 3  TEST DATE: UNK	ZEPTOMETRIX HCV RT PCR  TEST DATE: UNK	ORTHO ECi HBsAg  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½  TEST DATE: 11/16/2018
6226-01	2.379	230000	1900000	254000	0.05	0.00	0.13
6226-02	0.852	65000	670000	58300	0.05	0.00	0.13
6226-03	3.788	160000	710000	196000	0.05	0.01	0.12
6226-04	1.542	140000	730000	362000	0.06	0.01	0.14
6226-05	0.232	2500	50000	27300	0.07	0.01	0.14
6226-06	2.382	170000	740000	158000	0.05	0.01	0.13
6226-07	2.367	250000	1100000	334000	0.05	0.01	0.12
6226-08	1.915	140000	1300000	174000	0.05	0.02	0.12
6226-09	1.184	74000	580000	107000	0.06	1.01	0.13
6226-10	0.955	54000	250000	169000	0.05	2.47	0.11
6226-11	0.402	41000	320000	113000	0.05	6.37	0.12
6226-12	0.283	31000	310000	53900	0.05	7.22	0.13

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)