



HCV Seroconversion Panel Donor No. 63318
Part Number: HCV6227

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT Normal Range (10-45 u/L) | ABBOTT MUREX ANTI-HCV 4.0 | ABBOTT MUREX HCV Ag/Ab Combo | ABBOTT Corzyme ANTI-HBc | ABBOTT ANTI-HCV 2.0 Lab # 1 | ABBOTT ANTI-HCV 2.0 Lab # 2 | ABBOTT Architect ANTI-HCV | ABBOTT PRISM ANTI-HCV |
|--------------|--------------|------------------------------------|------------------------------------|---------------------------------------|-------------------------------|--------------------------------------|--------------------------------------|---------------------------------|-----------------------------|
| | | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 11/6/1997 | TEST DATE: UNK |
| 6227-01 | 12/17/96 | 6 | 0.14 | 0.419 | 2.23 | 0.204 | 0.17 | 0.07 | 0.300 |
| 6227-02 | 01/08/97 | 14 | 0.15 | 0.405 | 2.22 | 0.231 | 0.19 | 0.06 | 0.310 |
| 6227-03 | 01/10/97 | 3 | 0.14 | 0.368 | 2.27 | 0.183 | 0.18 | 0.08 | 0.300 |
| 6227-04 | 01/28/97 | 34 | 0.15 | 1.784 | 2.00 | 0.235 | 0.20 | 0.09 | 0.330 |
| 6227-05 | 02/01/97 | 23 | 0.14 | 4.534 | 1.94 | 0.218 | 0.22 | 0.07 | 0.340 |
| 6227-06 | 03/01/97 | 59 | 4.40 | 11.696 | 1.99 | 0.855 | 0.86 | 6.55 | 3.690 |
| 6227-07 | 03/03/97 | 212 | 5.23 | 11.892 | 2.22 | 1.376 | 1.45 | 7.41 | 3.990 |

| PANEL MEMBER | ROCHE DIAGNOSTICS Cobas ANTI- HCV | ABBOTT AxSYM ANTI-HCV | ORTHO ANTI-HCV 3.0 (SAVe) | ORTHO ELISA ANTI-HCV 3.0 | ORTHO RIBA 2.0 | ORTHO RIBA 3.0 C100 | ORTHO RIBA 3.0 C33 | ORTHO RIBA 3.0 C22 |
|--------------|--|-----------------------------|------------------------------------|-----------------------------------|----------------------|------------------------------|-----------------------------|-----------------------------|
| | TEST DATE: 11/7/1997 | TEST DATE: 10/21/1997 | TEST DATE: 10/14/1997 | TEST DATE: UNK | TEST DATE: UNK | TEST DATE: 10/29/1997 | TEST DATE: 10/29/1997 | TEST DATE: 10/29/1997 |
| 6227-01 | 0.15 | 0.63 | 0.00 | 0.029 | - | - | - | - |
| 6227-02 | 0.10 | 0.63 | 0.00 | 0.067 | - | - | - | - |
| 6227-03 | 0.12 | 0.52 | 0.00 | 0.028 | - | - | - | - |
| 6227-04 | 0.16 | 0.56 | 0.01 | 0.042 | - | - | - | - |
| 6227-05 | 0.14 | 0.53 | 0.00 | 0.028 | - | - | - | - |
| 6227-06 | 0.99 | 1.75 | 2.10 | 3.686 | - | +/- | 3+ | 2+ |
| 6227-07 | 5.56 | 1.64 | 2.52 | 3.730 | INDETERMINATE | +/- | 3+ | 4+ |

| PANEL MEMBER | ORTHO RIBA 3.0 NS5 | ORTHO RIBA 3.0 SOD | CHIRON bDNA (MEq/mL) | ROCHE HCV MONITOR (copies/mL) | PCR RNA Quantitative Result Lab # 3 | ORTHO ECi HBsAg | ORTHO ECi ANTI-HCV | ORTHO ECi ANTI-HIV 1/2 |
|--------------|-----------------------------|-----------------------------|----------------------------|--|--|--------------------------|--------------------------|---------------------------------|
| | TEST DATE: 10/29/1997 | TEST DATE: 10/29/1997 | TEST DATE: 9/23/1997 | TEST DATE: UNK | TEST DATE: 10/25/1997 | TEST DATE: 11/16/2018 | TEST DATE: 11/16/2018 | TEST DATE: 11/16/2018 |
| 6227-01 | - | NEGATIVE | 0.226 | <166 | <100 | 0.06 | 0.03 | 0.11 |
| 6227-02 | - | NEGATIVE | <0.2 | <166 | <100 | 0.05 | 0.03 | 0.11 |
| 6227-03 | - | NEGATIVE | <0.2 | <166 | <100 | 0.05 | 0.05 | 0.12 |
| 6227-04 | - | NEGATIVE | 11.18 | 390000 | 2100000 | 0.06 | 0.05 | 0.11 |
| 6227-05 | - | NEGATIVE | 31.05 | 770000 | >5000000 | 0.06 | 0.04 | 0.12 |
| 6227-06 | 1+ | NEGATIVE | 55.14 | 820,000 | >5000000 | 0.07 | 6.17 | 0.12 |
| 6227-07 | +/- | NEGATIVE | 46.75 | 650,000 | 1300000 | 0.06 | 7.82 | 0.13 |

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com