



HCV Seroconversion Panel Donor No. 62886
Part Number: HCV6228

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

HCV Genotype: 1a

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT Corzyme ANTI-HBc	ABBOTT ANTI-HCV 2.0 Lab # 1	ABBOTT ANTI-HCV 2.0 Lab # 2	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 11/6/1997	TEST DATE: UNK	TEST DATE: UNK
6228-01	11/20/96	41	0.142	0.584	1.60	0.132	0.08	0.030	0.062
6228-02	11/22/96	26	0.127	0.416	1.42	0.134	0.11	0.040	0.081
6228-03	11/27/96	32	0.134	1.027/0.936	1.31	0.145	0.13	0.040	0.078
6228-04	11/29/96	36	0.128	0.580	1.46	0.120	0.12	0.030	0.066
6228-05	12/04/96	100	0.130	0.352	1.24	0.155	0.13	0.040	0.076
6228-06	12/06/96	248	0.110	0.297	1.50	0.128	0.11	0.030	0.080
6228-07	12/11/96	47	0.131	0.485	1.80	0.143	0.12	0.090	0.163
6228-08	12/14/96	196	0.124	0.608	1.72	0.128	0.14	0.100	0.137
6228-09	12/18/96	156	0.422	0.556	1.72	0.181	0.19	1.370	1.392
6228-10	12/21/96	53	0.985	1.092	1.59	0.252	0.21	4.520	4.582
6228-11	12/26/96	100	1.673	1.666	1.35	0.290	0.32	6.620	6.224
6228-12	12/28/96	96	1.679	1.529	1.44	0.370	0.31	7.120	6.665

PANEL MEMBER	ROCHE DIAGNOSTICS Cobas ANTI-HCV	ABBOTT AxSYM ANTI-HCV	ZEPTOMETRIX HCVRT PCR	ORTHO ANTI-HCV 3.0 (SAVe)	ORTHO ELISA ANTI-HCV 3.0	ORTHO ECi ANTI-HIV	ORTHO RIBA 2.0	ORTHO RIBA 3.0 C100
	TEST DATE: 11/7/1997	TEST DATE: 10/21/1997	TEST DATE: UNK	TEST DATE: 10/14/1997	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK
6228-01	0.09	0.52	150000	0.01	0.000	0.03	-	-
6228-02	0.09	0.47	46300	0.01	0.000	0.03	-	-
6228-03	0.1	0.46	138000	0.01	0.002	0.03	-	-
6228-04	0.1	0.50	49600	0.01	0.000	0.03	-	-
6228-05	0.09	0.47	35400	0.01	0.000	0.03	-	-
6228-06	0.1	0.47	1840	0.01	0.010	0.03	-	-
6228-07	0.1	0.54	61000	0.01	0.007	0.05	-	-
6228-08	0.09	0.51	169000	0.01	0.007	0.08	-	-
6228-09	0.06	0.52	407	0.28	0.642	1.33	-	-
6228-10	0.1	0.63	816	1.10	2.166	4.03	-	-
6228-11	0.15	0.62	38800	1.31	2.842	4.50	-	-
6228-12	0.17	0.62	3160	1.48	3.192	4.84	-	-

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV6228 Rev03
 Effective Date: 10/08/2021



**HCV Seroconversion Panel Donor No. 62886
Part Number: HCV6228**

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO RIBA 3.0 C33	ORTHO RIBA 3.0 C22	ORTHO RIBA 3.0 NS5	ORTHO RIBA 3.0 SOD	ORTHO RIBA 3.0	CHIRON bDNA (MEq/mL)	ROCHE HCV MONITOR (copies/mL)	PCR RNA Quantitative Result Lab # 3
	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/29/1997	TEST DATE: 9/23/1997	TEST DATE: UNK	TEST DATE: 10/25/1997
6228-01	-	-	-	-	-	1.486	270000	670000
6228-02	-	-	-	-	-	0.986	180000	260000
6228-03	-	-	-	-	-	2.480	320000	1000000
6228-04	-	-	-	-	-	0.916	400000	445000
6228-05	-	-	-	-	-	<0.2	99000	240000
6228-06	-	-	-	-	-	<0.2	5300	15500
6228-07	-	-	-	-	-	0.898	160000	1000000
6228-08	-	-	-	-	-	1.625	140000	1300000
6228-09	-	-	-	-	-	<0.2	340	500
6228-10	1+	-	-	-	INDETERMINATE	<0.2	1600	1000
6228-11	2+	-	-	-	INDETERMINATE	0.396	78000	440000
6228-12	2+	-	-	-	INDETERMINATE	<0.2	3000	7500

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com