

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 63911 Part Number: HCV9044

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV Lab # 2	ABBOTT ANTI-HCV 3.0	ORTHO ANTI-HCV 3.0
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/20/1998	TEST DATE: 6/15/1998	TEST DATE: 7/8/1998	TEST DATE: UNK
9044-01	04/14/97	14	0.106	8.846	0.045	0.219	0.08	0.005
9044-02	04/18/97	13	0.106	6.184	0.048	0.226	0.09	0.003
9044-03	05/01/97	28	0.166	7.693	0.039	0.180	0.10	0.003
9044-04	05/05/97	108	0.611	6.655	0.117	0.319	0.23	0.124
9044-05	05/09/97	124	2.548	7.133	0.693	1.586	1.15	1.364
9044-06	05/13/97	172	3.965	7.676	1.169	2.100	1.72	1.864

PANEL MEMBER	ABBOTT PRISM HCV	ABBOTT IMx HCVHCV 3.0 Reagent Pack	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE/BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA	ORTHO VITROS EC1 ANTI-HCV
	TEST DATE: 8/5/1998	TEST DATE: 8/4/1998	TEST DATE: 7/6/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/7/1998	TEST DATE: UNK
9044-01	0.06	0.17	0.13	0.050	0.090	0.1	0.005
9044-02	0.08	0.17	0.10	0.080	0.090	0.1	0.003
9044-03	0.07	0.19	0.16	0.070	0.120	0.1	0.003
9044-04	0.93	0.46	0.71	1.250	0.830	0.6	0.124
9044-05	3.42	1.73	4.40	5.430	3.900	3.3	1.364
9044-06	4.07	3.66	11.95	7.230	4.780	5.8	1.864

PANEL MEMBER	ORTHO Enhanced SAvE HCV 3.0 Lab # 4	ORTHO Enhanced SAvE HCV 3.0 Lab # 7	SANOFI Monolisa ANTI- HCV PLUS	SANOFI Monolisa ANTI- HCV PLUS Lab # 7	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4)	ORTHO Chiron Riba HCV 2.0 C100(NS4)	ORTHO Chiron Riba HCV 2.0 C33(NS3)
	TEST DATE: 6/29/1998	TEST DATE: 6/23/1998	TEST DATE: 6/30/1998	TEST DATE: 6/23/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998	TEST DATE: 7/7/1998
9044-01	0.00	0.009	0.38	0.097	-	-	-
9044-02	0.01	0.024	0.42	0.059	-	-	-
9044-03	0.00	0.021	0.43	0.073	-	-	-
9044-04	0.69	0.320	1.14	0.260	-	-	+/-
9044-05	3.70	1.860	3.05	1.358	-	-	1+
9044-06	4.51	3.160	3.60	2.258	-	-	2+

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9044 Rev03
Effective Date: 06/15/2021

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 63911 Part Number: HCV9044

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 2.0 C22(Core) TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 2.0 (SOD) TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 c100(p) TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: 7/7/1998	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: 7/7/1998/	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: 7/7/1998	CHIRON HCV RNA bDNA 2.0 (MEq/mL*) TEST DATE: 7/16/1998
9044-01	-	-	-	-	-	-	-	105.9
9044-02	-	-	-	-	-	-	-	52.71
9044-03	-	-	-	-	-	-	-	77.18
9044-04	-	-	-	+/-	-	-	-	71.52
9044-05	-	-	-	2+	-	+/-	INDETERMINATE	72.82
9044-06	-	-	+/-	3+	-	+/-	INDETERMINATE	48.83

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9044 Rev03
Effective Date: 06/15/2021