

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 64150 Part Number: HCV9045

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L) TEST DATE: UNK	ABBOTT MUREX ANTI-HCV 4.0 TEST DATE: UNK	ABBOTT MUREX HCV Ag/Ab Combo TEST DATE: UNK	ABBOTT ANTI-HCV Lab # 1 TEST DATE: 5/20/1998	ABBOTT ANTI-HCV Lab # 2 TEST DATE: 6/15/1998	ABBOTT ANTI-HCV 3.0 TEST DATE: 7/8/1998	ORTHO ANTI-HCV 3.0 TEST DATE: UNK	ABBOTT PRISM HCV TEST DATE: 8/5/1998
9045-01	05/09/97	27	0.084	4.833	0.043	0.112	0.07	0.008	0.06
9045-02	05/11/97	22	0.087	5.038	0.043	0.109	0.09	0.005	0.06
9045-03	05/16/97	30	0.095	5.587	0.043	0.128	0.11	0.007	0.07
9045-04	05/18/97	14	0.090	5.952	0.043	0.132	0.07	0.005	0.07
9045-05	06/04/97	27	0.107	6.119	0.065	0.155	0.11	0.007	0.06
9045-06	06/10/97	216	0.152	3.130	0.108	0.144	0.22	0.005	0.12
9045-07	06/15/97	304	1.213	5.205	0.240	0.399	0.34	0.654	2.03
9045-08	06/19/97	308	1.828	3.802	0.455	0.838	0.92	1.391	3.24

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack TEST DATE: 8/4/1998	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack TEST DATE: 7/6/1998	ABBOTT Architect ANTI-HCV TEST DATE: UNK	ABBOTT PRISM ANTI-HCV TEST DATE: UNK	ROCHE/BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA TEST DATE: 7/7/1998	ORTHO ECI ANTI-HCV TEST DATE: UNK	ORTHO Enhanced SAVE HCV 3.0 Lab # 4 TEST DATE: 6/29/1998	ORTHO Enhanced SAVE HCV 3.0 Lab # 7 TEST DATE: 6/23/1998
9045-01	0.12	0.14	0.040	0.060	0.1	0.03	0.00	0.016
9045-02	0.12	0.13	0.030	0.060	0.1	0.03	0.00	0.022
9045-03	0.14	0.11	0.040	0.060	0.1	0.03	0.00	0.006
9045-04	0.15	NOT TESTED	0.030	0.060	0.1	0.03	0.01	0.003
9045-05	0.13	0.17	0.040	0.070	0.1	0.04	0.00	0.000
9045-06	0.23	0.23	0.120	0.120	0.1	0.06	0.04	0.025
9045-07	1.28	2.39	3.410	2.240	0.5	2.68	2.61	1.599
9045-08	2.36	4.11	5.300	3.680	1.2	3.84	3.51	2.430

PANEL MEMBER	SANOFI Monolisa ANTI-HCV PLUS TEST DATE: 6/30/1998	SANOFI Monolisa ANTI-HCV PLUS Lab # 7 TEST DATE: 6/23/1998	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C100(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C33(NS3) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C22(Core) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 SOD TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c100(p) TEST DATE: UNK
9045-01	0.07	0.028	-	-	-	-	-	-
9045-02	0.06	0.037	-	-	-	-	-	-
9045-03	0.05	0.049	-	-	-	-	-	-
9045-04	0.05	0.042	-	-	-	-	-	-
9045-05	0.08	0.034	-	-	-	-	-	-
9045-06	0.25	0.069	-	-	-	-	-	-
9045-07	2.12	0.729	-	-	+/-	+/-	-	-
9045-08	2.89	2.399	+/-	+/-	1+	+/-	-	+/-

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9045 Rev03
Effective Date: 10/08/2021

ZeptoMetrix®

HCV Seroconversion Panel Donor No. 64150 Part Number: HCV9045

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: UNK	CHIRON HCV RNA bDNA 2.0 (MEq/mL*) TEST DATE: 7/16/1998	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
9045-01	-	-	-	-	17.73	0.05	0.01	0.13
9045-02	-	-	-	-	16.42	0.05	0.01	0.13
9045-03	-	-	-	-	36.19	0.05	0.01	0.12
9045-04	-	-	-	-	38.07	0.05	0.01	0.13
9045-05	-	-	-	-	44.86	0.06	0.01	0.14
9045-06	-	-	-	-	17.78	0.06	0.02	0.13
9045-07	1+	-	-	-	18.36	0.06	1.41	0.13
9045-08	2+	-	-	INDETERMINATE	2.322	0.05	1.82	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9045 Rev03
Effective Date: 10/08/2021