



HCV Seroconversion Panel Donor No. 64273
Part Number: HCV9047

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4 0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV Lab # 2	ABBOTT ANTI-HCV 3.0	ORTHO ANTI-HCV 3.0
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/20/1998	TEST DATE: 6/15/1998	TEST DATE: 7/8/1998	TEST DATE: UNK
9047-01	06/04/97	11	0.142	8.840	0.108	0.189	0.14	0.008
9047-02	06/06/97	5	0.148	10.014	0.108	0.235	0.16	0.012
9047-03	06/14/97	7	0.136	7.945	0.108	0.200	0.21	0.008
9047-04	06/16/97	3	0.124	8.655	0.152	0.182	0.16	0.010
9047-05	06/23/97	26	0.103	6.478	0.108	0.194	0.18	0.008
9047-06	06/25/97	26	0.136	5.102	0.108	0.187	0.18	0.012
9047-07	07/02/97	324	0.944	6.038	0.195	0.305	0.36	1.286
9047-08	07/04/97	272	2.849	12.167	0.998	0.977	1.91	4.139
9047-09	07/09/97	252	5.472	6.007	2.230	2.772	4.25	4.139
9047-10	07/11/97	276	5.421	6.389	2.857	3.016	1.00	4.139

PANEL MEMBER	ABBOTT PRISM HCV	ABBOTT IMx HCV 3.0 Reagent Pack	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE/ BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA	ORTHO Enhanced SAVE HCV 3.0 Lab # 4	ORTHO Enhanced SAVE HCV 3.0 Lab # 7
	TEST DATE: 8/5/1998	TEST DATE: 8/4/1998	TEST DATE: 7/6/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/7/1998	TEST DATE: 7/15/1998	TEST DATE: 6/25/1998
9047-01	0.09	0.13	0.16	0.060	0.080	0.4	0.01	0.102
9047-02	0.10	0.20	0.13	0.070	0.070	0.4	0.01	0.122
9047-03	0.09	0.15	0.17	0.060	0.070	0.4	0.01	0.086
9047-04	0.09	0.20	0.16	0.060	0.070	0.3	0.01	0.052
9047-05	0.11	0.20	0.18	0.060	0.080	0.4	0.01	0.102
9047-06	0.13	0.24	0.19	0.070	0.080	0.4	0.01	0.090
9047-07	2.47	3.29	6.40	4.840	2.850	0.6	1.78	1.513
9047-08	3.88	16.25	26.79	9.600	4.960	2.7	3.70	3.924
9047-09	4.79	26.75	43.67	10.170	5.840	9.8	MAX. ABS.	7.355
9047-10	5.01	28.21	48.08	9.940	6.640	9.2	MAX. ABS.	6.770

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com



HCV Seroconversion Panel Donor No. 64273
Part Number: HCV9047

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS TEST DATE: 6/30/1998	SANOFI Monolisa ANTI- HCV PLUS Lab # 7 TEST DATE: 6/25/1998	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C100(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C33(NS3) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C22(Core) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 SOD TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c100(p) TEST DATE: UNK
9047-01	0.07	0.091	-	+/-	+/-	-	-	-
9047-02	0.07	0.123	-	+/-	+/-	-	-	-
9047-03	0.07	0.103	-	+/-	+/-	-	-	-
9047-04	0.11	0.067	-	+/-	+/-	-	-	-
9047-05	0.07	0.119	-	+/-	+/-	-	-	-
9047-06	0.07	0.155	-	+/-	+/-	-	-	-
9047-07	2.50	0.725	-	+/-	+/-	-	-	-
9047-08	5.89	4.119	-	+/-	2+	-	-	-
9047-09	6.51	5.888	+/-	+/-	4+	-	-	+/-
9047-10	7.10	6.448	+/-	+/-	3+	-	-	2+

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: UNK	CHIRON HCV RNA bDNA 2.0 (MEq/mL) TEST DATE: 7/16/1998	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
9047-01	-	-	-	NEGATIVE	65.95	0.05	0.02	0.13
9047-02	-	-	-	NEGATIVE	81.76	0.05	0.01	0.12
9047-03	-	-	-	NEGATIVE	54.85	0.06	0.02	0.12
9047-04	-	-	-	NEGATIVE	64.03	0.05	0.02	0.11
9047-05	-	-	-	NEGATIVE	55.45	0.06	0.03	0.11
9047-06	-	-	-	NEGATIVE	42.97	0.06	0.02	0.11
9047-07	2+	-	-	INDETERMINATE	9.132	0.05	2.27	0.13
9047-08	4+	-	-	INDETERMINATE	26.58	0.06	6.32	0.12
9047-09	4+	-	-	INDETERMINATE	22.77	0.06	11.0	0.12
9047-10	4+	+/-	+/-	POSITIVE	NOT DETECTED	0.05	12.0	0.11

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9047 Rev03
 Effective Date: 10/11/2021