



HCV Seroconversion Panel Donor No. 64437
Part Number: HCV9048

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV Lab # 1	ABBOTT ANTI-HCV Lab # 2	ABBOTT ANTI-HCV 3.0	ORTHO ANTI-HCV 3.0	ABBOTT PRISM HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 5/20/1998	TEST DATE: 6/15/1998	TEST DATE: 7/8/1998	TEST DATE: UNK	TEST DATE: 8/5/1998
9048-01	07/14/97	15	0.716	0.614	0.866	0.811	0.96	0.061	0.79
9048-02	07/16/97	11	0.582	0.529	0.910	0.847	0.91	0.052	0.78
9048-03	07/21/97	16	0.504	0.567	0.952	0.777	0.91	0.123	0.95
9048-04	07/24/97	11	0.605	0.567	0.910	0.830	1.11	0.051	0.66
9048-05	07/29/97	21	0.724	0.635	1.125	0.920	1.47	0.080	0.87
9048-06	07/31/97	14	0.594	0.679	1.364	1.055	1.04	0.154	1.07
9048-07	08/05/97	5	0.685	0.635	1.125	1.011	0.93	0.229	0.91
9048-08	08/07/97	11	0.649	0.631	1.255	0.880	0.56	0.137	0.85

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack	ABBOTT AxSYM ANTI-HCV 3.0 Reagent Pack	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ROCHE/BOEHRINGER MANNHEIM Cobas Core ANTI-HCV EIA	ORTHO ECi ANTI-HCV	ORTHO Enhanced SAVE HCV 3.0 Lab # 4	ORTHO Enhanced SAVE HCV 3.0 Lab # 7	SANOFI Monolisa ANTI-HCV PLUS
	TEST DATE: 8/4/1998	TEST DATE: 7/6/1998	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 7/7/1998	TEST DATE: UNK	TEST DATE: 7/15/1998	TEST DATE: 6/25/1998	TEST DATE: 6/30/1998
9048-01	0.85	0.82	1.110	0.819	3.5	1.16	1.22	0.490	1.42
9048-02	0.69	0.71	0.930	0.883	2.9	1.14	1.29	0.560	1.68
9048-03	0.63	0.77	0.860	0.809	2.9	1.03	1.11	0.334	1.49
9048-04	0.77	0.68	1.000	0.931	2.8	1.05	1.36	0.451	1.61
9048-05	0.72	0.78	1.660	1.192	3.9	2.05	1.84	0.757	2.04
9048-06	0.85	0.89	1.350	1.294	4.0	1.49	2.29	0.381	2.62
9048-07	0.76	0.82	1.340	1.231	3.8	1.39	1.49	0.674	1.99
9048-08	0.79	0.71	1.200	1.204	3.7	1.07	1.18	0.290	1.59

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9048 Rev03
 Effective Date: 10/11/2021



HCV Seroconversion Panel Donor No. 64437
Part Number: HCV9048

Donor Demographics: Donor source plasma was collected from a U.S. licensed blood collection facility.
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	SANOFI Monolisa ANTI- HCV PLUS Lab # 7 TEST DATE: 6/25/1998	ORTHO Chiron Riba HCV 2.0 5-1-1(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C100(NS4) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C33(NS3) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 C22(Core) TEST DATE: UNK	ORTHO Chiron Riba HCV 2.0 SOD TEST DATE: UNK	ORTHO Chiron RIBA HCV 2.0 Interpretation TEST DATE: UNK	ORTHO Chiron Riba HCV 3.0 c100(p) TEST DATE: 7/22/1998
9048-01	0.725	-	+/-	+/-	-	-	-	+/-
9048-02	0.824	-	+/-	+/-	-	-	-	+/-
9048-03	0.574	-	+/-	+/-	-	-	-	+/-
9048-04	0.717	-	+/-	+/-	-	-	-	+/-
9048-05	1.132	-	+/-	1+	-	+/-	INDETERMINATE	+/-
9048-06	1.255	-	+/-	1+	-	+/-	INDETERMINATE	+/-
9048-07	1.006	-	+/-	1+	-	+/-	INDETERMINATE	+/-
9048-08	1.009	-	+/-	1+	-	+/-	INDETERMINATE	+/-

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c TEST DATE: 7/22/1998	ORTHO Chiron Riba HCV 3.0 c22(p) TEST DATE: 7/22/1998	ORTHO Chiron Riba HCV 3.0 NS5 TEST DATE: 7/22/1998	ORTHO Chiron Riba HCV 3.0 Interpretation TEST DATE: 7/22/1998	CHIRON HCV RNA bDNA 2.0 (MEq/mL*) TEST DATE: 7/16/1998	ORTHO ECi HBsAg TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½ TEST DATE: 11/16/2018
9048-01	+/-	-	-	-	<0.2	0.05	0.61	0.13
9048-02	1+	-	-	INDETERMINATE	<0.2	0.05	0.78	0.12
9048-03	+/-	-	-	-	<0.2	0.05	0.72	0.11
9048-04	1+	-	-	INDETERMINATE	<0.2	0.05	0.81	0.10
9048-05	2+	-	+/-	INDETERMINATE	<0.2	0.06	1.53	0.10
9048-06	2+	-	+/-	INDETERMINATE	<0.2	0.05	1.39	0.11
9048-07	3+	-	+/-	INDETERMINATE	<0.2	0.05	1.21	0.11
9048-08	1+	-	+/-	INDETERMINATE	<0.2	0.05	0.88	0.12

Notes on Data:

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.
 For Customer Support, please visit www.zeptometrix.com or email zepto.customerservice@antylia.com

PIHCV9048 Rev03
 Effective Date: 10/11/2021