



**HCV Seroconversion Panel Donor No. 64766**  
**Part Number: HCV9049**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | DATE OF DRAW | ALT<br>Normal Range<br>(10-45 u/L)<br><br>TEST DATE:<br>UNK | ABBOTT<br>MUREX<br>ANTI-HCV<br>4 0<br><br>TEST DATE:<br>UNK | ABBOTT<br>MUREX<br>HCV Ag/Ab<br>Combo<br><br>TEST DATE:<br>UNK | ABBOTT<br>ANTI-HCV<br>Lab # 1<br><br>TEST DATE:<br>5/20/1998 | ABBOTT<br>ANTI-HCV<br>Lab # 2<br><br>TEST DATE:<br>6/15/1998 | ABBOTT<br>ANTI-HCV<br>3.0<br><br>TEST DATE:<br>7/8/1998 | ORTHO<br>Chiron Riba<br>HCV<br>3.0<br><br>TEST DATE:<br>UNK | ABBOTT<br>PRISM<br>HCV<br><br>TEST DATE:<br>8/5/1998 |
|--------------|--------------|---|---|--|--|--|---|---|--|
| 9049-01      | 09/10/97     | 22  | 0.330   | 0.338  | 0.660  | 0.660  | 1.37  | 0.111   | 0.25   |
| 9049-02      | 09/12/97     | 16  | 0.911   | 0.345  | 1.927  | 2.060  | 0.84  | 0.303   | 0.92   |
| 9049-03      | 09/18/97     | 18  | 0.596   | 0.317  | 1.163  | 1.210  | 0.50  | 0.296   | 0.64   |
| 9049-04      | 09/22/97     | 12  | 0.407   | 0.317  | 0.781  | 0.793  | 4.17  | 0.124   | 0.40   |
| 9049-05      | 09/26/97     | 11  | 0.303   | 0.328  | 0.556  | 0.600  | 1.63  | 0.103   | 0.28   |

| PANEL MEMBER | ABBOTT<br>IMx<br>HCV<br>3.0<br>Reagent Pack<br><br>TEST DATE:<br>8/4/1998 | ABBOTT<br>AxSYM<br>ANTI-HCV<br>3.0<br>Reagent Pack<br><br>TEST DATE:<br>7/6/1998 | ABBOTT<br>Architect<br>ANTI-HCV<br><br>TEST DATE:<br>UNK | ABBOTT<br>PRISM<br>ANTI-HCV<br><br>TEST DATE:<br>UNK | ROCHE/<br>BOEHRINGER<br>MANNHEIM<br>Cobas Core<br>ANTI-HCV EIA<br><br>TEST DATE:<br>7/7/1998 | ORTHO<br>Enhanced<br>SAVe<br>HCV 3.0<br>Lab # 2<br><br>TEST DATE:<br>UNK | ORTHO<br>Enhanced<br>SAVe<br>HCV 3.0<br>Lab # 7<br><br>TEST DATE:<br>7/2/1998 | SANOFI<br>Monolisa<br>ANTI- HCV<br>PLUS<br>Lab # 4<br><br>TEST DATE:<br>6/30/1998 |
|--------------|---|--|--|--|--|--|---|---|
| 9049-01      | 0.28  | 0.35   | 0.390  | 0.388  | 0.3  | 0.140  | 0.412   | 1.09  |
| 9049-02      | 0.46  | 0.77   | 1.430  | 1.522  | 0.4  | 0.583  | 2.324   | 3.45  |
| 9049-03      | 0.32  | 0.63   | 0.870  | 0.881  | 0.3  | 0.489  | 0.882   | 2.21  |
| 9049-04      | 0.24  | 0.43   | 0.550  | 0.576  | 0.2  | 0.340  | 0.463   | 1.36  |
| 9049-05      | 0.27  | 0.27   | 0.380  | 0.433  | 0.2  | 0.160  | 0.370   | 1.06  |

| PANEL MEMBER | SANOFI<br>Monolisa<br>ANTI- HCV<br>PLUS Lab # 7<br><br>TEST DATE:<br>7/2/1998 | ORTHO<br>Chiron Riba<br>HCV 2.0<br>5-1-1(NS4)<br><br>TEST DATE:<br>UNK | ORTHO<br>Chiron Riba<br>HCV 2.0<br>C100(NS4)<br><br>TEST DATE:<br>UNK | ORTHO<br>Chiron Riba<br>HCV 2.0<br>C33(NS3)<br><br>TEST DATE:<br>UNK | ORTHO<br>Chiron Riba<br>HCV 2.0<br>C22(Core)<br><br>TEST DATE:<br>UNK | ORTHO<br>Chiron Riba<br>HCV 2.0<br>SOD<br><br>TEST DATE:<br>UNK | ORTHO<br>Chiron RIBA<br>HCV 2.0<br>Interpretation<br><br>TEST DATE:<br>UNK | ORTHO<br>Chiron Riba<br>HCV 3.0<br>c100(p)<br><br>TEST DATE:<br>UNK |
|--------------|---|--|---|--|---|---|--|---|
| 9049-01      | 0.507   | -  | -   | 1+   | -   | -   | INDETERMINATE  | -   |
| 9049-02      | 2.258   | -  | -   | 1+   | -   | -   | INDETERMINATE  | -   |
| 9049-03      | 1.268   | -  | -   | 1+   | -   | -   | INDETERMINATE  | -   |
| 9049-04      | 0.678   | -  | -   | 1+   | -   | -   | INDETERMINATE  | -   |
| 9049-05      | 0.241   | -  | -   | 1+   | -   | -   | INDETERMINATE  | -   |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV9049 Rev03  
 Effective Date: 10/11/2021



**HCV Seroconversion Panel Donor No. 64766  
Part Number: HCV9049**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

| PANEL MEMBER | ORTHO<br>Chiron Riba<br>HCV 3.0<br>c33c | ORTHO<br>Chiron Riba<br>HCV 3.0<br>c22(p) | ORTHO<br>Chiron Riba<br>HCV 3.0<br>NS5 | ORTHO<br>Chiron Riba<br>HCV 3.0<br>Interpretation | CHIRON<br>HCV RNA<br>bDNA 2.0<br>(MEq/mL*) | ORTHO<br>ECi<br>HBsAg    | ORTHO<br>ECi<br>ANTI-HCV | ORTHO<br>ECi<br>ANTI-HIV<br>½ |
|--------------|---|---|--|---|--|--------------------------|--------------------------|-------------------------------|
|              | TEST DATE:<br>UNK                       | TEST DATE:<br>UNK                         | TEST DATE:<br>UNK                      | TEST DATE:<br>7/22/1998                           | TEST DATE:<br>7/16/1998                    | TEST DATE:<br>11/16/2018 | TEST DATE:<br>11/16/2018 | TEST DATE:<br>11/16/2018      |
| 9049-01      | 2+                                      | -   | +/-                                    | INDETERMINATE                                     | <0.2                                       | 0.05                     | 0.39                     | 0.12                          |
| 9049-02      | 3+                                      | -   | +/-                                    | INDETERMINATE                                     | <0.2                                       | 0.04                     | 1.58                     | 0.10                          |
| 9049-03      | 3+                                      | -   | +/-                                    | INDETERMINATE                                     | <0.2                                       | 0.05                     | 0.73                     | 0.11                          |
| 9049-04      | 2+                                      | -   | +/-                                    | INDETERMINATE                                     | <0.2                                       | 0.05                     | 0.39                     | 0.12                          |
| 9049-05      | 1+                                      | -   | +/-                                    | INDETERMINATE                                     | <0.2                                       | 0.05                     | 0.23                     | 0.11                          |

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV9049 Rev03  
Effective Date: 10/11/2021