



**HCV Seroconversion Panel Donor No. 66732**  
**Part Number: HCV9055**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 3a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ABBOTT ANTI-HCV	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT ANTI-HCV 3.0	ALPHA HCV by PCR (NGI)	ABBOTT PRISM HCV
		TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/8/1999	TEST DATE: 6/16/1999	TEST DATE: 7/19/1999	TEST DATE: UNK	TEST DATE: 6/15/1999
9055-01	02/03/98	10	0.150	0.328	0.216	0.28	0.12	NEGATIVE	0.10
9055-02	02/05/98	9	0.148	0.341	0.200	0.27	0.12	NEGATIVE	0.09
9055-03	02/11/98	7	0.174	0.307	0.171	0.25	0.11	NEGATIVE	0.09
9055-04	02/27/98	8	0.130	0.310	0.171	0.21	0.10	NEGATIVE	0.07
9055-05	03/02/98	12	0.135	0.307	0.171	0.23	0.10	NEGATIVE	0.09
9055-06	03/06/98	7	0.130	0.369	0.167	0.19	0.11	POSITIVE	0.09
9055-07	03/09/98	8	0.126	6.076	0.176	0.24	0.11	POSITIVE	0.08
9055-08	03/13/98	13	0.124	6.848	0.180	0.20	0.12	POSITIVE	0.08
9055-09	03/18/98	28	0.204	6.100	0.194	0.38	0.10	POSITIVE	0.08
9055-10	03/21/98	30	0.156	3.707	0.178	0.31	0.33	POSITIVE	0.17
9055-11	04/09/98	56	2.006	2.410	0.561	1.39	0.35	POSITIVE	0.15

PANEL MEMBER	ABBOTT IMx HCV 3.0 Reagent Pack	ABBOTT AxSYM ANTI-HCV3.0 Reagent Pack	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV	ORTHO Enhanced SAve HCV 3.0 Lab # 5	SANOFI Monolisa ANTI- HCV PLUS Lab # 5	ORTHO Chiron Riba HCV 3.0 c100(p)/ 5-1-1
	TEST DATE: 6/23/1999	TEST DATE: 6/28/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 6/9/1999	TEST DATE: 6/7/1999	TEST DATE: 2/3/2000
9055-01	0.16	0.17	0.090	0.121	0.00	0.05	-
9055-02	0.13	0.20	0.090	0.107	0.00	0.05	-
9055-03	0.09	0.22	0.100	0.109	0.00	0.02	-
9055-04	0.09	0.26	0.090	0.104	0.00	0.09	-
9055-05	0.10	0.22	0.090	0.090	0.00	0.03	-
9055-06	0.11	0.21	0.080	0.117	0.00	0.05	-
9055-07	0.14	0.20	0.080	0.094	0.00	0.09	-
9055-08	0.15	0.22	0.080	0.097	0.00	0.04	-
9055-09	0.14	0.22	0.120	0.106	0.00	0.08	-
9055-10	0.18	0.28	0.080	0.099	0.00	0.04	-
9055-11	0.22	0.46	0.890	0.299	0.25	0.57	+/-

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg. Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)



**HCV Seroconversion Panel Donor No. 66732**  
**Part Number: HCV9055**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
 Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	ORTHO Chiron Riba HCV 3.0 c33c  TEST DATE: 2/3/2000	ORTHO Chiron Riba HCV 3.0 c22(p)  TEST DATE: 2/3/2000	ORTHO Chiron Riba HCV 3.0 NS5  TEST DATE: 2/3/2000	ORTHO Chiron RIBA HCV 2.0 Interpretation  TEST DATE: 2/3/2000	ORTHO ECi HBsAg  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HCV  TEST DATE: 11/16/2018	ORTHO ECi ANTI-HIV ½  TEST DATE: 11/16/2018
9055-01	-	-	-	-	0.11	0.03	0.12
9055-02	-	-	-	-	0.09	0.04	0.12
9055-03	-	-	-	-	0.09	0.03	0.12
9055-04	-	-	-	-	0.10	0.02	0.12
9055-05	-	-	-	-	0.12	0.03	0.12
9055-06	-	-	-	-	0.12	0.03	0.12
9055-07	-	-	-	-	0.13	0.03	0.12
9055-08	-	-	-	-	0.12	0.03	0.12
9055-09	-	-	-	-	0.11	0.04	0.12
9055-10	-	-	-	-	0.10	0.03	0.12
9055-11	+	1+	-	INDETERMINATE	0.11	2.25	0.12

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg. Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
 For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)