



**HCV Seroconversion Panel Donor No. 66647  
Part Number: HCV9058**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

**HCV Genotype: 1a**

PANEL MEMBER	DATE OF DRAW	ALT Normal Range (10-45 u/L)	ABBOTT MUREX ANTI-HCV 4.0	ABBOTT MUREX HCV Ag/Ab Combo	ALPHA HCV by PCR (NGI)	CHIRON bDNA	ABBOTT ANTI-HCV	ABBOTT Architect ANTI-HCV	ABBOTT PRISM ANTI-HCV
		TEST DATE: 3/26/1999	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: UNK	TEST DATE: 10/27/1999	TEST DATE: 6/4/1999	TEST DATE: UNK	TEST DATE: UNK
9058-01	03/06/98	19	0.303	10.972	POSITIVE	90.58	0.679	0.270	0.243
9058-02	03/09/98	18	0.313	11.524	POSITIVE	85.34	0.653	0.290	0.344
9058-03	03/13/98	55	0.457	10.583	POSITIVE	37.64	0.653	0.720	0.616
9058-04	03/16/98	75	1.185	10.852	POSITIVE	48.33	0.862	1.970	1.995
9058-05	03/20/98	156	2.900	12.293	POSITIVE	29.46	1.619	4.440	4.126

PANEL MEMBER	ORTHO Enhanced SAVE ANTI-HCV 3.0	ORTHO Chiron RIBA HCV 3.0 c100(p)/5-1-1	ORTHO Chiron RIBA HCV 3.0 c33c	ORTHO Chiron RIBA HCV 3.0 c22(p)	ORTHO Chiron RIBA HCV 3.0 NS5	ORTHO Chiron RIBA HCV 2.0 Interpretation	ORTHO ECi HBsAg	ORTHO ECi ANTI-HCV	ORTHO ECi ANTI-HIV 1/2
	TEST DATE: 2/29/2000	TEST DATE: 2/3/2000	TEST DATE: 2/3/2000	TEST DATE: 2/3/2000	TEST DATE: 2/3/2000	TEST DATE: 1/17/2000	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
9058-01	0.073	-	-	+/-	-	-	0.07	0.25	0.17
9058-02	0.078	-	-	+/-	-	-	0.05	0.20	0.15
9058-03	0.147	-	+/-	+	-	-	0.05	0.35	0.15
9058-04	0.660	-	1+	1+	-	+	0.05	1.63	0.15
9058-05	3.163	-	1+	2+	-	+	0.06	5.21	0.13

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ Corporation and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

**This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.**

ZeptoMetrix Corporation • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [custserv@zeptometrix.com](mailto:custserv@zeptometrix.com)