



**HCV Seroconversion Panel Donor No.1038851**  
**Part Number: HCV9095**

**Donor Demographics:** Donor source plasma was collected from a U.S. licensed blood collection facility.  
Donor was un-treated and asymptomatic throughout the collection period.

PANEL MEMBER	DATE OF DRAW	HCV	DIASORIN HBsAb	QUEST HCV Ab	ROCHE HCV Ampliscreen	ROCHE HCV Monitor	ORTHO Eci HBsAg	ORTHO Eci ANTI-HCV	ORTHO Eci ANTI-HIV 1/2
		TEST DATE: UNK	TEST DATE: 2/17/2010	TEST DATE: 2/25/2010	TEST DATE: 4/19/2010	TEST DATE: 4/20/2010	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018	TEST DATE: 11/16/2018
9095-01	06/03/2006	NEGATIVE	NOT TESTED	0.04	POSITIVE	159000	0.06	0.01	0.14
9095-02	06/06/2006	NEGATIVE	NOT TESTED	0.04	POSITIVE	54000	0.06	0.04	0.14
9095-03	06/13/2006	NEGATIVE	NOT TESTED	11.70	POSITIVE	141000	0.08	12.4	0.12
9095-04	06/24/2006	POSITIVE	NOT TESTED	18.10	POSITIVE	17500	0.15	17.7	0.14
9095-05	06/27/2006	POSITIVE	0.08	18.40	POSITIVE	42600	0.06	17.6	0.13

**Notes on Data:**

1. This panel is for **Research Use Only. Not for use in diagnostic procedures.**
2. Data is generated at ZeptoMetrix™ LLC and by our Laboratory Partners in the United States and Europe.
3. All specimens collected in this longitudinal series are unadulterated 4% sodium citrated plasma samples collected from a single donor in the United States.
4. No preservatives have been added.
5. All material has been tested and found negative for HBsAg, Anti-HIV1/2, HIV p24 Ag by FDA cleared tests.

ZeptoMetrix LLC • 25 Kenwood Circle, Franklin, MA 02038 • Tel (508) 553-5800 • Fax (508) 520-1525  
This Product was manufactured in a facility which has a Quality Management System that is ISO 13485 certified.  
For Customer Support, please visit [www.zeptometrix.com](http://www.zeptometrix.com) or email [zepto.customerservice@antylia.com](mailto:zepto.customerservice@antylia.com)

PIHCV9095 Rev02  
Effective Date: 10/11/2021